








-  **Warning: Please read all instructions before use. Do not use this product without proper instruction from a healthcare professional.**
-  **Warning: The 6990A Universal Raised Toilet Seat with Armrests has a maximum user weight limitation of 300 lbs.**
-  **Warning: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.**
-  **Warning: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.**
-  **Warning: Before use, ensure that Universal Raised Toilet Seat with Armrests is properly installed.**
-  **Warning: Inspect Universal Raised Toilet Seat with Armrests weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Graham-Field Health Products dealer for repair and/or appropriate replacement parts.**

### Assembly Instructions

1. Remove toilet seat and lid from toilet bowl.
2. Install metal restraint bar (see picture at right): Place the mounting bolts from the toilet lid assembly through the holes in the metal restraint bar and slide a nylon spacer onto each bolt. Re-attach the toilet lid (now fitted with the metal restraint bar and nylon spacers) to the toilet fixture by engaging the bolts.
3. Prepare the Universal Adjustable Raised Toilet Seat with Armrests for installation on the toilet (see picture at right): Position the armrest structure over the toilet seat as shown so that the free ends are positioned closest to the L-shaped bars (L-bars) mounted at the rear of the toilet seat. Use the six metal screws provided to connect the armrest structure to the toilet seat (this will require two people).

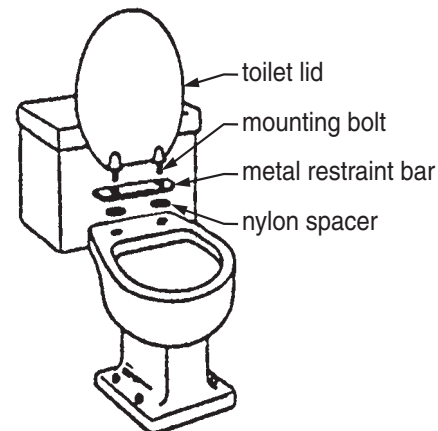

**install metal restraint bar and toilet lid**



## Installation Instructions

1. Place the **Universal Adjustable Raised Toilet Seat with Armrests** on the toilet bowl and secure the unit by ensuring that the rear L-bars pass under the metal restraint bar installed beneath the **Universal Adjustable Raised Toilet Seat with Armrests**. Slight inward pressure may be necessary to engage the front brackets over the edge of the toilet bowl.



**Warning: For safe use, it is essential that rear L-bars pass under the metal restraint bar.**

2. If the front brackets do not fit over the rim of the toilet bowl, add an appropriate number of small nylon washers to the existing nylon washers between the front leg and bracket and repeat Step 1 of the Installation Instructions.
3. To remove the **Universal Adjustable Raised Toilet Seat with Armrests** from the toilet, pull upward on the front side of the armrests.

## Instructions for Use on Shorter, Non-Institutional -Type toilet Bowls:

1. Remove the rear L-bars from the **Universal Adjustable Raised Toilet Seat with Armrests**, remove the appropriate number of nylon spacers, and re-attach the rear L-bars. Test to ensure that the rear L-bars pass under the metal restraint bar, and that the front brackets fit securely over the edge of the toilet bowl.



**Warning: For safe use, it is essential that rear L-bars pass under the metal restraint bar.**

## Maintenance and cleaning

Check weekly that **Universal Adjustable Raised Toilet Seat with Armrests** is securely installed. To clean, wash **Universal Adjustable Raised Toilet Seat with Armrests** weekly with soap and water or a mild household detergent and rinse with clear water. Never use abrasive cleansers or scouring pads on any part of **Universal Adjustable Raised Toilet Seat with Armrests**.



## WARRANTY

Lifetime limited warranty. For additional warranty information, please contact your Graham-Field Health Products dealer.

Graham-Field Health Products  
Atlanta, Georgia 30360 U.S.A.

