



**⚠ CAUTION:** Before using the recliner, slide the anti-tip wheels all the way to the rear of the chair as shown. Tighten nuts with enclosed wrench to secure brackets in place.

**Clamp Installation:**

1. Position chair into semi-recline position.
2. Detach lower rear end of side panel rod from chair frame by pulling rod toward rear exposing side rail.
3. Place clamp onto side rail with tube side in and slide into place as far to the rear as possible. Insert bolts (head side out).
4. Hold spacer tubes in place between sides of clamp. Tighten lock nuts with wrench.
5. Make certain that square end of bolt shank is fully seated in square hole on clamp.
6. Make sure the tip wheel is as far to the rear of the chair as possible.

