

Installation tools and supplies: Level, screwdriver, one #10 x 1 1/2" sheet metal screw, one #10 drywall anchor and screw

Fuse replacement: Phillips screwdriver

⚠ WARNING: Important! Read and understand these instructions before installing the X-Ray Illuminator. If you do not understand any part of these instructions, contact your medical supply distributor for direction in the use of this product. If the X-Ray Illuminator is not properly installed, personal injury and/or damage to the X-Ray Illuminator could result.

⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.

⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation or use of this product.


INSTALLATION (ON WALL OR TABLE)

⚠ WARNING: Locate unit so that electrical cord reaches the power outlet and will not be stepped on, tripped over, or otherwise subjected to damage or stress. Do not use this unit with an extension cord.




INSTALLATION ON WALL

⚠ WARNING: The X-Ray Illuminator mounts on two screws which are installed in the wall. You must mount at least one screw in a wall stud; otherwise, the X-Ray Illuminator could fall, resulting in damage to the X-Ray Illuminator and/or personal injury.

1. The one-bank X-Ray Illuminator is shown at right. Open the front cover of the X-Ray Illuminator to access the mounting slots in the back wall: Grasp the upper corners at the **OPEN** labels and pull forward. Rotate the front of the X-Ray Illuminator forward carefully and let it hang.
2. Find the wall stud where you will mount one side of the X-Ray Illuminator (see preceding warning). Hold the X-Ray Illuminator, with level on top, up to the wall to find the desired mounting height. Place one of the mounting slots directly over the wall stud.

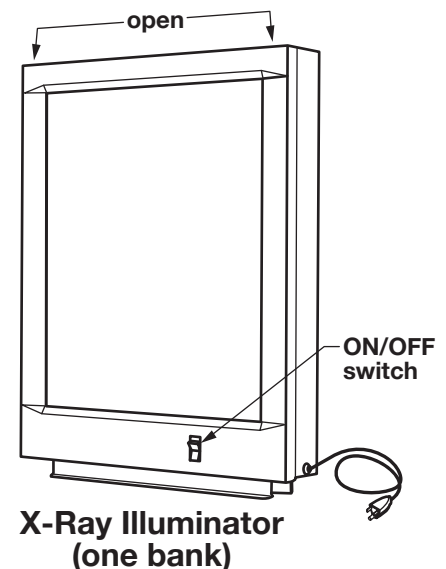

3. Mark in the center of both slots, at the locations shown in the picture at right. Remove the X-Ray Illuminator.
4. Install the sheet metal screw in the marked location in the wall stud. Use screwdriver to secure, but not completely.
5. Install the drywall anchor and screw in the other marked slot (per the drywall anchor and screw mounting instructions). Use screwdriver to secure, but not completely.
6. Lift the X-Ray Illuminator and place the back flat against the wall with large round holes over screw heads. Slide X-Ray Illuminator down carefully so that tops of slots are resting on screws.
7. Use the screwdriver to tighten both screws securely.

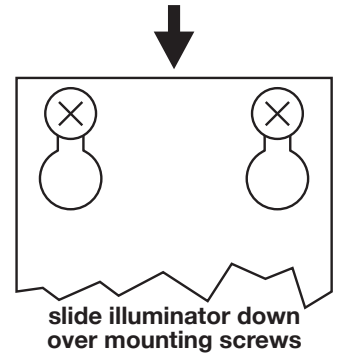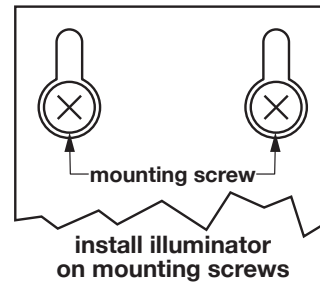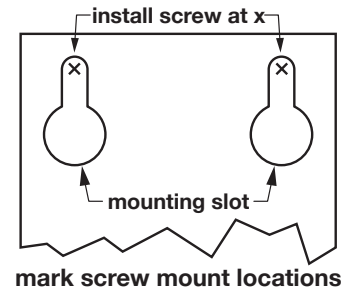

INSTALLATION ON TABLE

Set the X-Ray Illuminator on a sturdy table, ensuring that its base is flat and stable.

OPERATION

⚠ WARNING: This device is equipped with a three-prong (grounding) plug for protection against possible shock hazard. Plug only into a properly grounded three-prong wall receptacle.

1. Plug the X-Ray Illuminator into a properly grounded power outlet.
2. Turn the X-Ray Illuminator ON by pressing the ON/OFF switch on front of unit.
3. Slide X-Ray up into holder on top front of unit.

MAINTENANCE

⚠ WARNING: Always unplug this device from the wall receptacle before replacing light bulbs or fuse; otherwise, shock or electrocution could occur.

Replacing fluorescent tubes: Replace with T-8 15W Medium Bi-Pin 18" (one-bank requires three tubes, two-bank requires six tubes, three-bank requires nine tubes).

Replacing fuse: Replace fuse with same size and type. Use screwdriver to remove fuse holder. Remove old fuse and replace with new fuse. Replace fuse holder. Use screwdriver to secure.

WARRANTY

GF Health Products, Inc. warrants the Grafco X-Ray Illuminator for a period of ninety days for defects in workmanship and materials. During the warranty period, defective items will be repaired or replaced at manufacturer's option at no charge.



www.grahamfield.com

GF, Graham-Field and Grafco are registered trademarks of GF Health Products, Inc.

© GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601 • www.grahamfield.com