

LIFETIME LIMITED WARRANTY

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. We are dedicated to providing products that will give our customers years of dependable service.

This Lumex product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This warranty does not cover product failures due to normal wear and tear or device misuse. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized Lumex Dealer.



All illustrations and specifications in this manual are based on the latest product information available at the time of publication. Graham-Field reserves the right to make changes at anytime, without notice. © 2005, Graham-Field Health Products, 2935 Northeast Parkway, Atlanta, Georgia 30360. Printed in China. Doc Number: 616270A 05/04

LUMEX®

For The Quality Of Life®



INSTRUCTIONS & WARRANTY INFORMATION

Dual Release Folding Walker
With Wheels
616270A, 616370A, 616470A, 616570A

IMPORTANT: READ BEFORE USE

INSTRUCTIONS FOR USE –

PLEASE READ ALL INSTRUCTIONS BEFORE USE.

The chart below can be used as a guide to select a height-appropriate walker. Please consult a healthcare professional for the proper adjustment.

DUAL RELEASE FOLDING WALKER SIZE SELECTION CHART

Model Number	Size	User Weight	Adjustment Range/ Handgrip to Floor
616270A-4	Adult	5'4"- 6'2"	34 1/2"- 41 1/2"
616370A-4	Junior	4'7"- 5'3"	29 1/2"- 34 1/2"
616470A	Adult	5'4"- 6'2"	33 1/2"- 40 1/2"
616570A	Junior	4'7"- 5'3"	28 1/2"- 33 1/2"

Leg extensions are available for users over 6' 2".
Please consult your Lumex dealer.

HEIGHT ADJUSTMENT

NOTE: For ease of shipping, the walker is shipped in its lowest height position. The walker should be adjusted to the desired height before operation.

Refer To Illustration Below:

The legs of each walker are equipped with an adjustment button and an adjustable leg attachment with eight adjustment positions.

1. Adjust the height of the walker by aligning all four leg attachments to the same position height.
2. To adjust the height of a leg attachment, depress the adjustment button and twist the leg attachment to the desired height.



TO OPEN UNIT

When removed from box, walker is in a closed position.

1. Grasp one handgrip and pull away from center of walker.
2. Repeat with other handgrip.
3. Push handgrips away from each other until the release mechanisms click into place.

TO CLOSE UNIT

1. Push downward on yellow release button and hold.
2. Grab nearest handgrip and push toward center of walker.
3. Repeat for other handgrip.

MAINTENANCE

- Wipe walker weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent. DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the walker.
- Check all hardware weekly to ensure that it is securely attached.
- Replace walker tips as needed.
- To guarantee the best service from your walker, always specify Lumex replacement parts, available from your Lumex dealer.

SAFETY GUIDELINES – PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 300 lb.
3. Before using, be sure that walker is locked in the open position.
4. Lumex assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
5. If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.
6. Inspect walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
7. Before use, ensure that push button protrudes fully through height adjustment hole and the release mechanisms are locked in the "open" position.
8. Always use caution when using this product on wet or slippery surfaces.