JOHN BUNN®
A Graham-Field Brand

Optium®

JB0180 / JB0180S OXYGEN CONCENTRATOR USER MANUAL

Important: Do not use the Oxygen Concentrator without first reading and understanding this manual! Save this manual for future use.

⚠ Federal Law (USA) restricts this device to sale by or on the order of a physician. This product should not be used unless the operator has been instructed by a qualified healthcare professional.

CONTENTS

1	INTRODUCTION	3
	INTENDED USE OF THIS DEVICE	3
2	SAFETY INFORMATION.....	4
	DANGER:	4
	WARNING:	5
	CAUTION:	5
3	HANDLING.....	6
	GF HEALTH PRODUCTS, INC. FREIGHT POLICY.....	6
	APPARENT LOSS OR DAMAGE.....	6
	CONCEALED LOSS OR DAMAGE.....	6
	UNPACKING	6
	INSPECTION.....	6
	STORAGE	6
4	INSTALLATION	7
	INSTALLATION LOCATION SAFETY INFORMATION.....	7
	MAINTENANCE.....	8
	RADIO FREQUENCY INTERFERENCE	8
	PRODUCT DESCRIPTION	9
	CONCENTRATOR ILLUSTRATION	9
5	OPERATION	10
	SETUP	10
	TURNING THE CONCENTRATOR ON	10
	CIRCUIT BREAKER (RESET BUTTON)	11
	TURNING THE CONCENTRATOR OFF.....	11
	OPERATION WITH A HUMIDIFIER BOTTLE (IF SO PRESCRIBED).....	12
	LED INDICATOR TABLE.....	13
6	TRANSPORTING THE OXYGEN CONCENTRATOR	14
	USING THE LUGGAGE HANDLE.....	14
	SAFETY DURING TRANSPORT	14
7	MAINTENANCE.....	15
	REPLACEMENT PARTS	15
	USER MAINTENANCE	15
	CLEANING THE CABINET FILTER	15
	CLEANING THE CABINET	16
	CLEANING THE (OPTIONAL) HUMIDIFIER BOTTLE	16
	PREVENTIVE MAINTENANCE RECORD	17
8	TROUBLESHOOTING.....	18
	TROUBLESHOOTING TABLE.....	19
9	SPECIFICATIONS	20
10	LIMITED WARRANTY	21
11	INDEX	23

Graham-Field, John Bunn, and Optium are registered trademarks of GF Health Products, Inc.

Packaging, warranties, products, and specifications are subject to change without notice. GF Health Products, Inc. is not responsible for typographical errors.

⚠ Federal Law (USA) restricts this device to sale by or on the order of a physician. This product should not be used unless the operator has been instructed by a qualified healthcare professional.

⚠ WARNING: Users who require continuous oxygenation must plan for alternate reserve sources of power and oxygen in the event of a failure or loss of power and oxygen. This device is to be used ONLY as an oxygen supplement device and is NOT intended to be a life-supporting or life-sustaining device.

1 INTRODUCTION

This user manual contains important information and safety precautions for the John Bunn® Optium® Oxygen Concentrator.

Before using your Oxygen Concentrator, please read and understand this entire user manual. Take special note of all safety precautions marked DANGER and WARNING. Save this user manual for future reference.

Note: The most current version of this manual can be found online at www.grahamfield.com.

INTENDED USE OF THIS DEVICE

The John Bunn Optium Oxygen Concentrator is intended for use as an oxygen supplement device in a home or care facility. Use the concentrator only as prescribed by a physician. This user manual serves as a reference for your oxygen concentrator.

⚠ WARNING: THIS DEVICE IS NOT INTENDED FOR LIFE SUPPORT. Power interruption could prevent the device from operating.

There are different accessories that can be used with an oxygen concentrator. Such accessories include oxygen tubing, humidifiers, cannulas and masks. Your physician or provider will recommend the different types of accessories to use with your oxygen concentrator. John Bunn accessories are available and recommended for use with the John Bunn Optium Oxygen Concentrator.

⚠ WARNING: Failure to observe the safety statements in this manual can result in severe fire, property damage, physical injury, and/or DEATH.

⚠ WARNING: Under certain diagnoses, oxygen therapy can be hazardous; use this concentrator *only* as prescribed by a physician.

⚠ WARNING: To ensure user safety, use this device only with accessories recommended by Graham-Field, your Graham-Field authorized distributor, your physician, or your respiratory therapist.

2 SAFETY INFORMATION

The safety statements presented in this chapter refer to the basic safety information that the operator of the oxygen concentrator shall pay attention to and abide by. There are additional safety statements in other chapters or sections, which may be the same as or similar to the following, or specific to the operations. Please note the following special statements, used throughout this manual, and their significance:

⚠ DANGER: Indicates an imminent hazard situation that, if not avoided, will result in death or serious personal injury.

⚠ WARNING: Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.

▲ Caution: Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury or product/property damage.

Note: Provides application recommendations or other useful information to ensure that you get the most from your product.

DANGER:

To reduce the risk of Electrocution, Fire and/or Explosion

⚠ DANGER: Do not use this device while bathing. If continuous usage is required by the direction of a physician, the concentrator must be located in another room at least 9 feet (2.8m) from the bath.

⚠ DANGER: Do not place or store this device where it can fall or be pulled into a tub or sink.

⚠ DANGER: Do not place this device in or drop it into water or other liquid.

⚠ DANGER: Do not reach for a device that has fallen into water—unplug it immediately!

⚠ DANGER: DO NOT come in contact with the concentrator when you are wet.

⚠ DANGER: The oxygen concentrator contains no serviceable parts—do not open the cabinet! Opening of the concentrator cabinet by anyone other than authorized service personnel could void the warranty and create a hazardous condition, which could result in death or serious personal injury.

⚠ DANGER: Avoid creation of any spark near medical oxygen equipment. This includes sparks from static electricity created by any type of friction.

WARNING:

To reduce the risk of burns and/or serious personal injury

- ⚠ **WARNING:** Federal Law (USA) restricts this device to sale by or on the order of a physician.
- ⚠ **WARNING:** Use this device only for its intended use as described in this manual.
- ⚠ **WARNING:** This device is to be used only in accordance with the prescription of a physician and this user manual. **NEVER** make adjustments to the flow rate unless prescribed by a physician.
- ⚠ **WARNING:** If at any time you feel that you are not receiving the correct amount of oxygen, contact your provider and/or physician immediately.
- ⚠ **WARNING:** **DO NOT** allow any powder or powder-like substance near the oxygen concentrator and/or its tubing; the powder could cause contamination of the device, and the possible resulting device malfunction could cause oxygen to drop to ineffective levels.
- ⚠ **WARNING:** Supervise use of this device closely when it is used by, on, or near children or those who require extra care.
- ⚠ **WARNING:** Always unplug this device immediately after use.
- ⚠ **WARNING:** Do not leave this device unattended when it is plugged in.

CAUTION:

To reduce the risk of minor personal injury and/or product/property damage

- ▲ **Caution:** During transport, always keep the concentrator upright to prevent damage. If the concentrator is transported in the trunk of a car, ensure that the concentrator is secured so that it will remain upright.
- ▲ **Caution:** Ensure that the temperature at the concentrator's location during transport does not fall below -40°F (-40°C) or exceed 158°F (70°C).

3 HANDLING

GF HEALTH PRODUCTS, INC. FREIGHT POLICY

For your protection, read carefully

The concentrator should ALWAYS be kept in the upright position to prevent damage while being transported. The Optium packaging has been designed to assure maximum protection of the concentrator.

The carrier accepted this merchandise "in good condition" and is responsible for safe delivery. Before signing the freight bill, inspect the shipment carefully for damage or missing pieces.

Apparent loss or damage

Should visual inspection show loss or damage, this MUST be noted on the freight bill and signed by the carrier's agent. Failure to do so may result in the carrier failing to honor the claim. Please contact the carrier to obtain the paperwork necessary to file a claim or contact Graham-Field Customer Service at the number on the back cover.

Concealed loss or damage

If damage is discovered after delivery is made, a concealed damage claim must be entered with the freight carrier. When this occurs, make a written request to the carrier for inspection. This request for inspection must be made within 15 days of delivery. The carrier will provide all paperwork necessary to file a concealed damage or loss claim, since such damage or loss is the carrier's responsibility.

UNPACKING

Note: Unless the oxygen concentrator is to be used immediately, retain containers and packing materials for storage until concentrator use is required.

1. Check for obvious damage to the carton or its contents. If damage is evident, please notify the carrier and your Graham-Field authorized distributor.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.

Inspection

Examine exterior of the oxygen concentrator for nicks, dents, scratches or other damages. Inspect all components.

STORAGE

Store the repackaged oxygen concentrator in a dry area. DO NOT place anything on top of the repackaged concentrator.

▲ **Always store the concentrator upright to prevent damage.**

▲ **Caution: Ensure that the temperature at the concentrator's location during storage does not fall below -40°F (-40°C) or exceed 158°F (70°C).**

4 INSTALLATION

INSTALLATION LOCATION SAFETY INFORMATION

Select a *well-ventilated, fume-free* room in your house where using the oxygen concentrator would be most convenient. The concentrator can be easily moved from room to room on its casters.

To Reduce the Risk of Electrocution, Fire, Explosion Burns, Serious Personal Injury, and/or Property Damage

- ⚠ **WARNING:** Keep the oxygen concentrator away from flammable and explosive areas.
- ⚠ **WARNING:** To reduce the risk of fire, special care must be taken when using an oxygen concentrator. **DO NOT SMOKE** while using this device. Keep all matches, lighted cigarettes, and other sources of ignition out of the room in which this concentrator is located. In public settings or when used in hospital or long-term care environments, prominently display **NO SMOKING** signs.
- ⚠ **WARNING:** A spontaneous and violent ignition could occur if oil, grease, or greasy substances come in contact with oxygen under pressure. **ALWAYS** keep these substances away from the oxygen concentrator and all other oxygen equipment.
- ⚠ **WARNING:** Textiles and materials that normally would not easily burn can ignite and burn with great intensity in oxygen-enriched air; **ALWAYS** keep these materials away from the oxygen concentrator and all other oxygen equipment.
- ⚠ **WARNING:** The oxygen concentrator is intended to be rolled from room to room. Do not place the oxygen concentrator in surroundings where its airflow is obstructed.
- ⚠ **WARNING:** Be certain to place the concentrator so that all sides are at least 12 inches (30cm) away from walls, draperies, furniture, or other obstructions. Do not place the concentrator in a confined area.
- ⚠ **WARNING:** The oxygen concentrator **MUST** be kept away from heat, fire, flammable and explosive areas, and excessive water sources and conditions. Keep all oxygen tubing, cords and oxygen concentrator away from heated or hot surfaces, including space heaters, blankets, stoves and similar electrical appliances.
- ⚠ **WARNING:** Do not use a household extension cord if the AC cord does not reach the power outlet. Use of an extension cord could result in fire and electric shock.
- ⚠ **WARNING:** **DO NOT** connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.
- ⚠ **WARNING:** **DO NOT** plug the concentrator into the same outlet as any other major appliance.

- ⚠ **WARNING:** Always place the concentrator on a hard surface. Never place the concentrator on a surface such as a bed or couch, where the concentrator could tip or fall.
- ⚠ **WARNING:** Do not place items on top of the concentrator.
- ⚠ **WARNING:** NEVER block the concentrator's air openings.
- ⚠ **WARNING:** Keep the concentrator openings free from lint, hair, etc.
- ▲ **Caution:** The concentrator should be appropriately placed to keep it from falling, strong vibration or other mechanical damage.
- ▲ **Caution:** Only use stable and safe electrical power sources.
- ▲ **Caution:** Do not pull the oxygen concentrator by the power cord.
- ▲ **Caution:** If the oxygen concentrator has been subjected to sub-freezing temperatures for an extended period of time, allow it to warm up to the stated operating temperatures before power up. Failure to do so could result in improper performance and or alarm conditions until the device reaches normal operating temperatures.

MAINTENANCE

Proper care and maintenance are essential to keep your oxygen concentrator in safe working condition. When you believe that a component or part of your concentrator is not functioning properly, contact your Graham-Field authorized distributor immediately, as a potentially hazardous condition could result. Only excellent condition is acceptable where safety is concerned.



RADIO FREQUENCY INTERFERENCE

Most electronic equipment is influenced by Radio Frequency Interference (RFI). ALWAYS exercise caution with regard to the use of portable communication equipment in the area around the oxygen concentrator.

PRODUCT DESCRIPTION

Concentrator Illustration

Concentrator features are shown below.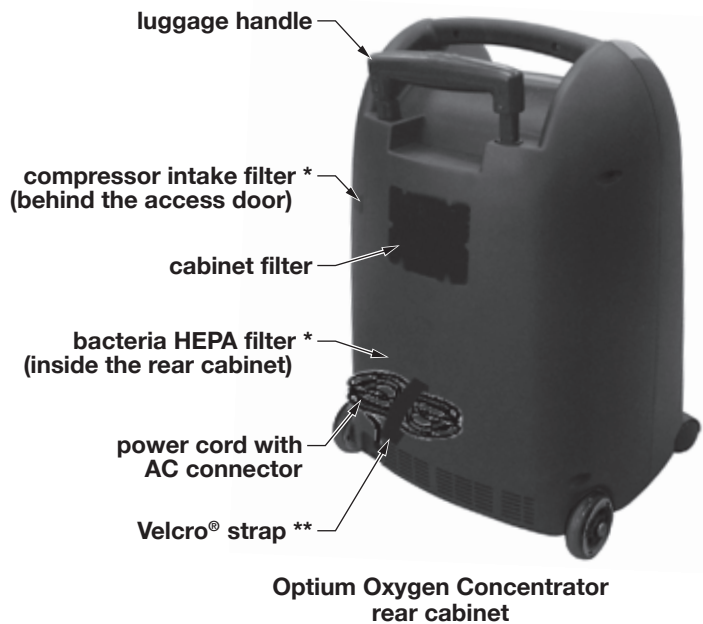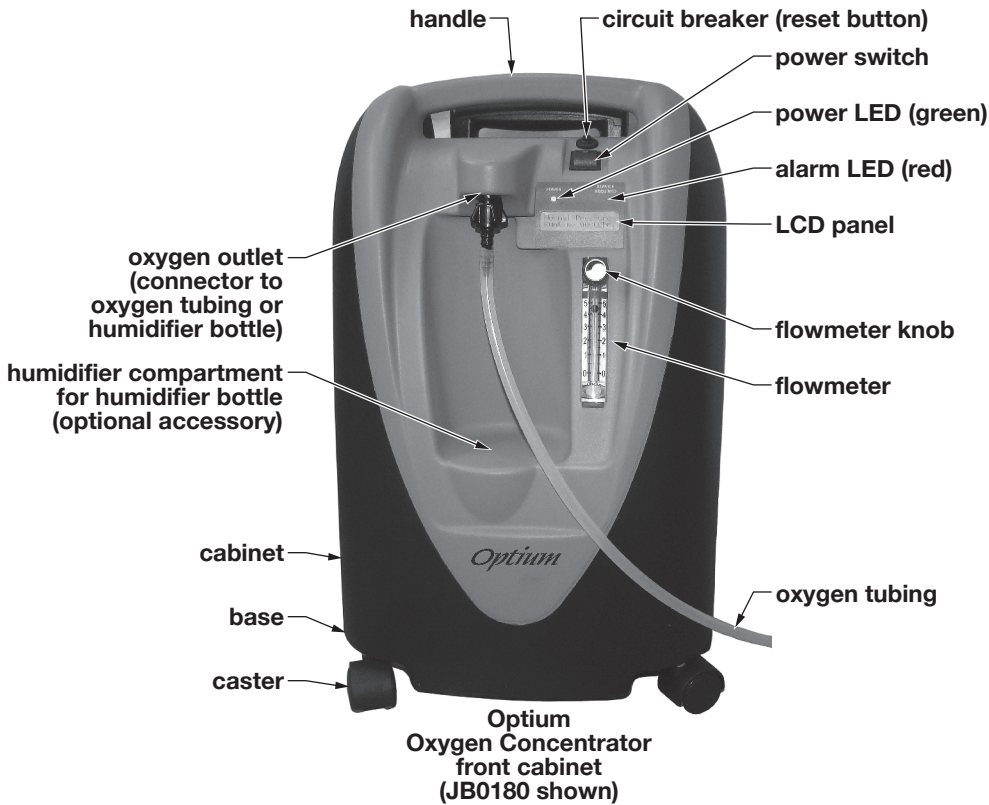

* Service to be performed by authorized service personnel ONLY

** Velcro is a registered trademark of Velcro Industries B.V. Corporation

5 OPERATION

⚠ WARNING: Read the safety section and operating instructions in this manual before use.

SETUP

1. Before proceeding, ensure that all instructions in Section 4, Installation, have been followed, and that you have read the SAFETY section of this manual.
2. Ensure that the concentrator power switch is in the OFF position. Plug the concentrator's AC connector into a properly grounded power outlet.

⚠ WARNING: As a safety feature, this device has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit completely in the outlet, reverse the plug. DO NOT attempt to defeat this safety feature. Contact a qualified electrician or your Graham-Field authorized distributor if the plug does not fit in the polarized outlet.


TURNING THE CONCENTRATOR ON

*Note: For use with a humidifier bottle, please see **OPERATION WITH A HUMIDIFIER BOTTLE (IF SO PRESCRIBED)** on the following page.*

1. Connect the oxygen tubing securely to the oxygen outlet; connect the other end to the patient's cannula or mask outlet (these may already be connected).
2. Move the concentrator's ON/OFF switch to the ON position to turn on concentrator power.

Model JB0180: The green (power) LED will illuminate.

Model JB0180S: the yellow low oxygen LED will illuminate for approximately three minutes while the system performs an Oxygen Purity test, then the green (power) LED and appropriate O₂ LED will illuminate.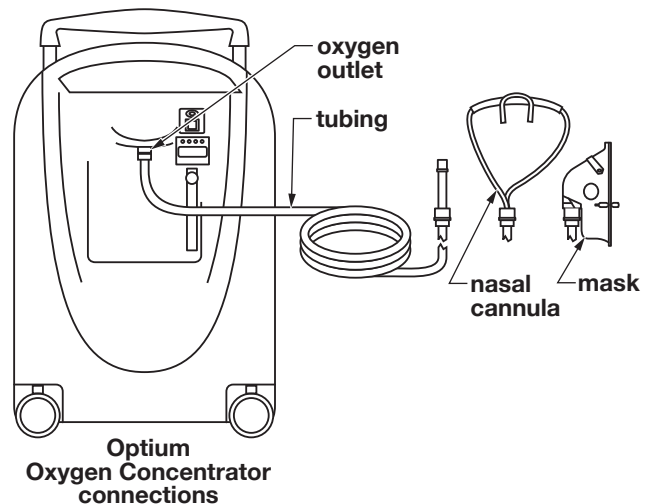

Ensure that oxygen is flowing through the tubing before proceeding.

Note: If the oxygen outlet is blocked or if the flowmeter is set to <0, the Optium will alarm and the LCD will display "NO FLOW".

⚠ WARNING: If the flow rate on the flowmeter ever falls below 0.5 LPM, check tubing or accessories for blocked or kinked tubing or for a defective humidifier bottle.

3. Turn the flowmeter knob to center the flowmeter ball on the LPM prescribed. The flowmeter can be adjusted only after the unit is powered on.

⚠ WARNING: Do not change LPM setting unless prescribed by a physician.

Note: Oxygenation times and flow rate ranges are prescribed by your physician.

CIRCUIT BREAKER (RESET BUTTON)

Your oxygen concentrator is equipped with a circuit breaker (reset button), to protect the electrical circuits from overload, next to the power switch. In the event of an overload, the circuit breaker button will pop out. To reset the circuit breaker, push the button in.

TURNING THE CONCENTRATOR OFF

After using the concentrator for the time prescribed by your physician, move the concentrator's ON/OFF switch to the OFF position to turn off concentrator power. Unplug the concentrator's AC connector from the power outlet.

⚠ WARNING: Never operate this device if it has a damaged cord or plug; is not working properly; or has been dropped, damaged, or dropped into water. Return the device to your Graham-Field authorized distributor for examination and repair.


OPERATION WITH A HUMIDIFIER BOTTLE (IF SO PRESCRIBED)

⚠ **WARNING:** The use of a humidifier bottle with a concentrator must be prescribed by a physician.

⚠ **WARNING:** DO NOT overfill the humidifier bottle.

⚠ **WARNING:** DO NOT reverse the oxygen inlet and output connections, or water from the humidifier bottle will travel through the oxygen hose back to the patient.

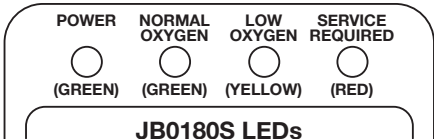
1. Unscrew the humidifier bottle cover.
2. Fill the humidifier bottle with purified or distilled water to a point between the maximum and minimum water level lines. If needed, add medication to the water as prescribed by your physician.
3. Replace the cover on the humidifier bottle and tighten it securely.
4. Place the humidifier bottle in the oxygen concentrator's humidifier compartment, attaching the humidifier bottle cover nut to the oxygen outlet.

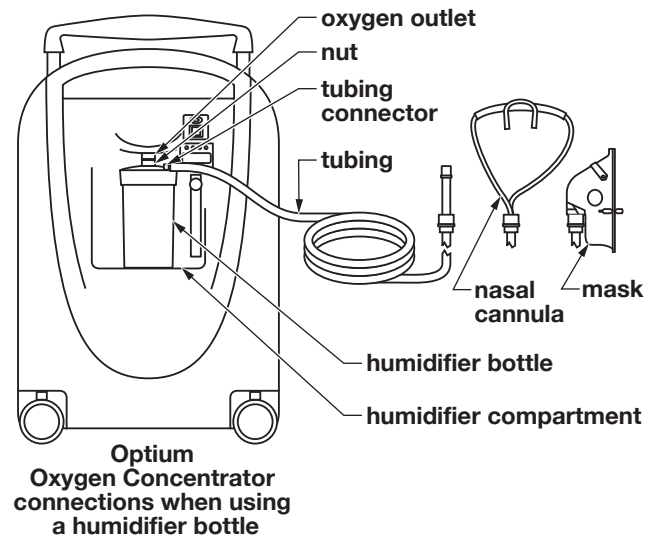5. Turn the nut *counter-clockwise* to secure it to the oxygen outlet. **Ensure that the humidifier bottle cover nut is securely attached to the oxygen outlet before continuing.**
6. Attach oxygen tubing from the humidifier bottle to the patient's cannula or mask outlet (these may already be connected).
7. Ensure that the power switch is in the OFF position. Plug the concentrator's AC connector into a properly grounded power outlet.
8. Turn the concentrator on: See the previous section, **TURNING THE CONCENTRATOR ON**, and follow steps 2-3.

LED INDICATOR TABLE

The following LED Indicator Table lists LED indicator patterns and causes so that you can become familiar with the concentrator's normal indicators. If you are having a problem with the concentrator, please see Section 8, **TROUBLESHOOTING**, where there is a list of symptoms and actions.

Model JB0180 only	 <p style="text-align: center;">JB0180 LEDs</p>		
	Indicator	LED(s) and/or Audible Alarm	Probable Cause
	Power	POWER LED illuminates nonflashing	Power is on
	Alarm	SERVICE REQUIRED LED illuminates; audible alarm sounds	1)Power failure 2)Concentrator failure 3)High pressure 4)Low pressure 5)O ₂ < 72% - do not use concentrator, O ₂ too low Contact your GF authorized distributor
Alert	SERVICE REQUIRED LED illuminates; intermittent audible alarm sounds	No oxygen flow	
Model JB0180S only	 <p style="text-align: center;">JB0180S LEDs</p>		
	Indicator	LED(s) and/or Audible Alarm	Probable Cause
	Power	POWER LED illuminates nonflashing	Power is on
	Alarm	SERVICE REQUIRED LED illuminates; audible alarm sounds	1)Power failure 2)Concentrator failure 3)High pressure 4)Low pressure 5)O ₂ < 72% - do not use concentrator, O ₂ too low Contact your GF authorized distributor
	Alert	SERVICE REQUIRED LED illuminates; intermittent audible alarm sounds	No oxygen flow
	O ₂ Indicator	NORMAL OXYGEN LED illuminates	O ₂ > 85%
		LOW OXYGEN LED illuminates; SERVICE REQUIRED LED flashes every two seconds; audible alarm sounds	O ₂ > 72%, < 85%
LOW OXYGEN LED illuminates for approximately 3 min. at startup; then POWER LED and appropriate O ₂ LED illuminate		O ₂ Purity Test performs automatically at startup	

6 TRANSPORTING THE OXYGEN CONCENTRATOR

The Optium Oxygen Concentrator is equipped with a luggage handle and casters for transport. To transport the concentrator:

1. Remove and store any accessories, such as tubing, cannulas, masks, and humidifier bottle, if so equipped.
2. Fold and wrap the power cord and attach it to the concentrator cabinet using the Velcro strap, as shown below.

USING THE LUGGAGE HANDLE

Extend the luggage handle: Pull the handle up completely. Ensure that the handle is completely extended and securely in place.

Retract the luggage handle: Lower the handle completely. Ensure that the handle is completely retracted and securely in place.

▲ Caution: Do not lift oxygen concentrator by the luggage handle.

SAFETY DURING TRANSPORT

Stow the oxygen concentrator upright in a safe place, observing the following:

- ⚠ **WARNING: Keep the oxygen concentrator away from flammable and explosive areas.**
- ⚠ **WARNING: To reduce the risk of fire, DO NOT SMOKE while transporting this device. Keep all matches, lighted cigarettes, and other sources of ignition out of the vehicle or room in which this concentrator is located.**
- ⚠ **WARNING: A spontaneous and violent ignition could occur if oil, grease, or greasy substances come in contact with oxygen under pressure. ALWAYS keep these substances away from the oxygen concentrator and all other oxygen equipment.**
- ⚠ **WARNING: Textiles and other materials that normally would not easily burn can ignite and burn with great intensity in oxygen-enriched air; ALWAYS keep these materials away from the oxygen concentrator and all other oxygen equipment.**
- ▲ **Caution: During transport, always keep the concentrator upright to prevent damage. If the concentrator is transported in the trunk of a car, ensure that the concentrator is secured so that it will remain upright.**
- ▲ **Caution: Ensure that the temperature at the concentrator's location during transport does not fall below -40°F (-40°C) or exceed 158°F (70°C).**

7 MAINTENANCE

Note: There is a Preventive Maintenance Record on the next page. When service is performed on your concentrator, please make this record available to the service personnel performing the service, to ensure that accurate maintenance records are kept.

*Note: All maintenance except **Cleaning the cabinet filter**, **Cleaning the cabinet**, and **Cleaning the (optional) humidifier bottle**, described in those sections directly following, is to be performed only by authorized service personnel.*

REPLACEMENT PARTS

Replacement parts and accessories are available through your Graham-Field authorized distributor. For a list of parts, please visit www.grahamfield.com.

USER MAINTENANCE

- ⚠ **DANGER:** The concentrator was specifically designed to minimize routine preventive maintenance. Preventive maintenance or adjustments to the concentrator are to be performed only by authorized service personnel; the performance of such maintenance or adjustment by any other personnel could void the warranty and create a hazardous condition, which could result in death or serious personal injury.
- ⚠ **WARNING:** Ensure that the concentrator is unplugged from the AC power source and the power switch is in the OFF position before performing maintenance on the concentrator.

Cleaning the cabinet filter

Clean the cabinet filter weekly, or more often if it gets dirty or your environment has high dust or pollutant levels.

1. Ensure that the power cord is unplugged and the power switch is in the OFF position.
2. Remove the filter carefully from the rear of the cabinet (see picture at right).
3. Vacuum the filter *carefully* or wash it with a mild detergent and tap water. If washed, allow the filter to air dry before reinstalling it in the concentrator.
4. Reinstall filter in concentrator.

- ▲ **Caution:** Do not operate the concentrator without the cabinet filter installed, or while the filter is wet. These actions could permanently damage the concentrator.

Cleaning the cabinet

Clean the cabinet monthly, or more often if it gets dirty or your environment is very dirty.

1. Ensure that the power cord is unplugged and the power switch is in the OFF position.
2. Clean the cabinet with a mild household cleaner and a clean, non-abrasive cloth or sponge.

⚠ WARNING: Do not immerse the concentrator in water or other liquid. Do not pour liquids on the concentrator while cleaning. Do not allow any liquid to enter the concentrator.

▲ Caution: Many cleansers must be diluted before use. Follow the manufacturer's directions carefully to avoid damaging the cabinet.

▲ Caution: Do not use strong solvents, such as acetone.

▲ Caution: Never use an abrasive such as steel wool or metal polish.

Cleaning the (optional) humidifier bottle

1. Change the water in the humidifier bottle every day.
2. Wash the humidifier bottle weekly with warm soapy water. Rinse with a solution of 1 part vinegar to 10 parts water, and rinse thoroughly with hot water before refilling.

PREVENTIVE MAINTENANCE RECORD

User and GF authorized distributor: Please record the concentrator's item number and serial number, and maintain the maintenance record below. Proper care and maintenance are essential to keep your oxygen concentrator in safe working condition.

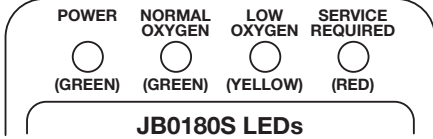
User: when you believe that a component or part of your concentrator is not functioning properly, contact your Graham-Field authorized distributor immediately, as a potentially hazardous condition could result. Only excellent condition is acceptable where safety is concerned.

Note: Failure to maintain your device could void the warranty.

Preventive Maintenance Record for Oxygen Concentrator:			Item Number:				Serial Number:			
Performed by	Frequency	Service	Service Date	Service Date	Service Date	Service Date	Service Date	Service Date	Service Date	Service Date
			Elapsed Hrs	Elapsed Hrs	Elapsed Hrs	Elapsed Hrs	Elapsed Hrs	Elapsed Hrs	Elapsed Hrs	Elapsed Hrs
User	Before each use	Check prescribed LPM flow rate	_____	_____	_____	_____	_____	_____	_____	_____
	Weekly, or more often if it gets dirty or your environment has high dust or pollutant levels.	Clean cabinet filter	_____	_____	_____	_____	_____	_____	_____	_____
		Clean cabinet	_____	_____	_____	_____	_____	_____	_____	_____
⚠ The following procedures are to be performed <i>ONLY</i> by authorized service personnel.										
Authorized distributor	Annually or between patients (whichever happens first)	Check bacteria filter (outlet HEPA filter)	_____	_____	_____	_____	_____	_____	_____	_____
		Check compressor intake filter	_____	_____	_____	_____	_____	_____	_____	_____
		Clean / replace cabinet filter	_____	_____	_____	_____	_____	_____	_____	_____
		Check oxygen concentration	_____	_____	_____	_____	_____	_____	_____	_____
		Check power loss alarm	_____	_____	_____	_____	_____	_____	_____	_____
	As required	Rebuild compressor (contact GF Tech Support)	_____	_____	_____	_____	_____	_____	_____	_____
		Replace exhaust muffler (contact GF Tech Support)	_____	_____	_____	_____	_____	_____	_____	_____

8 TROUBLESHOOTING

The following LED Indicator Table lists LED indicator patterns and causes for each model; the Troubleshooting Table lists troubleshooting solutions for potential problems. Use the LED Indicator Table to identify the problem, if any, and the Troubleshooting Table to solve it. If the problem persists, please contact your Graham-Field authorized distributor.

Model JB0180 only	 <p>POWER (GREEN) SERVICE REQUIRED (RED) JB0180 LEDs</p>		
	Indicator	LED(s) and/or Audible Alarm	Probable Cause
	Power	POWER LED illuminates nonflashing	Power is on
	Alarm	SERVICE REQUIRED LED illuminates; audible alarm sounds	1)Power failure 2)Concentrator failure 3)High pressure 4)Low pressure 5)O ₂ < 72% - do not use concentrator, O ₂ too low Contact your GF authorized distributor
Alert	SERVICE REQUIRED LED illuminates; intermittent audible alarm sounds	No oxygen flow	
Model JB0180S only	 <p>POWER (GREEN) NORMAL OXYGEN (GREEN) LOW OXYGEN (YELLOW) SERVICE REQUIRED (RED) JB0180S LEDs</p>		
	Indicator	LED(s) and/or Audible Alarm	Probable Cause
	Power	POWER LED illuminates nonflashing	Power is on
	Alarm	SERVICE REQUIRED LED illuminates; audible alarm sounds	1)Power failure 2)Concentrator failure 3)High pressure 4)Low pressure 5)O ₂ < 72% - do not use concentrator, O ₂ too low Contact your GF authorized distributor
	Alert	SERVICE REQUIRED LED illuminates; intermittent audible alarm sounds	No oxygen flow
	O ₂ Indicator	NORMAL OXYGEN LED illuminates LOW OXYGEN LED illuminates; SERVICE REQUIRED LED flashes every two seconds; audible alarm sounds LOW OXYGEN LED illuminates for approximately 3 min. at startup; then POWER LED and appropriate O ₂ LED illuminate	O ₂ > 85% O ₂ > 72%, < 85% O ₂ Purity Test performs automatically at startup

TROUBLESHOOTING TABLE

The following Troubleshooting Table lists troubleshooting solutions for potential problems. Use the Symptom column to identify the problem, and the rest of the Troubleshooting Table to solve it. If the problem persists, contact your Graham-Field authorized distributor.

Symptom	Possible Cause	Check / Action
<p>Concentrator power switch is on, but concentrator will not turn on</p> <p>Red LED illuminates (nonflashing)</p> <p>Audible alarm sounds</p>	No power to the device	<p>1)Ensure that power cord is connected to the wall outlet.</p> <p>2)Ensure that there is power in the wall outlet.</p> <p>3)Ensure that an extension cord is not used.</p> <p>4)If the wall outlet is connected to a light switch, ensure that the light switch is in the ON position.</p> <p>5)Ensure that circuit breaker (reset button) is not popped out; if so, push it in.</p> <p>6)If condition persists, contact your GF authorized distributor for repair.</p>
Yellow or red LED illuminates	<p>1)Low oxygen purity</p> <p>2)Blocked tubing or cannula</p> <p>3)Flowmeter set at <0.5</p> <p>4)Device overheating</p>	<p>1)Call GF authorized distributor to change filters.</p> <p>2)Inspect tubing or cannula; replace if necessary.</p> <p>3)Check flowmeter and set to 0.5 or higher (as prescribed by your physician).</p> <p>4)Ensure that concentrator is placed so that all sides are at least 12 inches (30cm) away from walls, draperies, furniture, or other obstructions; that concentrator is not placed in a confined area; and that all concentrator openings are unobstructed and unclogged.</p> <p>5)If condition persists, contact your GF authorized distributor for repair.</p>
Concentrator works, but little or no oxygen from outlet	Possible blockage	<p>1)Inspect tubes and cannula to ensure that they are not kinked or blocked. Smooth tubes to unblock.</p> <p>2)Check humidifier bottle, if so equipped, for leaks or jamming. Remove humidifier bottle, clean with warm soapy water, rinse, reinstall, and tighten.</p> <p>3)If condition persists, contact your GF authorized distributor for repair.</p>
Flowmeter ball located on "0" position, or flowmeter float is unstable	Possible flowmeter problem	Adjust flowmeter knob. If condition persists, contact your GF authorized distributor for repair.

9 SPECIFICATIONS

ITEM		SPECIFICATIONS	
Voltage		115VAC - 60 Hz	
Power consumption		300 W (average)	
Flow range		0.5 to 5 LPM	
Oxygen concentration *		93% \pm 3% @0.5 to 5 LPM	
Oxygen outlet pressure (no flow)		8.5 psi \pm 0.5 psi (58.6 kPa \pm 3.4 kPa)	
Operating pressure		14 to 26 psi (96.5 kPa to 179.3 kPa) @5 LPM	
Environment	Temperature	Operating	50°F to 95°F (10°C to 35°C)
		Storage/Transport	-40°F to 158°F (-40°C to 70°C)
	Humidity	Operating/ Storage/Transport	10~100% RH non-condensing
		Atmospheric pressure	Storage/Transport
Altitude		\leq 7,500 ft. (\leq 2,286m)	
Sound level		60Hz \leq 45 dBa	
Safety system		a) Current overload	
		b) High temperature compressor shutdown	
		c) High pressure alarm with compressor shutdown	
		d) Low pressure alarm with compressor shutdown	
		e) Power loss alarm	
		f) Model JB0180S only: SensO2 oxygen system alarm	
Dimensions (W x D x H)		14.2 x 13 x 21 inches (36 x 33 x 53 cm)	
Weight		33 lb (15kg)	
Pressure relief valve (safety feature)		35 psi \pm 3 psi (241.3 kPa \pm 20.7 kPa)	
LED indicators		Please see previous LED indicator table	
<p>* Oxygen concentrator levels are for operation within specific temperature and altitude ranges. Operation outside specified ranges may decrease oxygen concentration levels. Outlet pressure may need adjustment to compensate for higher altitude.</p>			

Note: Specifications are subject to change without notice.

10 LIMITED WARRANTY

GF Health Products, Inc. warrants this product to be free from manufacturing defects in material and workmanship for its period of warranty that is in accordance with industry standards. This warranty is extended only to the original purchaser/consumer or distributor/non-consumer of this new product and to no other purchaser or transferee.

GF Health Products, Inc. warrants the John Bunn Optium Concentrator and its components to be free from defects in workmanship and materials for five years.

The Warranty period for the consumer commences on the first date a product is delivered to consumer by seller/distributor. If the product is rented or leased, the warranty period commences on the invoice date from *GF Health Products, Inc.* A copy of the invoice showing date of purchase must be provided when submitting warranty claims. When proof of purchase date is not provided, warranty coverage shall commence upon *GF Health Products, Inc.*'s invoice date to the distributor/purchaser.

If within the warranty period, the product or component part is proven to *GF Health Products, Inc.*'s satisfaction to be defective, *GF Health Products, Inc.* shall provide, at its option, one of the following: (1) repair or replacement of any defective or nonconforming part or product or (2) a credit and/or refund of the original selling price. GF HEALTH PRODUCTS, INC.'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO SUCH REPAIR, REPLACEMENT, CREDIT AND/OR REFUND. This warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to the manufacturer.

For warranty service, please contact the authorized distributor from whom you acquired your *GF Health Products, Inc.* product. Upon receiving notice of an alleged defect in a product, *GF Health Products, Inc.* will issue a return authorization. The defective product or part(s) must then be returned, at the purchaser's cost, for warranty inspection using the serial number as identification (or, if the product is not serialized, lot number and date code) within thirty (30) days of return authorization issue date. In the event you do not receive satisfactory warranty service, please contact *GF Health Products, Inc.* at the address below. DO NOT return products to our factory without prior authorization.

LIMITATIONS AND EXCLUSIONS: The foregoing warranty excludes consumables, such as filters, tubing, and casters, as these require routine maintenance. The foregoing warranty shall not apply to serial numbered products if the serial number has been removed or defaced. Products subjected to negligence, abuse, misuse, improper operation, improper maintenance, improper cleaning, improper storage, or damages beyond *GF Health Products, Inc.*'s control are not covered by this warranty, and that evaluation shall be solely determined by *GF Health Products, Inc.* This warranty shall not apply to problems arising from normal wear and tear or failure to follow instructions. The warranty shall also not apply to products modified without *GF Health Products, Inc.*'s express written consent; nor shall it apply if parts not manufactured by *GF Health Products, Inc.*, or if parts not complying with original equipment specifications are added to *GF Health Products, Inc.* products, or if the product or part is serviced by an entity not authorized by *GF Health Products, Inc.*

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES AND IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FIT-

NESS FOR A PARTICULAR PURPOSE, AND SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN, AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO THE REPAIR, REPLACEMENT, CREDIT AND/OR REFUND OF THE DEFECTIVE PRODUCT OR PART PURSUANT TO THE TERMS CONTAINED HEREIN. GF HEALTH PRODUCTS, INC. SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state (province to province). Some states (provinces) do not allow the exclusion or limitation of incidental or consequential damage, or limitation on how long an implied warranty lasts, so the above exclusion and limitations may not apply to you.

This warranty contains the entire agreement between the parties and supersedes any prior, contrary or additional representations or understandings, whether oral or written, concerning our warranty policy.

GF Health Products, Inc. (“Graham-Field”)
2935 Northeast Parkway
Atlanta, GA 30360
Tel 800-347-5678, Fax 800-726-0601

11 INDEX

B

Bathing, use of this device while 4

C

Cabinet, cleaning 16

Cabinet filter, cleaning 15

Caution statement, significance 4

Circuit breaker (reset button) 11

Concentrator Illustration 9

Concentrator setup 10

D

DANGER statement, significance 4

F

Freight policy, GF's 6

H

Handling 6

Humidifier bottle (optional), cleaning 16

I

Installation location safety information 7

Intended use of this device 3

Introduction 3

L

LED Indicator table 13

Life support, this device as 3

Location of the concentrator 7

Luggage handle, using 14

M

Maintenance 15

Maintenance record, preventive 17

N

Note, significance 4

O

Operation 10

Operation with a humidifier bottle (if so prescribed) 12

P

Product description 9

R

Radio Frequency Interference (RFI) 8

Replacement parts 15

Reset button (circuit breaker) 11

S

Safety information 4

Specifications 20

Storage of the concentrator 6

T

Transport, safety during 14

Transporting the oxygen concentrator 14

Troubleshooting 18

Troubleshooting table 19

Turning the concentrator off 11

Turning the concentrator on 10

U

Unpacking the concentrator 6

User maintenance 15

V

Velcro strap 14

W

WARNING statement, significance 4

Warranty, limited 21

USA Corporate Headquarters:

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360**

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