Model PA2293

The HealthTeam Paraffin Heat System is designed to:

- Provide heat therapy to soothe the pain caused by arthritis, joint stiffness and muscle spasms.
- Condition and moisturize coarse, dry hands, feet and elbows.

IMPORTANT: Please read instructions before use

PA2293-INS-LAB-RevB08

CONTENTS

IMPORTANT SAFETY GUIDELINES – PLEASE READ BEFORE USE	3
HEALTHTEAM PARAFFIN HEAT SYSTEM OPERATION.....	6
FILLING THE PARAFFIN HEAT SYSTEM.....	6
HEATING SYSTEM OVERVIEW	7
USING THE PARAFFIN HEATING SYSTEM: FOR HANDS, FEET, OR ELBOWS	7
CLEANING THE UNIT AND REPLACING WAX	10
WARRANTY.....	11

GF, Graham-Field, and HealthTeam are registered trademarks of **GF Health Products, Inc.**

Packaging, warranties and products are subject to change without notice.
GF Health Products, Inc. is not responsible for typographical errors.

**PA2293: HEALTHTEAM PARAFFIN HEAT SYSTEM
PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE**

Congratulations! You have just purchased a **HealthTeam Paraffin Heat System** from Graham-Field Health Products. Please read the following information carefully as it is designed to help provide optimum heat therapy.

IMPORTANT SAFETY GUIDELINES – PLEASE READ BEFORE USE

- ⚠ WARNING: Do not use this product without proper instruction from a Healthcare Professional.**
- ⚠ WARNING: Read and understand all instructions before using the HealthTeam Paraffin Heat System. Failure to use as instructed could result in personal injury and/or damage to the Paraffin Heat System.**

- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.**
- ⚠ WARNING: Always unplug the unit from the electrical outlet immediately after using.**
- ⚠ WARNING: Do not operate this product if it has a damaged cord or plug.**
- ⚠ WARNING: This device is equipped with a three-prong (grounding) plug for protection against possible shock hazard. Plug only into a properly grounded three-prong wall receptacle.**
- ⚠ WARNING: Always place this unit in a location where the electrical cord will not be stepped on, tripped over, or otherwise subjected to damage or stress. Do not use this unit with an extension cord.**

- ⚠ WARNING: Do not use, store or place this unit where it could fall or be pulled into water.**
- ⚠ WARNING: If the unit accidentally falls into water or any liquid, DO NOT TOUCH; unplug immediately!**
- ⚠ WARNING: Use the HealthTeam Paraffin Heat System on a level surface.**
- ⚠ WARNING: Avoid applying hot paraffin wax near eyes, heat sensitive areas, or any open skin injury.**
- ⚠ WARNING: Allow the appliance to cool completely before handling, cleaning or storing.**
- ⚠ WARNING: Do not plug or unplug this appliance when hands or body are wet.**
- ⚠ WARNING: Do not heat paraffin wax in microwave, stove or using any method other than the HealthTeam Paraffin Heat System.**

HEALTHTEAM PARAFFIN HEAT SYSTEM OPERATION

Filling the Paraffin Heat System

1. Ensure that the unit is on a level, flat, sturdy, heat resistant surface.
2. Remove the lid and all plastic packaging inside the unit.
3. Open the one lb. paraffin wax pellets or blocks and place half of the contents in the unit. Add the remaining when these have melted. Do not overfill the unit. Do not fill the system past the maximum fill line indicated.
4. Plug the unit into a properly grounded, 120 volt, 60 cycle single phase power outlet. The red light will illuminate to indicate that power is being transferred to the unit and that the system is in the ON position.
5. Place the lid back onto the unit to retain heat.

6. It will take approximately 50 - 60 minutes to see the wax begin to melt. To melt six pounds of wax requires approximately 4 - 6 hours.

Heating System Overview

The illuminated light indicates the unit is on and operating. The heating element of this unit heats the paraffin wax to approximately 122 - 131 degrees Fahrenheit and will shut off automatically if the thermostat malfunctions.

Using the Paraffin Heating System: For Hands, Feet, or Elbows

1. Remove all jewelry. Wash and dry hands, feet or elbows thoroughly.
2. Remove lid from the unit. Test the temperature of the wax by lightly touching the melted wax with the inside of the wrist. DO NOT place any part of the body into the wax if the wax is too hot when tested on the inside of the wrist. If the temperature is too

hot, contact your Graham-Field equipment provider for assistance.


3. If the temperature of the wax was acceptable when tested on the inside of the wrist, place the desired hand, elbow or foot into the wax. For hands, start by opening the desired hand and spreading the fingers apart as shown in the picture at above right. Slowly immerse the hand, foot or elbow into the paraffin wax and then immediately remove it. Wait 3-5 seconds and re-dip. Waiting 3-5 seconds allows the paraffin wax to set. Repeat this process two or three times to create more wax layers. The paraffin usually takes 12 - 15 minutes to completely harden.

⚠ WARNING: Use with caution if skin is sensitive. Discontinue use immediately if any pain, irritation or discomfort is experienced.

Note: To retain heat, place the hand or foot which has been immersed in a plastic liner and cover with the bootie or mitt provided, shown at right, for fifteen minutes for longer to obtain the desired heat therapy.

4. After heat therapy is finished, peel the hardened wax from the hand, elbow or foot. For hand: begin with the wrist and work in a downward motion, removing all of the paraffin wax. Make sure that all of the wax is removed.
5. To maintain softness, massage oil or lotion of your choice onto the hands, elbows or feet.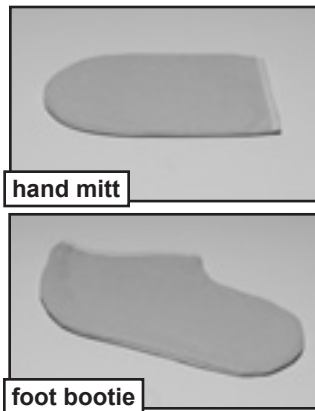

6. Discard the hard wax as it may contain cell build-up from the skin. DO NOT PUT PARAFFIN RESIDUE BACK INTO THE UNIT.

CLEANING THE UNIT AND REPLACING WAX

Clean the unit when the paraffin wax becomes dry or discolored, or when there is a build-up of skin on the bottom of the unit.

1. Unplug the unit. Allow the paraffin to solidify, which can take several hours.
2. After the paraffin is solid, plug the unit in for a few minutes to loosen the wax from the bottom and sides of the unit. Then unplug the unit.

⚠ WARNING: Do not leave the unit plugged in; it could get excessively hot and result in personal injury and/or damage to the unit.

3. Remove the paraffin and wipe the inside of the unit with a paper towel.

4. Make sure that the unit is unplugged and cool before cleaning. Clean the unit with an all-purpose cleaner (DO NOT USE CHLORINE OR BLEACH) and wipe dry.
5. The unit is then ready for new paraffin wax for next use. Store the Paraffin Heat System in a cool, dry place. *

** Paraffin wax beads available: Item # 2294 Wintergreen and Item # 2295 Unscented*

WARRANTY

GF Health Products, Inc. warrants the HealthTeam Model PA2293 Paraffin Heat System for a period of six months for defects in workmanship and materials. During the warranty period, defective items will be repaired or replaced at manufacturer's option at no charge.

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GF, Graham-Field y HealthTeam son marcas registradas de **GF Health Products, Inc.**

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© 2008 GF Health Products, Inc.