


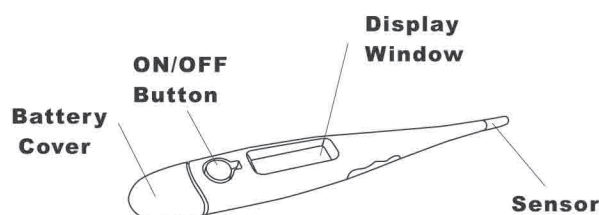
INSTRUCTIONS FOR USE

Please read carefully before using


2210-INS-LAB-RevB09

1. Fast oral temperature measurement in around 10 seconds.
2. Very sensitive unit, best for quick oral and rectal measurement.
3. Easy to read digital LCD (liquid crystal display) .
4. Flexible soft tip fits naturally under tongue or under arm. Won't poke or hurt. Flexible probe conforms to different mouths.
5. Compact, accurate and durable LSI (large scale integration) unit.
6. If the thermometer is inadvertently left on after temperature stabilization , it will automatically shut off in about 10 minutes.
7. Small, lightweight unit . Universal use for the whole family.
8. No hazard compared to broken mercury glass thermometers .
9. Low battery indicator: when the symbol "  " appears , the battery needs to be replaced .
10. This unit is waterproof.

PARTS DESCRIPTION




SPECIFICATIONS


Range :	90~109.9°F(32.0~43.9°C) Below 90.0°F(32.0°C) displays L°F(°C) Above 109.9°F (43.9°C) displays H°F(°C) 0.1 °F(°C)
Resolution	±0.2°F(93.2 F to 107.6°F); ± 0.4°F(other range)
Accuracy :	±0.1°C (34.0 C to 42.0°C); ±0.2°C(other range)
Display :	Liquid crystal display 3 1/2 digits
Battery : (included)	Micro alkaline battery 192, LR41 1.55V
Power Consumption :	0.15 milliwatts in measurement mode
Battery Life:	More than 200 hours of continuous operation
Dimensions :	132mm x 24mm x 12mm
Weight :	Approx. 14 grams including battery
Beeper :	Approx. 8 seconds sound signal when peak temperature reached
Memory :	Built-in memory enabling display of previously measured value
Working Conditions :	Temperature: 50~95°F (+10~35°C); Humidity: 30~75%
Storage Conditions:	Temperature:-13~140°F (-25~60°C); Humidity: 10~83%
Guarantee of Quality:	Certification ISO13485 Conformity EN12470-3 , EN60601-1
Product Classification :	Type BF  equipment

PRECAUTION

1. Do not boil the probe . Instead , clean the unit by wiping it with a dry cloth and disinfect the probe with ethyl (rubbing) alcohol .
2. Do not drop the thermometer or expose it to heavy shock . The unit is not shockproof .
3. Do not bend or bite the probe .
4. Do not store the unit under direct sunlight , at a high temperature , or in high humidity or dust. Performance may be degraded.
5. Stop using the thermometer if it operates erratically or if the display malfunctions .
6. Keep out of the reach of unsupervised children .
7. Clean the thermometer probe before storing .
8. Do not attempt to disassemble the unit except to replace the battery .
9. If the thermometer has been stored at below-freezing temperatures , allow it to warm naturally to room temperature before using .
10. Performance of the device may be degraded if operated or stored outside stated temperature and humidity ranges or the patient's temperature is below ambient (room) temperature.



BATTERY REPLACEMENT

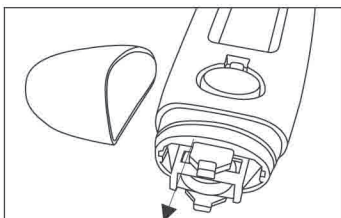1. When the symbol "  " appears , the battery needs to be replaced .
2. Pull the battery cover off in the direction shown.
3. If necessary, use a pointed object such as a toothpick or insulated probe to pry loose the battery cover and the old battery. Avoid using any sharp metal object in this operation.
4. Place a new 1.55 VDC button size battery type 192, LR41 or equivalent in the chamber with the negative (-) side up and the positive (+) side down .
5. Replace and fasten the battery cover securely .



Warning : Discard old battery carefully , out of reach of young children . Swallowing the battery may be fatal . If the battery is swallowed , contact a hospital immediately to have it removed . Do not dispose of the battery in a fire . It may explode .

HOW TO USE

1. Depress the ON/OFF button. The display will read  or .
2. Release the ON/OFF button and the display will show L°F(L°C) with °F(°C) flashing.
3. Disinfect the probe with ethyl (rubbing) alcohol. Place the probe in an appropriate position (oral, axillary, rectal).
4. Once the degree sign (°F or °C) on the display has stopped flashing (usually within 15-30 seconds), the correct temperature is indicated. The temperature reading will not change after the °F(°C) stops flashing.
For beeper models only, an alarm signal will sound for approximately 8 seconds when °F(°C) stop flashing.
5. The unit will automatically turn off in 9 minutes (approx.). However, to prolong battery life, it is best to turn the thermometer off by pressing the ON/OFF button once the temperature has been noted.

Special Features:

Normal Temperature Alarm: An alarm will sound when the peak temperature has registered and is ready-to-read. The "Normal Temperature" alarm (four quick beeps followed by a pause) will sound repeatedly for approximately 8 seconds.

Last-Temperature Memory Recall: When the ON/OFF button is depressed to activate the thermometer, hold down for approximately 5 seconds. The display of 188.8 will change to show the last measured temperature. This reading is displayed as long as the button is held down. When the button is released, the thermometer will be in "ready-to-measure" mode and the recalled temperature will be erased.

Fahrenheit and Centigrade Switchable: The Centigrade or Fahrenheit mode can be switched by pressing the ON/OFF button within the first 5 seconds.

NOTE : Always disinfect the thermometer before you use it.

NOTE : In Step 2, if the room temperature is greater than 90.0°F (32.0°C), it will be displayed rather than L°F(L°C).

NOTE : In Step 3, you may put a sanitary plastic probe shield on the sensor tip to help prevent cross-infection and contamination. Use of the probe cover may result in a 0.2°F(0.1°C) difference from actual temperature. Dispose of any used probe shield and store the unit in a safe place.


RELIABLE METHODS TO MEASURE TEMPERATURE

THE TAKING OF BODY TEMPERATURES: Change in body temperature is one of the most important indicators of illness. It is important that accurate temperatures be taken. Even though the digital thermometer is used in similar manner to the glass thermometer, it is important that the simple instructions for use of this thermometer be followed closely and that the person taking the temperature understands body temperature.

BODY TEMPERATURE: A person's body temperature will usually be several tenths to one full degree lower in the morning than it is in the late afternoon. Therefore, if your temperature is 97.9°F(36.6°C) in the morning, it could be 98.4°F(36.9°C) or more in the late afternoon and still be normal. To determine what your normal body temperature is you should take your temperature both in the early morning and in the late afternoon over several days. Children have the same normal temperature range as do adults. A fever for both children and adults is typically defined as a temperature over 99.5°F(37.5°C) (oral reading). Exercise, excessive clothing, hot bath, hot weather, warm food and drink can cause mild elevations of up to 100.4°F (38°C) to 101.2°F(38.4°C) in oral temperature. If you suspect one of these to be the cause of an elevated reading, eliminate the possible cause and retake the temperature in a half hour.

ORAL USE

Place the probe well under the patient's tongue. Instruct the patient to keep their mouth closed while the thermometer is reading. A normal temperature by this method is usually considered between 98.6°F (36.0°C) to 98.6°F(36.0°C to 37.5°C).

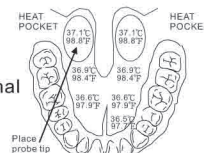

RECTAL USE

If rectal temperature is recommended by your doctor, you may use a probe cover. Insert thermometer into the probe cover. Lubricate the probe cover with a water soluble jelly for easier insertion. Do not use petroleum jelly. Insert tip of probe no more than 1/2 inch into rectum. STOP if you meet any resistance. The sensing device is on the very tip of the probe and there is no need to insert the probe deep into the rectum. In general, the rectal temperature is around 1.0 to 2.0°F(0.5°C to 1.0°C) higher than oral temperature.

AXILLARY USE

Wipe armpit with a dry towel. Place probe in the patient's armpit and keep the patient's arm pressed firmly against their side. In general, the axillary temperature is around 1.0 to 2.0°F(0.5 to 1.0°C) lower than oral temperature. Note that in order to achieve better axillary temperature measurement result, **a minimum measuring time of 3~5 minutes is recommended regardless of the beep sound.**

NOTE : Children should be attended by an adult during the entire temperature recording process. After use, the digital thermometers should be stored in a safe place, out of reach of young children. If you compare temperatures between two thermometers, it is possible to have two different readings even if both thermometers are within their accuracy tolerances. Differences in how you take each reading and the thermometer's tolerance range can account for nearly 0.2°F(0.1°C) variability.

REMEMBER : Find your normal temperature and watch for changes .

CLEANING INSTRUCTIONS

To clean the thermometer, wash the tip with a solution of mild detergent and cool water. Disinfect the thermometer by wiping the sensor and lower stem with a cloth dipped in a household antiseptic solution such as rubbing alcohol.

LIMITED WARRANTY

This thermometer is guaranteed for one year from the date of purchase against manufacturer's defects under normal, household use. Following the directions closely will ensure years of dependable operation. If the thermometer does not function properly, first check the battery and replace it if necessary (see instructions above). Repair Department: 1-800-672-8293.