

Info: The most current version of these instructions can be found online at www.grahamfield.com

- ⚠ **WARNING:** This product is for single patient use.
- ⚠ **WARNING:** Important! Read and understand these instructions before installing or using the Lumex Tendercloud Static Air Mattress Overlay. These instructions contain information that will aid you in achieving maximum therapeutic results. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the overlay is not properly installed, personal injury and damage to the overlay could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. Use only Lumex replacement parts. The use of non-Lumex replacement parts could cause personal injury, property damage, and void the warranty.
- ⚠ **WARNING:** This product has a maximum weight capacity of 300 lb.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.
- ⚠ **ATTENTION: EMERGENCY DEFLATION PROCEDURE**
Mattress must be deflated before starting CPR. For quick deflation, completely unscrew large red valve by turning counter-clockwise.
- ⚠ **PROCEDIMIENTO PARA DESINFLAR EN EMERGENCIA**
El colchón debe de estar desinflado antes de empezar RCP (Resucitación Cardiopulmonar). Para desinflar rápidamente, ajuste completamente la válvula roja grande volteándola hacia su izquierda.

INTENDED USE OF THIS DEVICE

The intended use of the Tendercloud Static Air Mattress Overlay is as a support and pressure reduction surface.

INSTALLATION

Inflation

1. From the foot end of the bed, unroll the overlay on top of regular mattress with valve on the left side. Smooth out and place each of the four corner straps around the standard hospital mattress.
2. Exclusive 2-part valve system:
 - a. Unscrew top black cap.
 - b. Tighten bottom red valve (it is essential that the bottom red valve be closed firmly).
3. Insert air-filling device into top of the red valve and inflate for 60 seconds if using an electric pump.
4. Replace black top cap and tighten firmly after inflation.
5. Make up bed as usual with sheets, leaving overlay valve exposed (incontinence pads may be used).
6. Patient may now be placed on overlay.

OPERATION

Hand Check

1. With patient in place (lying down), gently insert your hand palm up under overlay directly beneath the patient's buttocks.
2. Always keep the back of your hand flat against the hospital mattress. If the patient's weight is directly resting on the palm of your hand, overlay will need to be inflated for an additional 30 seconds. If you cannot feel the patient, overlay is working efficiently and additional inflation is not required.

⚠ **CAUTION: Periodically check patient's skin for redness or deterioration. Consult physician should any occur.**

⚠ **CAUTION: Perform regular hand checks. Mark the following calendar with an "X" through the date when Hand Check is performed.**

Date Tendercloud went into service: _____

MONTH

_____							_____							_____						
1	2	3	4	5	6	7	1	2	3	4	5	6	7	1	2	3	4	5	6	7
8	9	10	11	12	13	14	8	9	10	11	12	13	14	8	9	10	11	12	13	14
15	16	17	18	19	20	21	15	16	17	18	19	20	21	15	16	17	18	19	20	21
22	23	24	25	26	27	28	22	23	24	25	26	27	28	22	23	24	25	26	27	28
29	30	31					29	30	31					29	30	31				

MAINTENANCE

Cleaning

Wipe the overlay with a clean, damp cloth and mild detergent. The overlay may also be cleaned using a 10% solution of sodium hypochlorite (bleach) diluted in water. Air dry overlay thoroughly without direct exposure to sunlight before use.

▲ **NOTICE: DO NOT clean with phenolic products.**

LIMITED WARRANTY

GF Health Products, Inc. offers a 90-day limited warranty against manufacturer's defects on the Lumex AQTC8035 Tendercloud Static Air Mattress Overlay. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.



GRAHAM-FIELD

www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Graham-Field, Lumex and Tendercloud are registered trademarks of GF Health Products, Inc.

© 2011 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 770-368-4700