Info: The most current version of these instructions can be found online at www.grahamfield.com

IMPORTANT SAFETY INFORMATION: PLEASE READ

- ⚠ **WARNING:** Important! Read and understand these instructions before assembling or using the Bath Seat. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Bath Seat is not properly assembled and adjusted, personal injury and damage to the Bath Seat could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your Graham-Field distributor immediately. **DO NOT** use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Bath Seat.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

INTENDED USE

The Bath Seat is intended for use by individuals who require seating support while bathing or showering. It is designed to be placed fully inside the bathtub or shower.

Contraindications

- ⚠ **WARNING:** This product has a maximum weight capacity of 350 lb, **EVENLY DISTRIBUTED**.
- WARNING:** **DO NOT** use this product as a transfer bench or climbing device.
- WARNING:** Use this product only with tub floors wider than 18 3/4".

ASSEMBLY

Remove the Bath Seat from the box and place it on a flat, dry surface.

Attaching the Legs:

1. Locate the seat frame (see figure 1); place it upside down on a flat, dry surface.
2. Locate the legs (see figure 2).
3. Attach the legs to the seat frame (see figure 3). Depress the locking button at the top of each leg and slide it into the seat frame until the button protrudes from the hole.
4. Repeat step 3 for each leg.

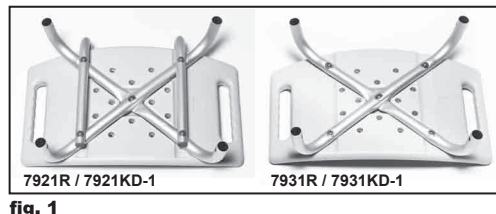
fig. 1

fig. 2

fig. 3

- ⚠ **WARNING:** Ensure legs are securely installed and all hardware is tight before proceeding.

Attaching the Backrest: (Model 7921R and 7921KD Series Only)

1. Locate the backrest (see figure 4).
2. Attach the backrest to the seat frame (see figure 5). Depress the locking buttons and slide the backrest onto the seat frame until the buttons protrude from the holes.

- ⚠ **WARNING:** Ensure backrest is securely installed and all hardware is tight before proceeding.

fig. 4

fig. 5

All Models

- ⚠ **WARNING:** Ensure that the Bath Seat is assembled as described. If the Bath Seat is not properly assembled and installed, personal injury and damage to the Bath Seat could result.

OPERATION

See assembled Bath Seat at right. Contact your GF distributor immediately if you see any issue with fit, form, function, or safety. For more information on GF Health Products, Inc. parts and services, please contact your GF distributor or www.grahamfield.com.

Adjusting Leg Height

Push in the adjustment button and move to the desired height position until it clicks into place. Ensure that legs are locked at the same height and all leg adjustment buttons protrude fully through adjustment holes.

⚠ **WARNING: Ensure all Bath Seat components are secure before each use.**

Center of Balance During Use

The point where the Bath Seat will tip forward, rearward or sideward is determined by its center of balance and stability, and the center of balance and stability is affected by the user's body position and weight distribution. If you fail to heed the following warnings, falling or tipping may occur and could cause severe personal injury.

⚠ **WARNING: To avoid falling or tipping:**

Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping.

Do not put downward weight on one side or corner of the Bath Seat.

Do not put downward weight on the backrest.

Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach.

Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat.

Never lean backward to reach an object.

MAINTENANCE

⚠ **WARNING: Check the Bath Seat weekly for signs of damage or cracking. If damage or cracking of any component of the Bath Seat is found, DO NOT use the Bath Seat. Contact your Graham-Field distributor.**

Inspect the Bath Seat weekly and ensure all components are secure.

To clean the Bath Seat, wash it with soap and water or a mild detergent, then rinse. Ensure that the Bath Seat is rinsed well before use.

WARRANTY

GF Health Products, Inc. ("Graham-Field") warrants the Lumex Model 7921R / 7921KD-1 / 7931R / 7931KD-1 Bath Seat as follows: Limited lifetime warranty against manufacturer's defects. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com.

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2012 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

fig. 6

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Graham-Field y Lumex son marcas registradas de GF Health Products, Inc.