

Tools required: M8 hex key, M6 hex key (included)

Info: The most current version of these instructions can be found online at www.grahamfield.com.

⚠ WARNING: Important! Read and understand these instructions before assembling or using the Stainless Steel 4-Hook IV Stand. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the IV Stand is not properly assembled, personal injury and damage to the IV Stand could result.

⚠ WARNING: If components are damaged or missing, contact your GF distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the IV Stand.

⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

INTENDED USE

The intended use of the Stainless Steel 4-Hook IV Stand is the suspension of IV and other solution bags administered to patients by medical professionals.

ASSEMBLY (REFER TO PICTURE AT RIGHT)

Unpacking

Remove the components from the carton:


Qty	Description
1	Adjustable rod set with clutch collar
1	Base star connector
5	Base leg with bolt
1	Base bolt and flat washer
1	IV hook assembly (ram's horns)
1	Tool set: one each M8 hex key & M6 hex key

Assemble base star connector and base legs

1. Install base leg in base star connector.
2. Install base leg screw into base bolt to secure base leg to base star connector. Use M6 hex key to tighten.
3. Repeat steps 1 and 2 for other four base legs.

Assemble IV holder

1. Install adjustable rod in center of base as shown at right.
2. Install flat washer on base screw; install base screw through base into lower rod set. Use M8 hex key to tighten.
5. Install IV hook assembly (ram's horns) on top of adjustable rod set.

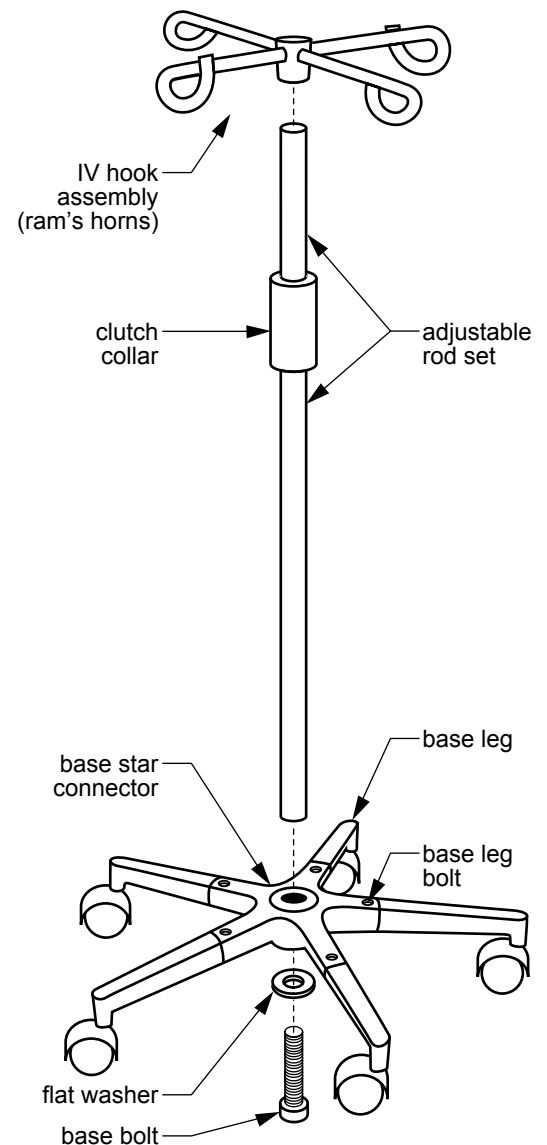

HEIGHT ADJUSTMENT

1. To adjust height, turn clutch collar counter-clockwise until upper rod is loose.
2. Adjust upper rod to desired height.
3. Turn clutch collar clockwise to tighten. DO NOT OVERTIGHTEN.

MAINTENANCE AND CLEANING

To clean the IV Stand, wash with a clean, soft cloth dampened with soap and water or a mild household detergent, wipe with a clean, soft cloth dampened with clear water, and dry with a clean, soft cloth.

⚠ WARNING: Inspect IV Stand before each use and tighten fasteners as needed to ensure secure installation.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the IV Stand.

WARRANTY

GF Health Products, Inc. warrants the Lumex 7016A Stainless Steel 4-Hook IV Stand against manufacturing defects in materials and workmanship for a period of two years. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this literature are based on the latest product information available at the time of printing. The most current product information, including the most current version of these instructions, can be found online at www.grahamfield.com. Graham-Field® and Lumex® are registered trademarks of GF Health Products, Inc.

© 2012 GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, GA 30360, tel 770-368-4700, fax 770-368-2386