The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).

Tools required: Phillips screwdriver

- ⚠ **WARNING:** Important! Read and understand these instructions before assembling or using the Everest & Jennings Adjustable Back Upholstery with Cushion. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the Upholstery is not properly assembled, personal injury and damage to the Upholstery could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your Graham-Field distributor immediately. DO NOT use substitute parts. Use only Everest & Jennings replacement parts. Non-Everest & Jennings replacement parts could cause personal injury and damage to the Upholstery.
- ⚠ **WARNING:** DO NOT exceed the wheelchair's maximum weight capacity.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

## INTENDED USE

The Adjustable Back Upholstery with Cushion is intended to provide a stable surface for propulsion and completion of wheelchair user's daily living activities.

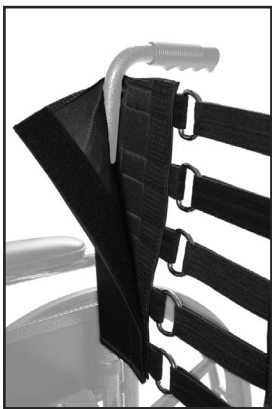
Fig. 1

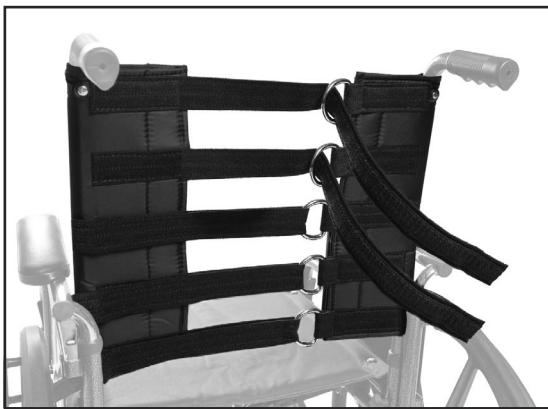
Fig. 2

Fig. 3

Fig. 4

## ASSEMBLY (REFER TO PICTURES ABOVE)

1. Use Phillips screwdriver to remove existing wheelchair back upholstery; keep hardware and upholstery for future use.
  2. Place Back Upholstery around backposts as shown in Fig. 1.
  3. Using existing hardware, install screw through grommet at top of Upholstery into backpost as shown in Fig. 2. Use Phillips screwdriver to tighten securely. Do this for both backposts.
  4. Place cushion over Upholstery as shown in Fig. 3, and use hook and loop strips to fasten. Wheelchair with Back Upholstery and Cushion installed is shown in Fig. 4.
  5. Straps can be tightened or loosened as shown in Fig. 2 depending on user preference.
- ⚠ **WARNING:** Straps must be securely fastened. Failure to do so could affect back support and result in personal injury.

## MAINTENANCE AND CLEANING

Clean Upholstery and Cushion when necessary with a mild soap and water solution. Inspect Upholstery at least once a month for rips, tears and worn spots; ensure that all Upholstery attaching screws are present, properly aligned, and well-secured. Use a Phillips screwdriver to tighten Upholstery mounting screws. Check attaching screws for sharp edges or stripped screws, and replace if found.

**⚠ WARNING: Worn or torn Upholstery, or Upholstery with loose hardware, must be replaced immediately. It may not support body weight.**

## LIMITED WARRANTY

GF Health Products, Inc. (“Graham-Field”) offers a one (1) year limited warranty against manufacturing defects for the 2055LS Adjustable Back Upholstery with Cushion. During the warranty period, defective items will be repaired or replaced at GF Health Products, Inc. option. Warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

**GRAHAM-FIELD**  
[www.grahamfield.com](http://www.grahamfield.com)

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at [www.grahamfield.com](http://www.grahamfield.com). **E&J, Everest & Jennings**, and **Graham-Field** are registered trademarks of GF Health Products, Inc.

© 2013 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in USA

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