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LUMEX®

Features

- Improves mobility and allows for greater independence for the user on the go
- Heavy duty steel frame
- Ergonomic hand grips and hand brakes allow user to operate with ease
- Complete with basket for storage of personal items
- 500 lb maximum weight capacity – EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.
2935 Northeast Parkway, Atlanta, Georgia 30360
Made in China www.grahamfield.com

GF1400029RevA14

LUMEX®

Deluxe 4-Wheel Rollator

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Item RJ4200A

User Instructions and Warranty Information

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Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER
THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

Instructions for Use – Please Read All Instructions Before Use.

Please consult a healthcare professional for the proper adjustment.

Opening the Rollator

- Remove Rollator and other components from carton.
- Unfold frame; spread front and back legs apart.
- Push down locking mechanism to secure legs in place.
NOTE: Ensure the side crossbars (locking mechanism) lock down.

Folding the Rollator

- If necessary, remove the accessories and lift the seat.
- Pull up on the locking mechanism underneath the seat.

Using the Hand Brake

Brakes should be used in one of the following method:

- To Remain Stationary: Hold the hand brake up.
- To Continue Mobility: Release the brake. To unlock, pull up on the top portion of the brake handle.
- To Sit on Seat: Push down on the bottom portion of the brake handle to lock.
- To unlock, pull up on the top portion of the brake handle.
Note: Always lock hand brakes when user is immobile; never sit on rest seat if hand brakes are not locked.

Adjusting the Height

- Remove the adjustment knob and bolt by turning counterclockwise.
- Consult with healthcare professional for proper adjustment.
- Install the adjustment knob into one of the adjustment holes by turning clockwise.
- Repeat STEPS 1–3 for the other side.

Make sure that the height adjustment is at the same height for both handles to ensure safety and proper use.

NOTE: After adjusting the height, the brakes of the rollator must be checked and, if necessary, adjusted.

Attaching Basket

Basket

Install the basket underneath the seat. On some rollator models, the basket is designed to fit underneath the seat or in the front by positioning the hooks on the front seat support frame.

Maintenance

- For dust and dirt, wipe Rollator gently with damp, soft cloth.
- Inspect wheels periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the rollator.
- Verify operation of the brakes and have them adjusted if necessary by your GF Health Products, Inc. authorized distributor.
- Inspect Rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrip or hand brake are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- For the best service from your rollator, always use replacement parts from GF Health Products, Inc.
- Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair or other debris that could interfere with free wheel operation.

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