

*Info: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).*

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the UpRise Onyx Folding Walker. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the UpRise Onyx Folding Walker is not properly assembled, personal injury and damage to the Walker could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. The use of non-Lumex replacement parts could cause personal injury, property damage, and void the warranty.**
- ⚠ **WARNING: This product has a maximum weight capacity of 400 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Always use caution when using this product on wet or slippery surfaces.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

### INTENDED USE

The Lumex 700175CR UpRise Onyx Folding Walker is intended as a dual-purpose mobility aid and rising aid. It is recommended for use by users whose height is between 5'4" and 6'2". Please consult a healthcare professional for proper adjustment, as recommended user heights are an approximation.

### UNPACKING

Remove walker frame and leg attachments from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your Graham-Field authorized distributor for further instruction.

### ASSEMBLY

#### Install Leg Attachments

1. Depress the leg assembly button and insert the leg attachment into the leg until the leg assembly button pops out of its hole on the frame, as shown at right.
2. Repeat for other three legs.

#### Adjust Walker Height


Handgrip-to-floor height is adjustable between 32" and 39". Each walker leg is equipped with several height adjustment positions, as shown at right.

1. To adjust leg height, depress the height adjustment button and twist the leg attachment to the desired height until the height adjustment button pops out of the appropriate hole.
2. Repeat for other three legs.

- ⚠ **WARNING: Before proceeding, ensure leg assembly buttons protrude fully from frame holes, height adjustment buttons protrude fully from height adjustment holes, and all legs are adjusted to the same height.**

#### Open Walker (see picture at right)

1. Grasp one handgrip and pull away from center of walker.
2. Repeat with other handgrip.
3. Push handgrips away from each other until the walker release mechanism clicks into place.  
*Info: You will hear the release mechanism click into place.*
4. Walker is properly opened when release mechanism push button is up.

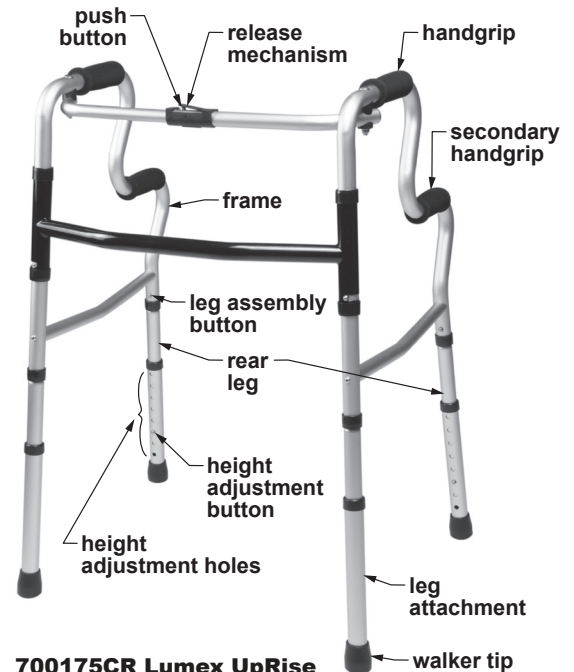

700175CR Lumex UpRise Onyx Folding Walker

## Close Walker

1. Push downward on release mechanism and hold—sides of walker will collapse inward when release mechanism is depressed.
2. Grasp one handgrip and push toward center of walker.
3. Repeat for other handgrip.

## WALKER OPERATION

- ⚠ WARNING: Before use, ensure walker is fully opened, leg assembly buttons protrude fully from frame holes, height adjustment buttons protrude fully from height adjustment holes, and all legs are adjusted to the same height.**

Once leg attachments are adjusted and the walker is opened, the user can operate the walker by grasping one hand grip in each hand, positioning body between the two rear walker legs, and ambulating. The rear walker legs should be in contact with the floor at all times.

## PORTABLE TOILET SAFETY FRAME OPERATION

The walker may also be used as a portable toilet safety frame in either position shown at right.



## MAINTENANCE

- ⚠ WARNING: Inspect the walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your dealer immediately for repair or appropriate replacement parts.**

Wipe walker weekly with a clean, damp cloth. Clean handgrips with mild soap or detergent.

- ▲ NOTICE: Never use solvents, abrasive cleaners, or scouring pads on any part of the walker.**

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, the frame is warranted for three (3) years, and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

A GF Customer Service Representative must authorize warranty service. Please contact the GF Customer Service department by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the Bill of Lading and must be made with immediacy. The ICC regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within thirty (30) days of the invoice date.



**GRAHAM-FIELD**  
[www.grahamfield.com](http://www.grahamfield.com)

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).  
Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2009 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China