

*Note: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).*

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Toilet Seat Rail. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If this product is not properly assembled and installed, personal injury and damage could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage.**
- ⚠ **WARNING: This product has a maximum weight capacity of 250 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

### **INTENDED USE**

The Toilet Seat Rail is intended to provide support and stability during toileting.


### **INSTALLATION**

#### **Unpacking**

Remove Toilet Seat Rail from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your Graham-Field authorized distributor for further instruction.

#### **Installation (Refer to Illustration at right)**

1. Remove existing toilet seat.
2. Place mounting bracket on flat surface of bowl and align holes. Ensure that mounting bracket tube is to the rear, closest to the tank.
3. Replace toilet seat, inserting bolts through mounting bracket holes and into toilet seat mounting holes. Replace original nuts and tighten.
4. Depress push buttons and install arms in mounting bracket as shown. Ensure the Toilet Seat Rail sleeves rest securely on the toilet rim.
5. Adjust Toilet Seat Rail arm width by engaging push buttons in desired holes in mounting bracket tube. Adjust the arms to desired width of either 17" (inner holes) or 19" (outer holes).

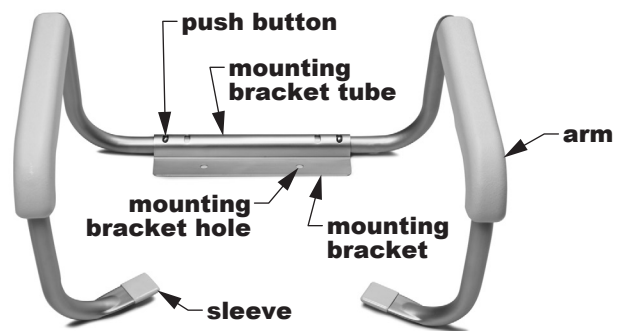

**Toilet Seat Rail,  
Width-Adjustable**

- ⚠ **WARNING: Ensure mounting bracket push buttons are locked in place before use.**

## **OPERATION**

- ⚠ WARNING: Always ensure this product is securely installed before use.**
- ⚠ WARNING: Users who have limited physical capabilities must be assisted or supervised when using this product.**

## **MAINTENANCE / CLEANING**

- ⚠ WARNING: Check the toilet seat rail weekly for signs of damage or cracking. If damage is found, DO NOT use the toilet seat rail. Contact your Lumex authorized distributor immediately.**

Loosen, retighten, and check for secure fit of toilet seat rail weekly. To clean toilet seat rail, wash with soap and water or a mild household detergent and rinse with clear water. Ensure the toilet seat rail is rinsed well before use.

**▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the toilet seat rail.**

## **LIMITED WARRANTY**

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com). GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

## **OBTAINING WARRANTY SERVICE**

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

## **EXCLUSIONS**

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

## **ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER**

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).



**GRAHAM-FIELD**  
[www.grahamfield.com](http://www.grahamfield.com)

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2011 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China