HT9007, HT9008, HT9009, ENGLISH STYLE ICE CAP: INSTRUCTIONS FOR USE

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Note: The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ **WARNING: Important! Read and understand these instructions before using the Lumex English Style Ice Cap. If you do not understand any part of these instructions, contact your medical supply distributor for direction in the use of this product.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

INSTRUCTIONS FOR USE

▲ **NOTICE: Do not put the ice cap in contact with sharp objects which could pierce the material and cause damage.**

Unscrew the top counter-clockwise to open. Carefully insert ice and water into opening. Ensure the ice is free from sharp edges which may puncture the bag and cause a leak. Screw the top clockwise to close. Ensure the top is securely closed.

INSTRUCTIONS FOR STORAGE

After use, dispose of contents and air dry. Store in a cool, dry place.

One-Year Limited Warranty

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the Lumex HT9007 / HT9008 / HT9009 English Style Ice Cap is warranted for one year. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

Obtaining Warranty Service

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.


Exclusions

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

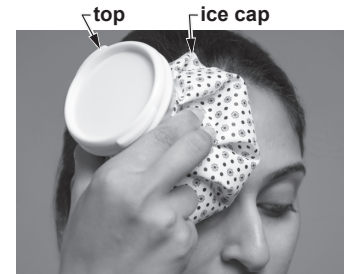HT9007

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com.

GRAHAM-FIELD
www.grahamfield.com

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

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Graham-Field y Lumex son marcas registradas de GF Health Products, Inc.