Info: The most current version of these instructions can be found online at www.grahamfield.com.

IMPORTANT SAFETY INFORMATION: PLEASE READ

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the Shower Head. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Shower Head is not properly assembled and installed, personal injury and damage to the Shower Head could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your Graham-Field distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Shower Head.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**


CONTENTS

One 84" shower hose with nut on each end; one handheld shower head; three rubber washers; and one three-way diverter valve and mounting bracket.

INSTALLATION

▲ NOTICE: Do not over-tighten any items.

1. Remove all contents from packaging.
2. Remove the existing shower head, leaving the shower arm.
3. Screw the diverter valve short male end into the new shower head.
4. Place a rubber washer in the diverter valve female end, if not already installed.
5. Screw the diverter valve female end into the shower arm.
6. Place a rubber washer in the nut at one end of the shower hose.
7. Screw that shower hose nut into the diverter valve long male end.
8. Place a rubber washer in the remaining shower hose nut.
9. Screw that hose nut onto the shower head.

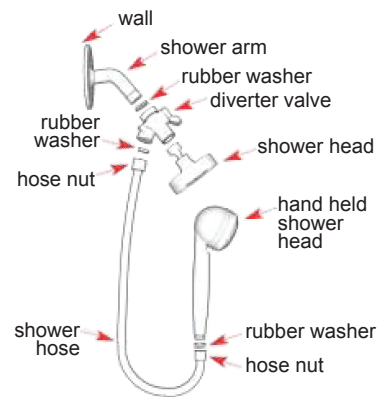MAINTENANCE / CLEANING

To clean the shower head, wash it with soap and water or a mild detergent, then rinse with clear water.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the shower head.

LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, this product is warranted for three (3) years. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com.

GRAHAM-FIELD
www.grahamfield.com

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc. © 2011 GF Health Products, Inc.

2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

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Graham-Field y Lumex son marcas registradas de GF Health Products, Inc. © 2011 GF Health Products, Inc.