7108R: 3-IN-1 FOLDING STEEL COMMODE ASSEMBLY & OPERATION INSTRUCTIONS

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Info: The most current version of these instructions can be found online at www.grahamfield.com.

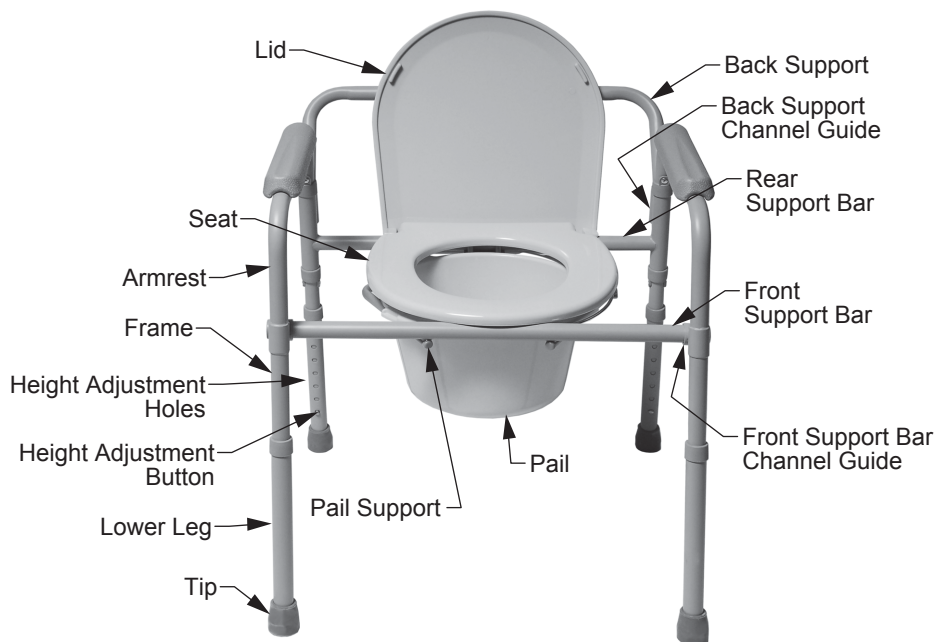
IMPORTANT SAFETY INFORMATION: PLEASE READ

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the 3-in-1 Folding Steel Commode. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the 3-in-1 Folding Steel Commode is not properly assembled and installed, personal injury and damage to the 3-in-1 Folding Steel Commode could result.**
- ⚠ WARNING: If components are damaged or missing, contact your distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the 3-in-1 Folding Steel Commode.**
- ⚠ WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65
WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INTENDED USE

The Lumex 7108R 3-in-1 Folding Steel Commode may be used as a bedside commode, raised toilet seat, and toilet safety frame. It is intended to provide assistance to those who have difficulty sitting down or standing up from the toilet or need additional support and stability during toileting.

SETUP (REFER TO PICTURE BELOW)

7108R Folding Steel Commode

▲ NOTICE: DO NOT use excessive force when installing the seat and lid.

1. Remove frame, seat, lid, pail, cover, and splash guard from carton.
2. Attach front support bar to armrest by depressing the locking button and sliding into the front support bar channel guide until it clicks into place. Repeat for other side. Ensure the locking buttons protrude fully from the holes.
3. Attach seat to rear support bar. Push down firmly until the seat snaps into place.

⚠ WARNING: When installing the seat on the frame, ensure the seat is evenly spaced between the side frames to ensure proper stability.

4. Attach lid to rear support bar.
5. In the upright position, place splash shield or pail and cover in frame.

Adjusting Commode Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for other 3 legs.

⚠ WARNING: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Inserting Back Support

1. Remove back support from carton.
2. Insert back support into channel guides until back support locking buttons click into place in channel guide holes.

⚠ WARNING: Ensure that locking buttons protrude fully through holes before use.

Use Commode over Toilet

1. Set commode to desired height (described in previous steps).
2. Remove commode pail and lid.
3. Position splash guard in place on pail supports.
4. Remove back support (described in previous steps).
5. Lift toilet bowl seat.
6. Place commode over toilet with splash shield positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat and splash guard with disinfectant after each use.

Use as Bedside Commode

1. Set commode to desired height (described in previous steps).
2. Lift commode seat and lid and place commode pail on pail supports. Ensure that pail is properly positioned below commode opening.
3. Place commode seat down. Commode is now ready for use.
4. When user is finished using commode, place commode pail lid over commode pail to contain odors.
5. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

MAINTENANCE

⚠ WARNING: Inspect frequently to ensure that all attaching hardware is tight.

⚠ WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

▲ NOTICE: Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.

After each use: Dry with a soft cloth.

Cleaning: Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.

To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com. **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.

© 2010 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

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