## 89180 / 89220 SERIES: PVC SHOWER CHAIR ASSEMBLY & OPERATION INSTRUCTIONS

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*Info: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).*

*Tools required: Phillips screwdriver (not included)*

### **IMPORTANT SAFETY INFORMATION: PLEASE READ**

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the Shower Chair. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Shower Chair is not properly assembled and adjusted, personal injury and damage to the Shower Chair could result.**
- ⚠ WARNING: This product should not be used without proper instruction from a healthcare professional.**
- ⚠ WARNING: If components are damaged or missing, contact your Graham-Field authorized distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Shower Chair.**
- ⚠ WARNING: To avoid possible tilt hazard, do not leave user unattended. Attendant must always be behind and in control of Shower Chair when user is being positioned or Shower Chair is in use.**
- ⚠ WARNING: Do not operate the Shower Chair on streets or roadways.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

### **INTENDED USE**

The Shower Chair is intended for use by individuals who require seating support while showering, with the approval of a treating physician or authorized medical personnel.

### **Contraindications**

- ⚠ WARNING: This product has the following maximum weight capacity:  
89180 Series: 300 lb, EVENLY DISTRIBUTED.  
89220 Series: 375 lb, EVENLY DISTRIBUTED.**

### **ASSEMBLY**

Remove Shower Chair and components from the box. Place Shower Chair on a flat surface.

**Info: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use Shower Chair. Contact carrier / dealer for further instruction.**

### Installing the Backrest Upholstery:

**Fig. 1**

**Fig. 2**

**Fig. 3**

1. Locate the backrest and backrest upholstery.
2. Lay the backrest on a flat surface.
3. Slide the backrest upholstery over the backrest tubing as shown in Figs. 1 and 2.
4. Attach the hook and loop over the top of the backrest as shown in Fig. 3.

### Attaching the Backrest:

**Fig. 4**

**Fig. 5**

1. Lock the rear casters to prevent Shower Chair from moving during assembly: step on rear caster lever and press down to lock (to unlock: push up rear caster lever).
2. Locate the channel guides (2 each) on the back side of the chair frame near the seat. Slide the backrest support tubes into the channel guides as shown in Fig. 4.
3. Ensure the tubes are pressed all the way in and the backrest support tubes are even.
4. Locate the two Phillips head screws.
5. Insert the screws through backrest tube predrilled holes; use a Phillips screwdriver to tighten screws as shown in Fig. 5.

**⚠ WARNING: Ensure backrest is securely installed and all hardware is tight before proceeding.**

## OPERATION

89180 Series

89220 Series

- ⚠ WARNING: Ensure the Shower Chair is assembled as described and toilet seat is in downward position before use. If the Shower Chair is not properly assembled and installed, personal injury and damage to the Shower Chair could result.**

### Center of Balance During Use

The point where the Shower Chair will tip forward, rearward or sideward is determined by its center of balance and stability, and the center of balance and stability is affected by the user's body position and weight distribution. If you fail to heed the following warnings, falling or tipping may occur and could cause severe personal injury.

- ⚠ WARNING: To avoid falling or tipping:**

**Have someone help you until you know the balance points of your Shower Chair and how to avoid tipping.**

**DO NOT put downward weight on one armrest at a time. If using the armrest for balance, hold both armrests at once to distribute your weight equally.**

**DO NOT put downward weight on the backrest.**

**DO NOT reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach.**

**DO NOT try to reach an object if you must move forward in your seat so much that your buttocks do not stay in full contact with the seat, creating an unsafe position; your buttocks must remain in full contact with the seat at all times.**

**DO NOT lean backward to reach an object.**

### MAINTENANCE

- ⚠ WARNING: Ensure all Shower Chair components and attaching hardware are secure before each use.**

To clean Shower Chair: use a mild soap and water solution, rinse with clear water, and then dry with a soft cloth. Ensure that Shower Chair is well rinsed before using.

- ▲ NOTICE: DO NOT use organic solvents, abrasive cleaners, or scouring pads on any part of Shower Chair.**

## LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for one (1) year, and all other components are warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

## OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

## EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

## ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

**GRAHAM-FIELD**  
[www.grahamfield.com](http://www.grahamfield.com)

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at [www.grahamfield.com](http://www.grahamfield.com). **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.

© 2013 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

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