


Info: The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ **WARNING: Important! Use the GF1871 Cervical Traction Set only under the care of a physician. Read and understand these instructions before assembling or using the GF1871 Cervical Traction Set. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the GF1871 Cervical Traction Set is not properly assembled and adjusted, personal injury and/or damage to the Cervical Traction Set could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and/or damage to the Cervical Traction Set.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65**
WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**


ASSEMBLY INSTRUCTIONS

Unpack box; remove components from bags.
During assembly, refer to pictures throughout instructions to identify components.


1. Insert top of pulley hanger into slot in bracket; turn hanger until it fits into slot. After insertion, ensure that bump traps pulley hanger in slot as shown at right.
2. Ensure that pulleys are oriented as shown at right (bumps keep pulleys separated). Insert bottom of hanger pulley through bottom bracket lip as shown (this will require a little force).
3. Install bracket and pulley hanger over door as shown at right.

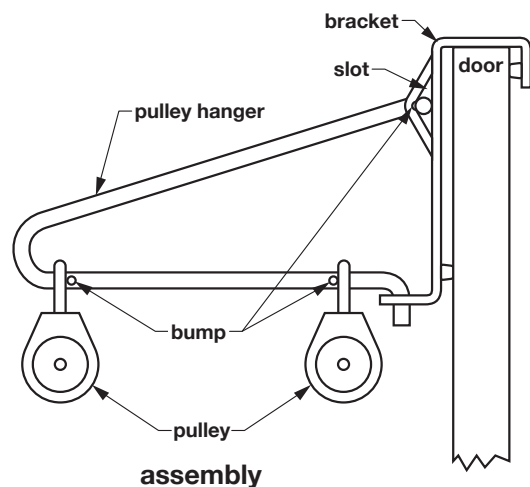


- Insert one of the rope ends (end 1 in the picture below) through the ring at the top of the water weight bag; tie a secure knot, such as a bowline with a back-up knot, to fasten it. If you do not know how to tie a bowline with back-up knot, follow steps A-D below:

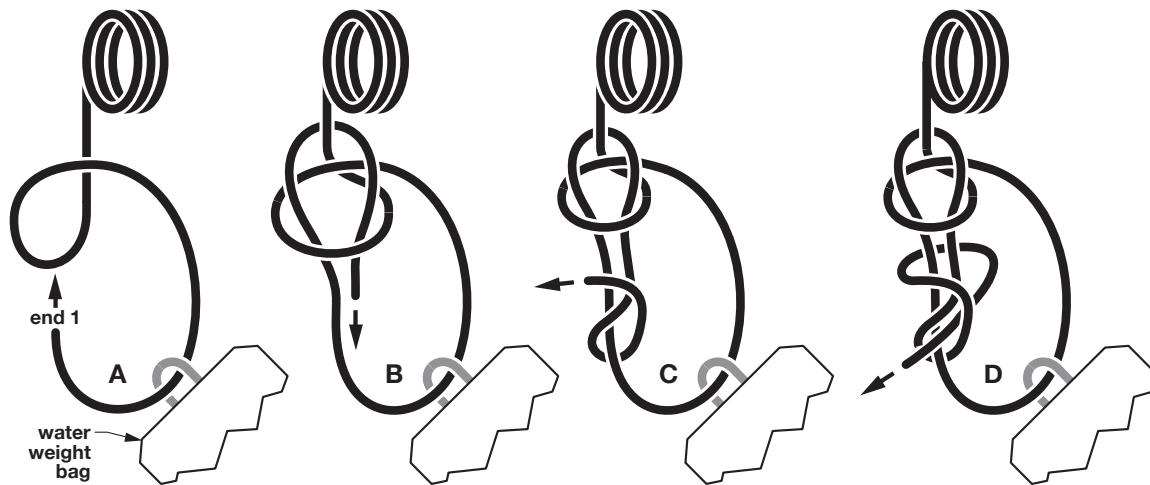

- Pull rope ends taut. Before continuing, ensure that rope is taut and tied as instructed in step 4, with water weight bag securely installed on rope.
- Fill water weight bag with physician-prescribed amount of water.

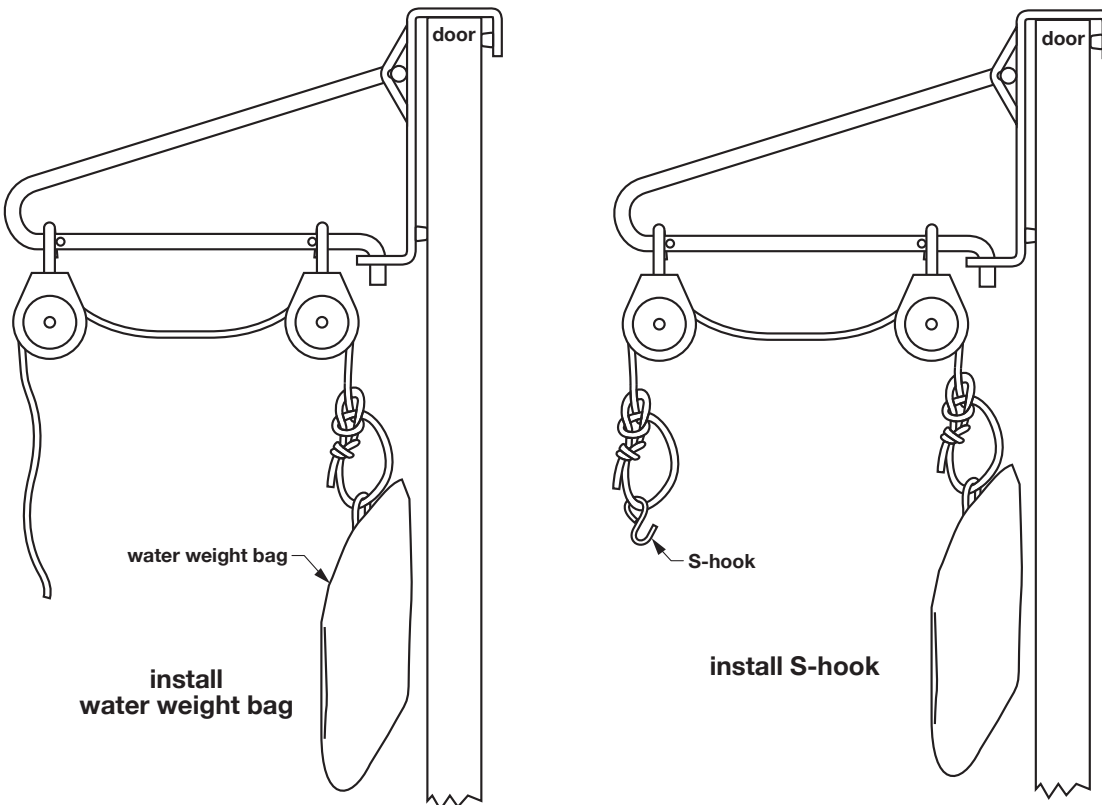

- Thread free end of rope through pulleys with water weight bag at closest point to door, as shown at above left.
- Install the closed loop of the S-hook on the free rope end and tie a secure knot, as you did in step 4. Pull rope ends taut. Before continuing, ensure that rope is taut and tied as instructed, with S-hook securely installed on rope, as shown at above right.

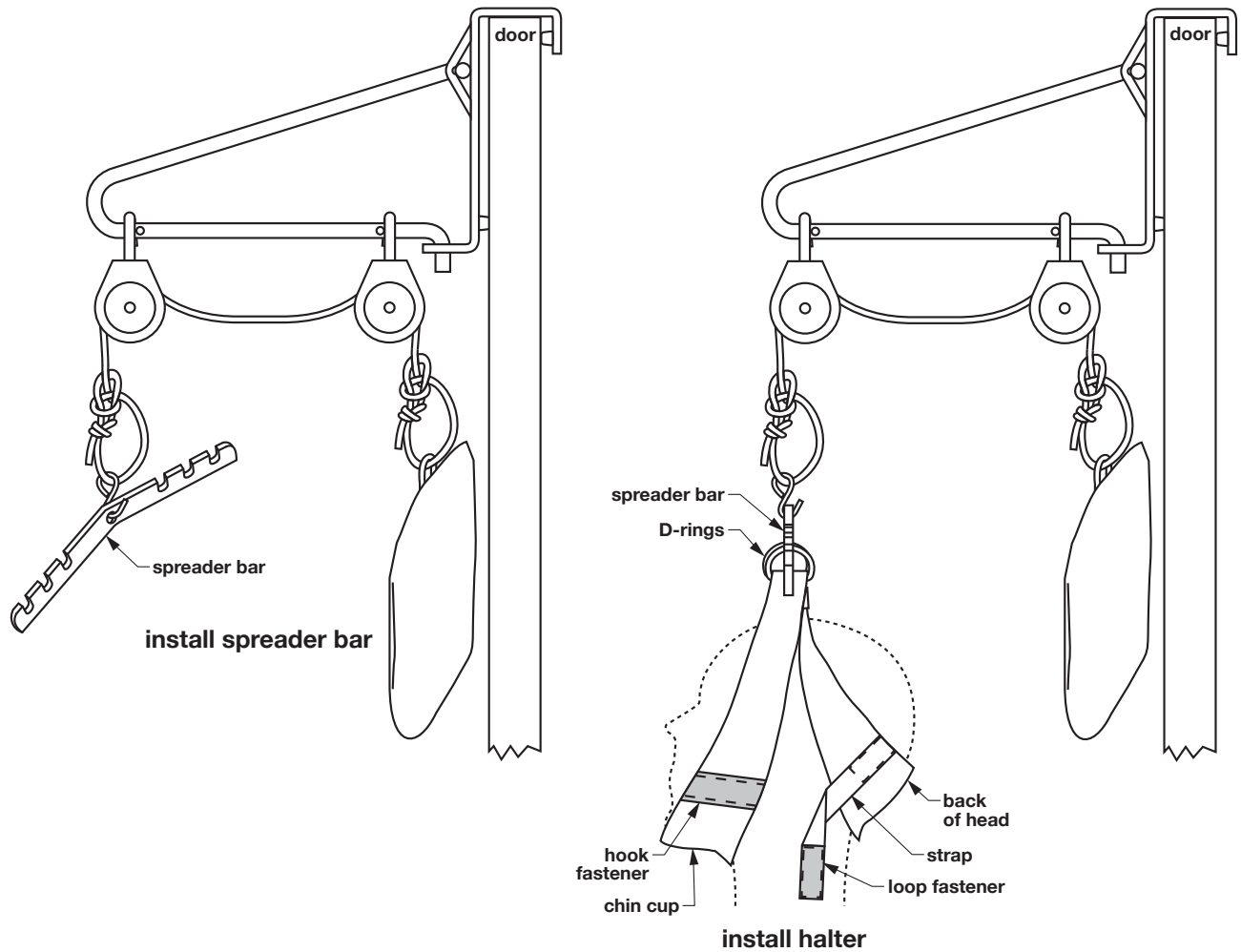


9. Hang spreader bar on S-hook as shown at above left.
10. Install halter by D-rings on spreader bar slots as shown at above right, with straps and hook and loop fasteners oriented as shown, in the position recommended by your healthcare provider.

OPERATION INSTRUCTIONS

⚠ WARNING: Before each use, ensure knots are tight, all hardware is secure, and rope is not frayed. Ensure rope remains aligned on top of pulleys and moves freely; rope may get off-track and bind between pulley wheel and casing.

1. Sit in chair and face away from door as shown at upper right (or as your physician prescribes). Slip halter, oriented as explained in step 10 above, onto your head.
2. Fasten hook and loop fasteners as shown at right. Ensure halter fits comfortably; adjust fit by moving halter straps up or down at D-rings. Avoid any sudden, jerky movement.
3. Use the Cervical Traction Set for the prescribed amount of time; do not move during this time.
4. At end of prescribed time, lift water weight bag to relieve tension on halter. Detach hook and loop fasteners, and carefully remove halter from your head.

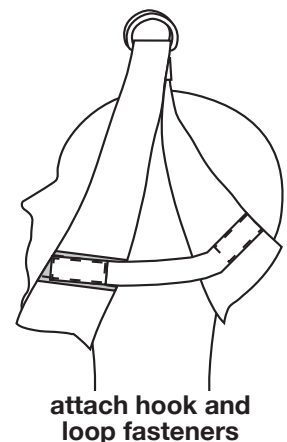

CLEANING AND MAINTENANCE

To clean, hand wash rope and halter gently, using mild soap, rinse with clear water, and air dry. Other components: use a clean, soft cloth and a mild soap and water solution to clean, rinse with clear water, and air dry or dry with a clean, soft cloth.

▲ NOTICE: DO NOT use harsh cleansers or bleach to clean Cervical Traction Set.

ONE-YEAR LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the Lumex Cervical Traction Set is warranted for one year. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

Obtaining Warranty Service

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

Exclusions

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com. **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.

© 2007 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China