

LUMEX®

Item 5358

Qty. 1

Inflatable Hair Shampooer



INSTRUCTIONS FOR USE

- Inflatable plastic ring supports neck while forming shallow basin
- Portable, folds flat for storage
- Complete with drainage hose
- Size: 22 1/2" x 23 1/2"

**PLEASE SAVE THESE
INSTRUCTIONS FOR
FUTURE REFERENCE**




5358-INS-LAB-RevC14

Info: The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the Lumex Inflatable Hair Shampooer.**
- ⚠ WARNING: If components are damaged or missing, contact your GF distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INTENDED USE

The Inflatable Hair Shampooer, shown below, is intended to be used as a portable shampoo basin.

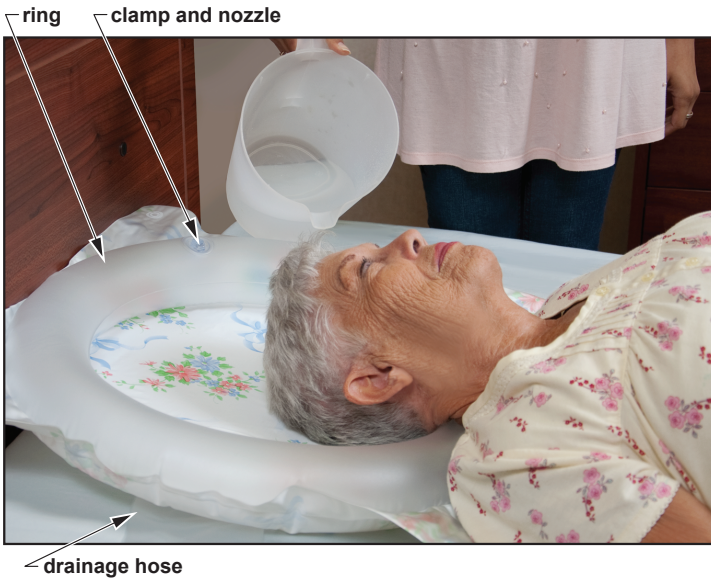

5358 Inflatable Hair Shampooer, shown in use

OPERATION INSTRUCTIONS

Refer to picture at left to see Inflatable Hair Shampooer in use.

1. Pull clamp off nozzle (if necessary for inflation) and inflate Hair Shampooer until ring is firm.
2. Close clamp on nozzle so that air does not escape.
3. Position the patient so that their head is close to the side of the bed where you will be working.
4. Slide pillow down so that it is under the patient's shoulders and neck, not under patient's head.
5. Place Hair Shampooer under patient's head.
6. Ensure drainage hose is hanging into collection bucket on floor, and that hose is clear and open to allow free flow of water into the bucket.
7. Using a pitcher to wet hair, proceed with shampoo. The water in the Hair Shampooer will pool and drain off through the hose into the bucket.
8. Rinse well to prevent soap build-up.
9. Wipe Hair Shampooer as dry as possible with a clean, dry cloth, then leave it inflated for several hours to allow complete drying before storage.
10. When dry, deflate the Hair Shampooer by pulling the clamp off the nozzle. When the Hair Shampooer is completely deflated, fold and store it away from direct sunlight and heat.

SIXTY-DAY LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the 5358 Inflatable Hair Shampooer is warranted for sixty days. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

Manufactured for:
GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
tel 770-368-4700
fax 770-368-2386

Made in Taiwan



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this literature are based on the latest product information available at the time of printing. The most current product information, including the most current version of these instructions, can be found online at www.grahamfield.com. **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.