

**Info:** The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).

- ⚠ **WARNING:** Important! Read and understand these instructions before installing the 3011 Inner Lock / 3012 Outer Lock. If the lock is not properly installed, personal injury and damage to the lock or Narcotic Safe may result.
- ⚠ **WARNING:** If components are damaged or missing, contact your GF authorized distributor immediately. DO NOT use substitute parts. Use only Grafco replacement parts. Non-Grafco replacement parts could cause personal injury and damage to the lock or Narcotic Safe.
- ⚠ **WARNING:** Notice for California Customers- California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

### COMPONENTS


Item	Description	Components Included
3011	Inner Lock, Narcotic Safe	1 Lock, 1 Locking Clip, and 2 Keys
3012	Outer Lock, Narcotic Safe	

### INSTALLATION (REFER TO PICTURES AT RIGHT)


**Tools Required:** *Needle-Nose Pliers and Rubber or Wooden Mallet*

1. Release the Locking Clip with Needle-Nose Pliers as shown in Figure 1.
2. Remove the damaged lock.
3. Locate the new lock; unlock and remove the key.
4. Place the new lock in the punched hole, aligned as shown in Figure 3.
5. To secure the new lock, slide the locking clip with the mallet as shown in Figure 4.

- ▲ **NOTICE:** Ensure lock is securely installed before using Narcotic Safe.

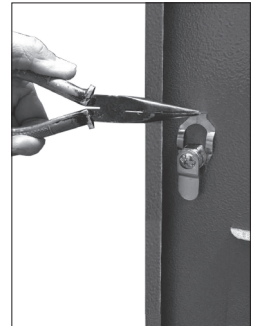

**Figure 1**

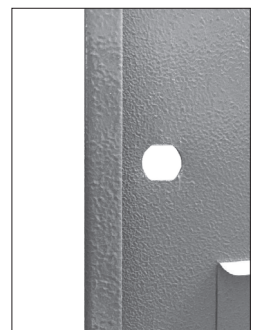

**Figure 2**



**Figure 3**



**Figure 4**



**GRAHAM-FIELD**  
[www.grahamfield.com](http://www.grahamfield.com)

GF Health Products, Inc. is not responsible for typographical errors. All design details, illustrations, specifications, packaging and warranties contained in this literature are based on the latest product information available at the time of printing and are subject to change. The most current product information can be found online at [www.grahamfield.com](http://www.grahamfield.com). Follow us on [in](#) [f](#) [v](#). **Graham-Field** and **Grafco** are registered trademarks of GF Health Products, Inc.

© 2015 GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, GA 30360, tel 770-368-4700, fax 770-368-2386