


LUMEX[®]

Sling Guide

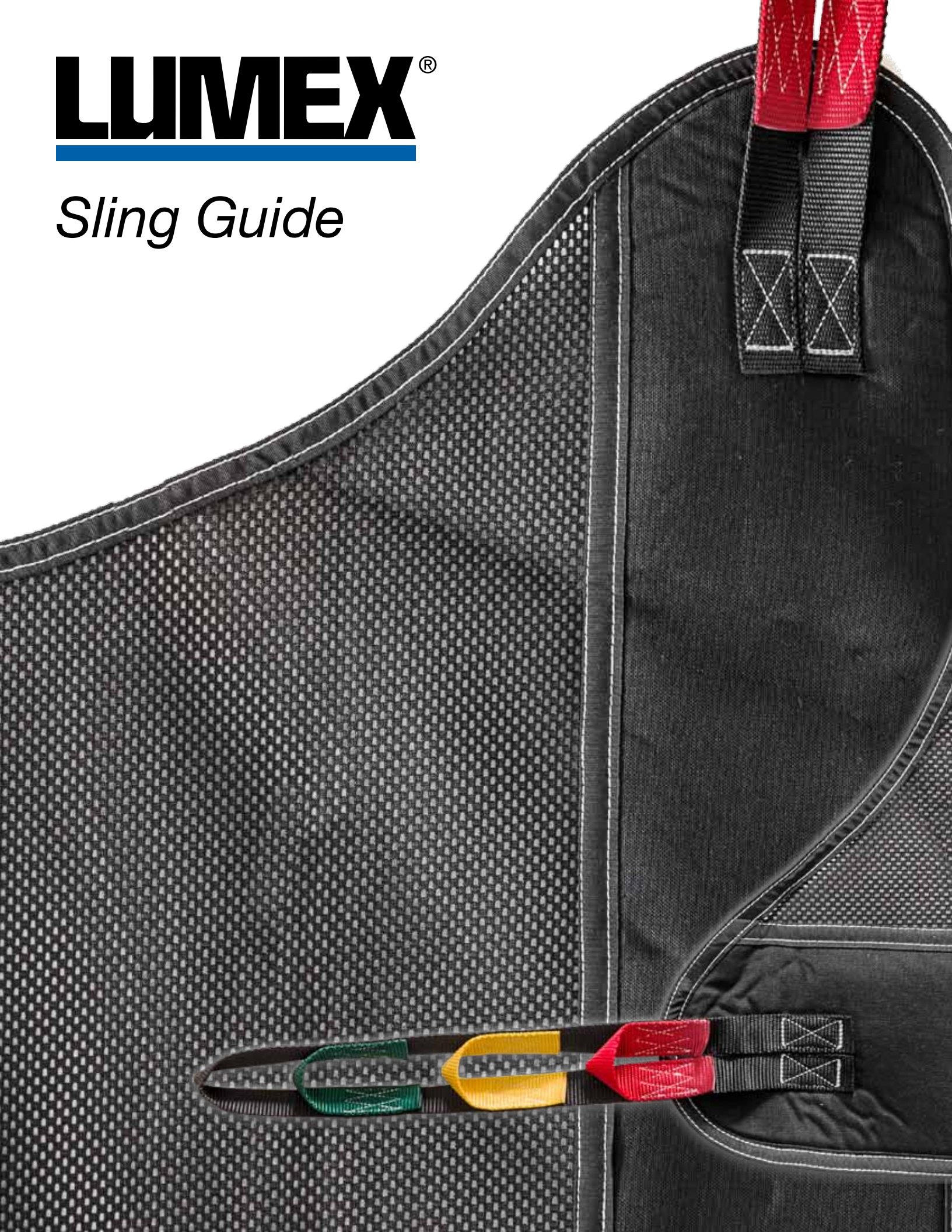

Full Body Slings

Full Body slings are the easiest sling to use while transferring a resident from a supine position (laying on ones' back). It can be used for transfers from a bed to a chair, wheelchair, or toilet, or from the floor to a bed. This sling style offers total head, neck, and thigh support. Full body slings should be used for residents who are totally or partially dependent, non-weight bearing, bariatric, or for those who have limited

head control. These are available in a solid fabric, mesh, or mesh with a commode opening. Mesh slings should be used for bathing. These slings are often left under the resident between lifts. Solid fabric slings should not be left under the resident for extended periods of time, as the fabric can retain body heat. LUMEX Full Body slings have four connection points and can be used with 2-, 4-, or 6-point spreader bars.



Style	Item	Size	Accent Color	Safe Working Load (lbs.)	Best Fits (lbs.)
Solid Fabric	F112	M	Red	450	100-195
	F113	L	Green	450	150-300
	F117	XL	Blue	450	200-400
Mesh	FM110	M	Red	450	100-195
	FM111	L	Green	450	150-300
	FM140	XXL	Yellow	600	290-600
Mesh with Commode Opening	FMC114	M	Red	450	100-195
	FMC115	L	Green	450	150-300
	FMC116	XL	Blue	450	200-400
	FMC141	XXL	Yellow	600	290-600

Universal Slings

Universal slings (also called divided leg slings) are the easiest sling to use while transferring a resident from a seated position. These can also be used for lifting from a supine position, making them suitable for most lifting situations. They are an easy-to-fit general purpose sling designed for the majority of users. They can be used for transfers from a bed to a chair, wheelchair, or toilet, chair to chair, or from the floor to a bed or chair.

This sling style is available with or without head support. They are available in solid fabric and mesh styles. Our Hoyer-styles also have a more robust padding in the back and leg for ease of

use and extra comfort, especially for those with delicate skin. Hoyer-style head support versions feature a sturdy internal plastic head shell; the shell is removable to facilitate laundering.

Universal slings should be used for residents that are totally or partially dependent, non-weight bearing, or bariatric. Those who have limited head control should use a universal sling with head support. The large opening accommodates toileting. Mesh slings should be used for bathing. LUMEX Universal slings have four or six connection points and can be used with 2-, 4-, or 6-point spreader bars.



Style	Item	Size	Accent Color	Safe Working Load (lbs.)	Best Fits (lbs.)
Solid Fabric	UP892	M	Yellow	400	140-200
	UP893	L	Green	400	200-400
	UP894	XXL	White	600	400-600
Solid Fabric with Head Support	UPH862	M	Yellow	400	140-200
	UPH863	L	Green	400	200-400
	UPH864	XXL	White	600	400-600
Mesh	MS882	M	Yellow	400	140-200
	MS883	L	Green	400	200-400
Mesh with Head Support	UMH832	M	Yellow	400	140-200
	UMH833	L	Green	400	200-400

Universal Slings, *continued*



Style	Item	Size	Accent Color	Safe Working Load (lbs.)	Best Fits (lbs.)
Hoyer Style, Solid Fabric	DSHC70000	XL	Blue	600	270-600
	DSHC70001	L	Green	500	198-350
	DSHC70002	M	Yellow	500	99-210
Hoyer Style, Solid Fabric with Head Support	DSHC70010	XL	Blue	600	270-600
	DSHC70011	L	Green	500	198-350
	DSHC70012	M	Yellow	500	99-210

Toileting Slings

Toileting Slings are designed to facilitate the toileting procedure by allowing the lowering of clothing. The sling is well padded under the arms and in the lumbar area for maximum comfort. A hook and loop belt and separate buckle strap hold the sling securely in place. Toileting slings must only be used when transferring a resident

from a seated position. They can be used for transfers from sitting on a bed to a chair, wheelchair, or toilet, or chair to chair.

LUMEX Toileting slings have four connection points and can be used with a Sit-to-Stand lift or 2-, 4-, or 6-point spreader bars.



Style	Item	Size	Accent Color	Safe Working Load (lbs.)	Torso (inches)
Deluxe Padded	TP332	M	Yellow	400	30-48
	TP333	L	Green	400	38-60
	TP334	XXL	White	600	58-73

Deluxe Sit-to-Stand Padded Slings

Sit-to-Stand Slings are designed to facilitate transfers using sit-to-stand lifts. It is only suitable for residents who have a greater degree of weight bearing ability. In addition to making seating transfers, this sling will facilitate the toileting procedure by allowing the lowering of clothing. The sling is well padded under the arms and in the lumbar area for maximum comfort. A textured belt and separate buckle strap hold the sling securely in place. Sit-to-Stand slings must only be used when transferring a resident from a

seated position. They can be used for transfers from sitting on a bed to a chair, wheelchair, or toilet, or chair to chair. LUMEX Sit-to-Stand slings have two connection points and can be used only with a Sit-to-Stand lift.

Sit-to-Stand slings can also be used in conjunction with an optional Buttock Strap and/or a Knee/Calf Strap for extra security and support.



Style	Item	Size	Accent Color	Safe Working Load (lbs.)	Torso (inches)
Deluxe Padded	DLSA1	S	Red	400	26-42
	DLSA2	M	Yellow	400	30-46
	DLSA3	L	Green	400	34-53
	DLSA4	XL	White	600	46-64
Buttock Strap	DLSA9	Std	N/A	400	N/A
	DLSA9B	Bari	N/A	600	N/A
Knee/Calf Strap	SAL1230-KB	Std	N/A	N/A	N/A