



2060R ROUND BATH STOOL: ASSEMBLY AND OPERATION INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Info: The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the Round Bath Stool. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Round Bath Stool is not properly assembled, personal injury and damage to the Round Bath Stool could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF authorized distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. The use of non-Lumex replacement parts could cause personal injury, property damage, and void the warranty.**
- ⚠ **WARNING: This product's maximum weight capacity is 300 lb, EVENLY DISTRIBUTED. DO NOT exceed the maximum weight capacity.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

INTENDED USE

The 2060R Round Bath Stool is intended as a bath aid.

UNPACKING

Remove Bath Stool components from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your GF authorized distributor for further instruction.

ASSEMBLY


1. Lay bath seat upside down with upper legs pointing up on a flat, dry surface.
2. Attach lower leg by pushing in the locking button and sliding the lower leg over the upper leg until it clicks into place. Repeat for other three legs. Ensure all legs are locked before proceeding.

- ⚠ **WARNING: Before proceeding, ensure all locking buttons protrude fully from leg attachment holes.**

Height Adjustment

Depress adjustment button and slide lower leg up or down until height adjustment button clicks into place at desired height. Repeat for other three legs. Ensure all legs are locked at the same height.

- ⚠ **WARNING: Before use, ensure all adjustment buttons protrude fully from height adjustment holes, and all legs are adjusted to the same height.**

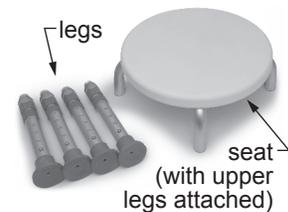

2060R Bath Stool components



2060R Bath Stool assembled



2060R Bath Stool height adjustment

Position

- ⚠ **WARNING: Always use caution when using Bath Stool on wet or slippery surfaces.**
- ⚠ **WARNING: Always ensure Bath Stool is securely positioned during use. For proper balance and stability, always sit on the middle of the seat.**
- ⚠ **WARNING: To avoid falling or tipping: Have someone help you until you know the balance points of your Bath Stool and how to avoid tipping. Do not put downward weight on one side of the Bath Stool. Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach. Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat. Never lean backward to reach an object.**
- ⚠ **WARNING: All four leg tips MUST be in contact with shower/tub floor at all times.**

MAINTENANCE

- ⚠ **WARNING: Inspect the Bath Stool weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF authorized distributor immediately for repair or appropriate replacement parts.**

To clean Bath Stool, use a mild soap and water solution and a clean cloth. Rinse with clear water, then dry with a soft cloth. Ensure that Bath Stool is well rinsed before using.

- ▲ **NOTICE: DO NOT use solvents, abrasive cleaners, or scouring pads on any part of the Bath Stool.**

LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years, and the normal wear components such as tips, wheels/casters, and handgrips are warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 770-368-4700, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. For the complete warranty for this product, please visit our website at www.grahamfield.com.



GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com. **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.

© 2009 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China