## Garantía Limitada

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by graham-field

### Features

- Soft, cushion grip reduces hand stress and fatigue
- Leg and forearm sections can be adjusted independently for a custom fit
- Vinyl-coated arm cuffs can be shaped to the user’s arm for extra comfort
- Extra-large, non-skid crutch tips provide added traction and stability
- Easy push-button height adjustment with locking collet for added security
- Grip-to-floor height adjusts in 1" increments
- Arm cuff to grip height adjusts in 1" increments
- Maximum Weight Capacity 300 lb - EVENLY DISTRIBUTED

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1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

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[www.grahamfield.com](http://www.grahamfield.com)

Follow us on [in](#) [YouTube](#) [f](#)

© 2018, GF Health Products, Inc. All Rights Reserved.  
Lumex is a trademark of GF Health Products, Inc.  
GF Health Products, Inc. is an ISO 13485 Certified Company.

GF0600009RevD18

# LUMEX®

by graham-field

## Deluxe Ortho

# Forearm Crutches

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Items 6345, 6346, 6347

### User Instructions and Warranty Information

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**grahamfield**

## Instructions For Use – Please Read All Instructions Before Use.

The chart below can be used as a guide to select height-appropriate crutches. Please consult a healthcare professional for assistance with proper height adjustment.

### Specifications

Item Number	Size	User Height	Adjustment Range / Handgrip to Floor
6345	Large	5'5" – 6'4"	33" – 42"
6346	Medium	5'0" – 6'0"	28" -37"
6347	Small	4'10" – 5'8"	21.5" – 30.5"

### Adjusting Crutch Height

- Loosen locking collet by twisting counter-clockwise.
- Adjust crutch height: Depress push button and slide lower section upward or downward to desired height.

**NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.**

- Secure locking collet by turning clockwise until tight.

### Adjust Cuff Height

- Loosen locking collet by twisting counter-clockwise.
- Adjust crutch height: Depress push button and slide lower section upward or downward to desired height.

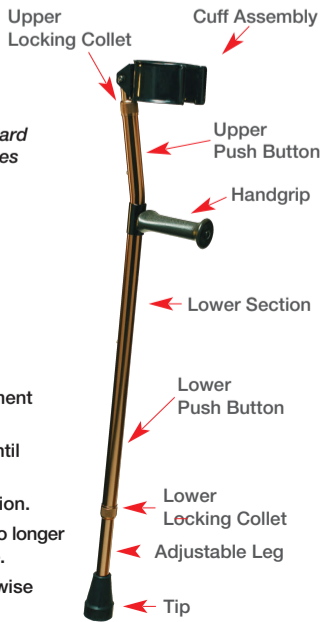
**NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.**

- Secure locking collet by turning clockwise until tight.

- Arm cuffs are adjustable; adjust both cuffs for proper fit.

### Ortho-Ease® Grip Adjustment

- Loosen – don’t remove – adjustment screw with slotted screwdriver.
- Pull grip away from the crutch until handgrip is free to rotate.
- Rotate handgrip to desired position.
- Push handgrip forward until it is no longer able to rotate and locks into place.
- Tighten adjustment screw clockwise until secure.

### Maintenance

- Wipe crutches weekly with a damp cloth.
- Grip and underarm pad may be cleaned with mild soap or detergent. DO NOT use organic solvents, such as acetone, which may damage grip.
- Never use abrasive cleaners or scouring pads on any part of the crutches.
- Check hardware and positioning weekly for tightness.
- Replace crutch tips as needed.
- To guarantee the best service from your crutches, always specify Lumex replacement parts, available from your GF Health Products distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## Safety Guidelines – Please Read Before Use

**WARNING:** This product should not be used without proper instruction from a healthcare professional.

**WARNING:** This product has a maximum user weight limitation of 300 lb.

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**WARNING:** If components are damaged or missing, contact your distributor immediately. DO NOT use substitute parts.

**WARNING:** Before use, ensure that crutch tips are not ripped, worn or missing and are properly seated and straight on each crutch.

**WARNING:** Always use caution when using this product on wet or slippery surfaces. DO NOT use crutches on stairs or steep inclines.

**WARNING:** Inspect crutches weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.

**WARNING:** Before use, ensure that push buttons protrude fully through adjustment holes.

**WARNING:** Notice for California Customers – California Proposition 65 WARNING: This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov/product](http://www.P65Warnings.ca.gov/product).

**WARNING:** Wing nuts MUST be tight at all times.

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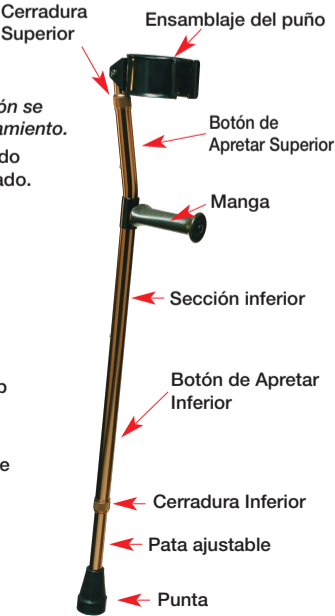
## Limited Warranty

The following warranty information is a **SUMMARY** of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

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