**LUMEX**<sup>®</sup>  
*by graham-field*

***LX5000 Allura Four-Wheel Rollator***  
***LX5000LX Allura LX Four-Wheel Rollator***  
***User Manual***

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**LX5000-INS-LAB-RevA18**

**Read this manual before assembling  
or using the Rollator.  
Save this manual for future reference.**

# CONTENTS

INTRODUCTION .....	2
PRODUCT DESCRIPTION .....	2
INTENDED USE .....	3
SAFETY GUIDELINES—PLEASE READ BEFORE USE .....	3
WARNING / CAUTION SUMMARY .....	3
RAMPS AND INCLINES .....	4
CURBS AND STEPS .....	4
UNPACKING .....	5
ASSEMBLY .....	6
FRAME .....	6
SEAT .....	6
BACKREST .....	6
HANDLE HEIGHT ADJUSTMENT .....	7
STORAGE BAG INSTALLATION .....	7
OPERATION .....	8
CLEANING AND MAINTENANCE .....	9
TRANSPORT AND STORAGE .....	9
SPECIFICATIONS .....	10
LIMITED WARRANTY .....	11

## INTRODUCTION

This manual contains assembly, operation and maintenance instructions for your rollator. Read the entire manual carefully before using your rollator, and refer to it during use if you have questions.

## PRODUCT DESCRIPTION

## INTENDED USE

The intended use of the LX5000 Allura Four-Wheel Rollator / LX5000LX Allura LX Four-Wheel Rollator is as a rollator, a walking aid with wheels, manipulated by both arms.

**⚠ WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.**

## SAFETY GUIDELINES—PLEASE READ BEFORE USE

The safety statements presented in this chapter refer to the basic safety information that the rollator user shall pay attention to and abide by. There are additional safety statements in other chapters or sections, which may be the same as or similar to the following, or specific to the operations. Please note the following special statements, used throughout this manual, and their significance:

- ⚠ WARNING:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.
- ⚠ CAUTION:** Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor or moderate personal injury.
- ▲ NOTICE:** Indicates a potential hazard or unsafe practice that, if not avoided, could result in product or property damage.

*Info: Provides application recommendations or other useful information to ensure that you get the most from your product.*

## WARNING / CAUTION SUMMARY

- ⚠ WARNING:** Important! Read and understand these instructions before assembling or using the rollator. Ensure that the rollator is assembled as described in the following pages before use. If the rollator is not properly assembled and adjusted, personal injury and/or damage to the rollator could result.
- ⚠ WARNING:** DO NOT use this product without proper instruction from a healthcare professional.
- ⚠ WARNING:** GF Health Products, Inc. ("Graham-Field") assumes no responsibility for any damage or injury caused by improper assembly or use of this product.
- ⚠ WARNING:** If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.
- ⚠ WARNING:** Inspect the rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your authorized GF distributor for repair and/or appropriate replacement parts.
- ⚠ WARNING:** DO NOT operate the rollator on streets or roadways.
- ⚠ WARNING:** DO NOT operate the rollator on hilly or rough terrain, sand, wet, slippery or icy surfaces, or surfaces with impaired traction. Ensure that pathway is clear of all obstacles.
- ⚠ WARNING:** The rollator does not offer seating or occupant restraint equivalent to the seat provided in a motor vehicle! To increase your safety while traveling in a motor vehicle, always transfer to the vehicle seat and use the restraint provided by the vehicle manufacturer.

- ⚠ WARNING: DO NOT tie down or attach anything to the wheels. This could cause tipping and possibly result in injury or damage to the rollator.**
- ⚠ WARNING: Ensure that the rollator is on a stable, level surface and engage handbrakes before and during transfer or using an elevator.**
- ⚠ WARNING: Unauthorized modification or the use of non-Lumex® replacement parts could change the structure of the rollator, void the warranty, and create a hazardous condition resulting in serious personal injury.**
- ⚠ WARNING: DO NOT use the rollator on stairs or escalators.**
- ⚠ WARNING: Notice for California Customers - California Proposition 65 WARNING: This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov/product](http://www.P65Warnings.ca.gov/product).**
- ⚠ WARNING: GF Health Products, Inc. specifically disclaims responsibility for any personal injury or property damage which may occur during any use which does not comply with federal, state, or local laws or ordinances.**

## **RAMPS AND INCLINES**

- ⚠ WARNING: DO NOT leave the rollator on an incline.**
- ⚠ WARNING: DO NOT turn the rollator while going downhill, as the rollator could become unstable or tip over.**
- ⚠ WARNING: DO NOT attempt any incline or decline greater than six degrees (10% grade, or one foot of rise or fall per ten feet of ramp length).**

Ramps and inclines should only be negotiated with the assistance of an attendant. To attendant: Practice with a healthcare professional first before attempting ramps or inclines. Always inspect the ramp for hazards such as holes, slippery or uneven surfaces, etc. before starting up or down. If you can not see the entire ramp, ask someone to inspect it for you.

## **CURBS AND STEPS**

- ⚠ WARNING: Never attempt to negotiate curbs or steps with your rollator without the assistance of an attendant.**
- ⚠ WARNING: Never attempt to negotiate stairs or escalators with your rollator under any circumstances.**


Curbs, steps and stairways are dangerous obstacles that confront the rollator user. When you encounter curbs, find a way around, or use the ramps now available in most locations. If you encounter stairs or escalators and there is no ramp available, avoid the stairs or escalators by utilizing the disabled designated elevators now required in most locations. Curbs and steps should only be negotiated with the assistance of an attendant.

## UNPACKING

Unless the rollator is to be used immediately, retain box and packaging materials for storage until use is required.

1. Check for obvious damage to the carton or its contents. If damage is evident, please notify the carrier and your GF authorized distributor.
2. Remove all loose packing from the box.
3. Carefully remove all the components from the carton.

Each box should contain the following components (see picture below):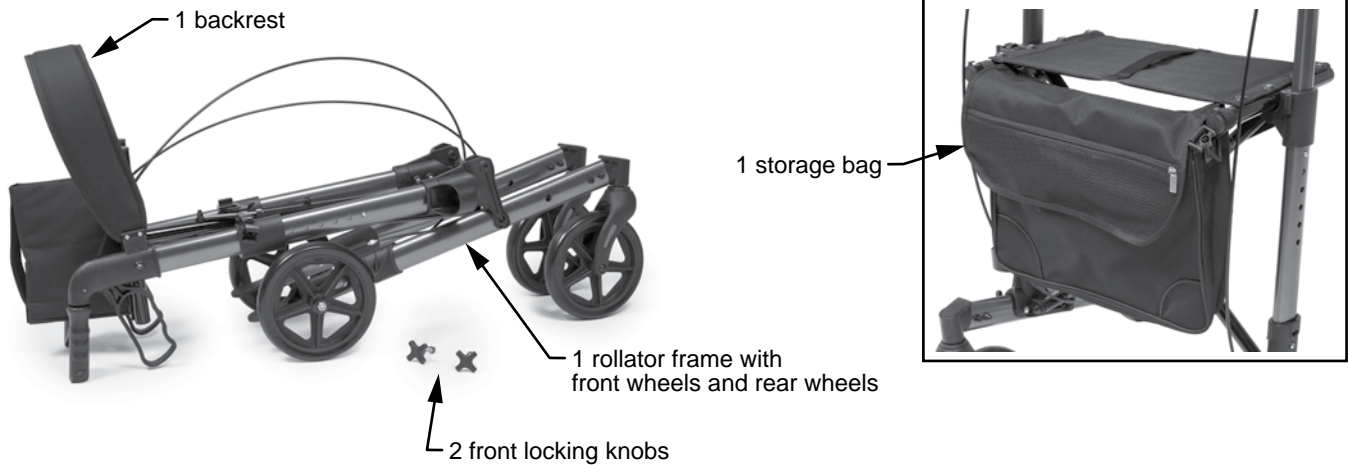


- 1 rollator frame with front wheels and rear wheels
- 1 backrest
- 1 storage bag
- 2 front locking knobs

## ASSEMBLY

### FRAME

1. Position the rollator in upright position.
2. Install both front locking knobs. Turn them clockwise to tighten.

**⚠ WARNING: Ensure front locking knobs are securely installed before proceeding.**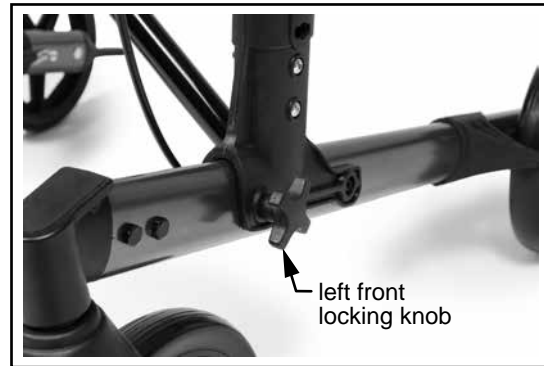


**Lock Frame into Place**

### SEAT

Push down on both the left and right side of the seat to securely lock the seat support bars into place.

***Info: You will hear a click when the seat locks into position.***

**⚠ WARNING: Avoid putting your fingers under the seat support bars when locking the seat into position.**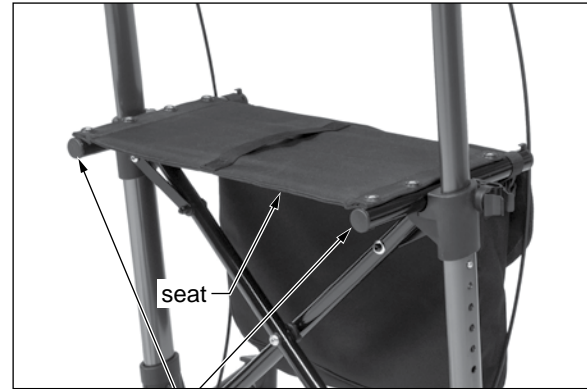


**Lock Seat Into Place**

### BACKREST

The backrest has a slot on each side where the frame key slides in to secure the backrest to the frame. Slide the frame key into the backrest slot on both sides as shown at right.

**⚠ WARNING: Ensure backrest is securely installed before proceeding.**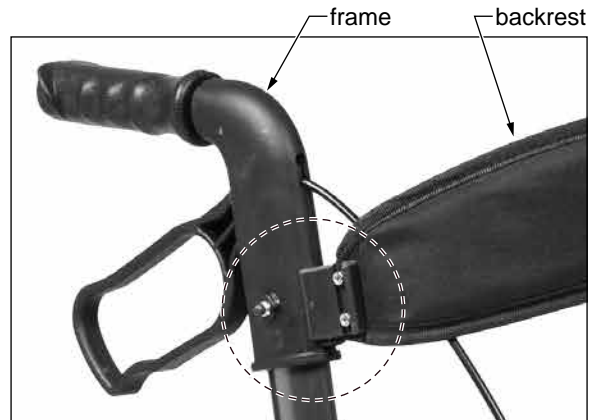

**Backrest Installation**

## HANDLE HEIGHT ADJUSTMENT

1. The rollator handle height can be adjusted very easily. To adjust the handle height, press the push button on the side of the rollator and adjust to the best height for the user. Ensure that the push button locks in one of the adjustment holes to secure its position.
2. Repeat for other side.


***Info: A recommended guideline for proper handle height positioning is to position the handle at wrist height of the user. Your healthcare professional, distributor or caregiver should assist you in finding the best handle height for you.***

Handle Height Adjustment

## STORAGE BAG INSTALLATION

Use the storage bag straps to install the storage bag on the seat support bars. The storage bag can be conveniently removed to place items and then reattached, no tools needed.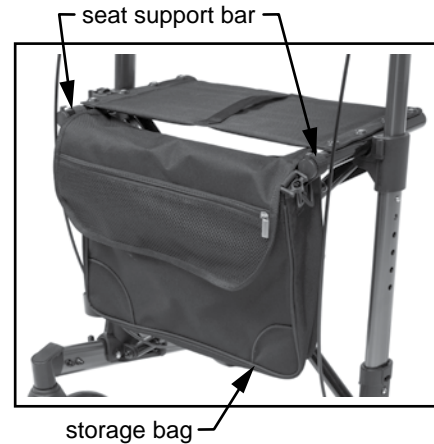

Storage Bag Installation

## OPERATION

- ⚠ **WARNING:** Read the safety section and operating instructions in this manual before use.
- ⚠ **WARNING:** Before using the rollator, ensure that the wheels are securely attached to the frame, that the seat is fully in its locked position, and that the push buttons protrude fully through the leg height adjustment holes, and the backrest is securely installed.

Before proceeding, ensure that all instructions in the previous section, **ASSEMBLY**, have been followed, and that you have read the **SAFETY** section of this manual.

### Rollator Setup

The photo at right shows the rollator. To keep the backrest away from your hands, position the backrest away from the handles as shown at right.

### Walking With Your Rollator

1. Move your rollator slowly forward and always be sure that the rear wheels are a few inches ahead of your body.
2. Select a pace that is comfortable for you.

### Stopping With Your Rollator

The rollator has loop locking handbrakes. To slow down, squeeze the handbrakes upward toward the handles. To hold the rollator in a stationary position, push the handbrakes down, away from the handles, until the handbrakes lock into place; the rear wheels will be locked in place. To release, pull the handbrakes upward toward the handles until the rear wheels move freely.

### To Sit in Rollator

1. Push the handbrakes down, away from the handles, until the handbrakes lock into the parked position.
2. Test the wheels before sitting down to ensure the rollator will not roll.
3. Before sitting, touch the back of your legs to the edge of the seat to ensure proper positioning of the rollator.
4. Hold onto handles of rollator if necessary to help you lower your body onto the seat.

- ⚠ **WARNING:** Distribute your body weight evenly on the rollator.
- ⚠ **WARNING:** DO NOT lean on one side of the rollator.
- ⚠ **WARNING:** Ensure that handbrakes stop rollator and lock in place appropriately before occupying or operating rollator. If your handbrakes are not operating correctly, please see your GF authorized distributor immediately for service.

## CLEANING AND MAINTENANCE

Use a clean, soft cloth and warm water to clean the rollator frame, handles, and wheels. Rinse and air dry.

**▲ NOTICE: Do NOT use solvents and other cleaning solutions to clean the frame or the fabric of this product.**

Check fit and tightness of all nuts, bolts and knobs weekly.

**⚠ WARNING: Inspect the rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF authorized distributor for repair and/or appropriate replacement parts.**

**⚠ WARNING: The use of non-Lumex® replacement parts will void the warranty, and could create a hazardous condition resulting in serious personal injury.**

## TRANSPORT AND STORAGE

The rollator is designed for easy folding and storage.

### To Fold for Transport

1. Pull the strap on the middle of the seat up to release the seat from the seat support bars.
2. Push the handles together; the rollator will fold easily and be ready for transport.

**Prepare for Transport / Storage**

### To Store

1. Adjust the handle height to the lowest position.
2. Pull the strap on the middle of the seat up to release the seat from the seat support bars.
3. Lay the rollator on the floor to fold the front wheels.
4. Remove the front locking knobs: Turn the lock knobs counterclockwise, remove them, and store them with the rollator. Fold the front wheels as shown at right.
5. It is not required that you remove the storage bag from the rollator.
6. For long periods of storage, remove the backrest from the frame by sliding the frame key out from the backrest slot on each side.

## SPECIFICATIONS

Dimension ↓	Model Number	
	LX5000	LX5000LX
Width Inside Handgrips	17" (431 mm)	17" (431 mm)
Overall Width	26" (665 mm)	26" (665 mm)
Overall Width Folded Upright	13" (330 mm)	13" (330 mm)
Overall Depth	28.5" (723 mm)	28.5" (723 mm)
Depth Folded Down Flat	36.4" (925 mm)	36.4" (925 mm)
Minimum Handle Height	33" (836 mm)	37" (935 mm)
Maximum Handle Height	38" (956 mm)	42" (1060 mm)
Seat to Floor Height	21.8" (556 mm)	26" (660 mm)
Approximate Height of Intended User	5'4" - 6'2"	6'2" - 6'8"
Maximum Weight Capacity, EVENLY DISTRIBUTED	300 lb (136 kg)	300 lb (136 kg)
Product Weight	18.5 lb (8.4 kg)	19.1 lb (8.7 kg)
Seat Width	18" (460 mm)	18" (460 mm)
Seat Depth	9.4" (239 mm)	9.4" (239 mm)
Wheel Size	8" x 1.5" (201 mm x 38 mm)	8" x 1.5" (201 mm x 38 mm)

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. The warranted components and time period are set forth below:

Frame: 5 years

All other components: 1 year

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto.

The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

**1.770.368.4700**

Information contained herein is subject to change.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

Follow us on   

© 2017, GF Health Products, Inc. All Rights Reserved.  
Graham-Field and Lumex are trademarks of  
GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140  
GF Health Products, Inc. is an ISO 13485 Certified Company.  
Manufactured for GF Health Products, Inc.  
Made in China

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© 2017, GF Health Products, Inc. Todos los derechos reservados.
GF Health Products, Inc. es una empresa certificada ISO 13485.
Fabricado para GF Health Products, Inc.