## Garantía Limitada

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by graham-field®

### Features

- Provides assistance to those who have difficulty stepping over the bathtub walls
- Comfortable, cushioned seat and backrest are water-tight and easy to clean
- Anodized aluminum frame is lightweight, durable, and rust resistant
- Drainage holes in seat minimizes water build up
- Reversible backrest to accommodate any bathroom
- Support Arm aids in lateral transfer
- Maximum weight capacity 350lbs EVENLY DISTRIBUTED
- Designed to meet the requirements of HCPCS code: E0247/E0248

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1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

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[www.grahamfield.com](http://www.grahamfield.com)

Follow us on   

© 2017, GF Health Products, Inc. All Rights Reserved.

Lumex is a trademark of GF Health Products, Inc.

GF Health Products, Inc. is an ISO 13485 Certified Company.

GF1700017RevB18

# LUMEX®

by graham-field®

## Padded Transfer Bench de Banco de Transferir Acolchonado

Items 7955KD-1, 7955KD-2, and 7956KD-1

### User Instructions and Warranty Information

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Important: Lea Antes de Usar

## Limited Warranty

The following warranty information is a **SUMMARY** of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

## Instructions for Use – Please Read All Instructions Before Use.

### Attaching the Leg Extensions (if not attached already)

1. Place frame on its side on a level surface.
2. Attach leg extensions with suction feet to the frame on the same side as where the support arm will be. Attach leg extensions with flange tips to frame on opposite side of frame.
3. The transfer bench legs with suction feet should be the two legs that are to be placed inside the tub.

NOTE: Ensure leg extension push buttons protrude fully through plastic guide holes.

### Attaching the Backrest

1. Determine the side on which the backrest should be mounted by knowing which direction the user will face when sitting on the bench in the tub.
2. Position the bench upright on a level surface.
3. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
4. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.
5. Slide back support onto the seat support tubes until the push buttons protrude fully.

NOTE: Ensure seat support tubes buttons protrude fully through channel guide holes.

CAUTION: Backrest is not designed to support the full weight of the user.

CAUTION: Ensure backrest is facing down correctly before inserting to avoid improper installation.

### Attaching the Support Arm

1. To attach the support arm, position the seat arm against the leg frame and slide the armrest tubing into the tube openings until it clicks into place. Make sure that the push buttons fully protrude through the holes in the bracket.

NOTE: An audible "click" will be heard.

2. To remove the support arm, depress the push buttons and pull up.

NOTE: The end with the Support Arm and Suction Feet is the side that should be inside the tub.

### Adjusting Leg Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for other 3 legs.

CAUTION: Ensure that all four legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

### Using Your Transfer Bench

NOTE: Be sure to move slowly throughout use to prevent injuries.

NOTE: Children or those who need assistance should NOT be left unattended during use of this product.

#### From a seated position (such as a wheelchair):

1. Place your wheelchair next to the bench, facing the front of the bathtub.
2. Engage wheelchair locks.
3. Transfer your torso to the transfer bench first, and then lift your legs into the tub. Move slowly to prevent injuries.
4. After bathing, lift your legs out of the tub first, then your torso.

#### From a standing position:

1. Sit on the side of the transfer bench that is outside of the tub.
2. Extend your arms to grab the support arm of the bench and slide your body over onto the bench. Lift each leg over the tub slowly.
3. Position yourself to a comfortable position in center of the bench.
4. After bathing, lift your legs out of the tub first, then your torso slowly.

## Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

### WARNING

**WARNING:** DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** Users with limited physical capabilities should be supervised or assisted when using the transfer bench.

**WARNING:** Item has a maximum weight capacity of 350 lb. EVENLY DISTRIBUTED

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

**WARNING:** Leg extensions with rubber tips MUST be in contact with the floor at all times.

**WARNING:** Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

**WARNING:** Make certain ALL screws and nuts are tight at all times.

**WARNING:** Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

**WARNING:** The transfer bench seat MUST be installed before sitting on the transfer bench.

**WARNING:** Notice for California Customers – California Proposition 65 **WARNING:** This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov/product](http://www.P65Warnings.ca.gov/product)

**WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately.

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