



LCMP100 : PUMP, SELECT SERIES CONVERTIBLE MATTRESS SYSTEM; OPERATION INSTRUCTIONS


SAVE THESE INSTRUCTIONS FOR FUTURE USE

SAFETY GUIDELINES - PLEASE READ BEFORE USE

- ⚠ **WARNING:** Important: Read and understand these instructions before using the pump. Ensure that the pump is used as described in the following paragraphs before use. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the pump is not properly used, personal injury and/or property damage could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your GF authorized distributor immediately. **DO NOT** use substitute parts.
- ⚠ **WARNING:** Cancer and Reproductive Harm - www.p65warnings.ca.gov.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

INTENDED USE

The Lumex Select Series Convertible Mattress System Pump is designed to aid in the treatment and prevention of pressure ulcers while optimizing user comfort.

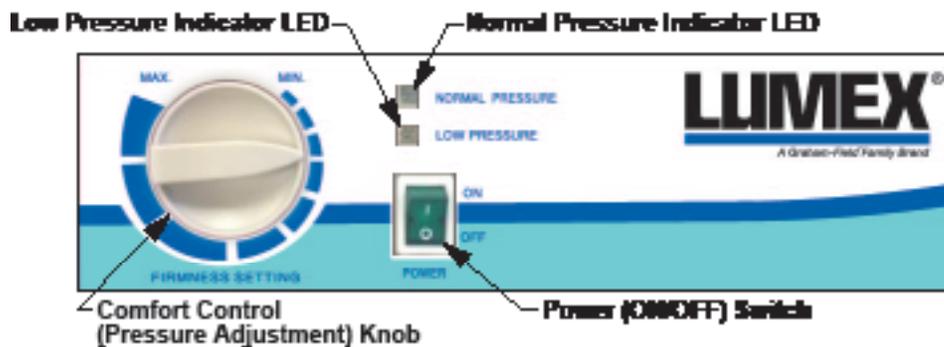

LCMP100 Pump, Select Series Convertible Mattress System

OPERATION

- ⚠ **DANGER:** **DO NOT** use the pump in the presence of flammable materials or chemicals. This could cause an explosion.
- ⚠ **DANGER:** There are no serviceable parts in the pump. **DO NOT** attempt to open it; electric shock could result.
- ⚠ **WARNING:** Read the user manual included with the mattress system before using the pump, and refer to it if you have questions about this product's use.

1. Place the pump on a flat horizontal surface or suspend it on the end of bed frame with the built-in hanger.

2. Place the air mattress on top of the bed mattress. Ensure the air mattress lies completely flat without any kinks. Place "FOOT END" of air mattress toward foot end of bed.
3. Connect air hoses to pump and air mattress — hose ends with fittings connect to air mattress. Ensure air hoses are not kinked and will not be pinched by any articulated bed mechanisms.
4. Ensure green Power switch is in the OFF position, then plug pump into a properly grounded wall outlet. (Verify power to this outlet is not controlled by a wall switch).
5. Press green Power switch to the ON position. Air will begin to flow into mattress.
6. Yellow LED illuminated indicates that air pressure is low and air mattress is not fully inflated; the green LED will automatically illuminate when proper air pressure has been reached and the air mattress is ready for patient placement.

Info: Each time the air mattress is inflated, set the Comfort Control Knob pressure to "MAX.", after which the user can adjust the mattress to desired firmness.

MAINTENANCE AND CLEANING

Refer to user manual included with mattress system for this information.



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com



© 2017, GF Health Products, Inc. All Rights Reserved.
Lumex is a trademark of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.
Manufactured for GF Health Products, Inc.
Made in Taiwan