Garantía Limitada

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LUMEX[®]
by graham-field

Features:

- Lightweight, durable, rust-resistant frame
- Water-tight cushioned seat and backrest for comfort
- Available with or without support arms
- Seat height adjusts in 1" increments for various user heights
- 300 lb weight capacity EVENLY DISTRIBUTED

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www.grahamfield.com

Follow us on 

© 2019, GF Health Products, Inc. All Rights Reserved. Lumex is a trademark of GF Health Products, Inc. GF Health Products, Inc. is an ISO 13485 Certified Company.

GF0600029V019

LUMEX[®]
by graham-field

Platinum Collection Padded Bath Seat

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User Instructions and Warranty Information

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Limited Warranty

The following warranty information is a **SUMMARY** of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

Instructions for Use – Please Read All Instructions Before Use.

Setup

1. Remove all components from the box.
2. Place the seat frame face down (padded seat on floor).

Installing and Adjusting Legs

1. Locate the 4 legs.
2. Press push button* on plastic leg sleeve end and slide the legs onto the frame.
3. Press push button* on the leg adjustment end and slide leg extension up or down to desired height.
4. Repeat STEPS 1 through 3 for all the legs.
5. Flip the chair over so that all 4 legs are level on the floor.

Note: Ensure all four legs are adjusted to the same height before use.

Installing Padded Backrest

1. Locate the backrest and two backrest support tubes.
2. Slide the backrest support tubes through the two channel guides on the bottom of the seat.
3. Press the push button* until it clicks through the hole in the channel guide.
4. Repeat Steps 2 and 3 for the other tube
5. Align the backrest tubes with the backrest support tubes and slide them onto the support tubes.
6. Press both push buttons simultaneously and slide the backrest down until the push buttons protrude through the holes.

*Note: Ensure push buttons fully protrudes through the holes. An audible "click" will be heard when the button is fully protruded.

Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Item has a maximum weight capacity of 300 lb. EVENLY DISTRIBUTED.

WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

WARNING: ALWAYS inspect the bath seat to ensure that it is properly positioned and stable before using. DO NOT use if bath seat is wobbly or unstable.

WARNING: If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. DO NOT use substitute parts.

WARNING: Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips IMMEDIATELY.

WARNING: DO NOT use this as product as a climbing device.

WARNING: To avoid falling or tipping: Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping. Do not put downward weight on one side or corner of the Bath Seat. Do not put downward weight on the backrest. Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach. Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat. Never lean backward to reach an object.

WARNING: All four leg tips MUST be in contact with shower/tub floor at all times.

WARNING: Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes BEFORE use.

WARNING: Only use GF Health Products, Inc. accessories with this product.

WARNING: This product should only be used with tub floors wider than 18 3/4 inches.

WARNING: Users with limited physical capabilities must be supervised or assisted when using the bath seat.

WARNING: Make certain ALL screws and nuts are tight at all times.

△WARNING:
Cancer and Reproductive Harm - www.P65Warnings.ca.gov

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ADVERTENCIA: Utilice solamente accesorios de GF Health Products, Inc. con este producto.