

**5782RG UNIVERSAL IV POLE HOLDER:
INSTALLATION INSTRUCTIONS FOR
FR565G, FR565WG, FR566DG, FR566G, 574G, FR574G,
FR577RG, FR587W AND FR587WD RECLINERS**

SAVE THESE INSTRUCTIONS FOR FUTURE USE

Tools required: 7/16" wrench or ratchet wrench with 7/16" socket

- ⚠ **WARNING:** Important! Read and understand these instructions before installing the Universal IV Pole Holder. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If you are unable to perform this installation as described, please contact a licensed professional for installation. If the IV Pole Holder is not properly installed, personal injury and property damage could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your Graham-Field authorized distributor immediately. **DO NOT** use substitute parts.
- ⚠ **WARNING:** Cancer and Reproductive Harm - www.p65warnings.ca.gov
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

INSTALLATION

Installation on FR565G, FR565WG, FR566DG, FR566G, FR577RG, FR587W, and FR587WD

1. Select the recliner frame leg to attach the 5782RG where the IV pole will not interfere with the operation of a tray or other attachment. The IV Pole Attachment should be mounted on the **FRONT RIGHT LEG** or **LEFT REAR LEG** of the recliner.
2. Remove the bolt and nut securing the caster from the recliner leg that will receive the 5782RG and discard them.
3. Remove the bolts, washers and nuts from the 5782RG.
4. Place the attachment clamps around the leg with the hinge pin inserted into the clamp hinge loops and insert the long carriage bolt through the clamps, leg and caster, as shown in fig. 1. Install a washer and locknut finger tight. **Ensure the carriage bolt is inserted from *inside* of recliner as shown in fig. 2.**
5. Insert the short carriage bolts through the clamps as shown in fig. 2 and install the remaining washers and locknuts finger tight. **Ensure the carriage bolt is inserted from *inside* of recliner as shown in fig. 2.**
6. Tighten all nuts securely with the wrench or socket. Verify that there is no looseness in the completed assembly.
7. The completed installation will look as shown in fig. 3.



Info: The 5782RG Universal IV Pole Holder is designed to be used with the Lumex Telescoping IV Pole (Model 2101A or 2100A).



Installation on 574G and FR574G Recliners with 1" Square Tubing Legs

1. Remove the bolts, washers and nuts from the 5782RG. Set aside the long carriage bolt and one washer and locknut, which will not be required.
2. Apply the adhesive-back pads to the aluminum angle as shown in fig. 4.
3. Place the attachment clamps around the right frame leg with the hinge pin inserted into the clamp hinge loops and aluminum angle with pads between the clamp and leg as shown in fig. 5, and insert the short carriage bolts through the clamps. Install the washers and locknuts finger tight.

Info: The carriage bolts may be inserted in either direction.

4. Position the attachment at the desired height on the leg while making certain that the clamp hinge loops are in contact with the retaining ring on the hinge pin, and tighten all nuts securely with the wrench or socket. Verify that there is no looseness in the completed assembly.
5. The completed installation will look as shown in fig. 6.

⚠ WARNING: Ensure that the IV Pole Holder is installed as described before use. If the IV Pole Holder is not properly installed, personal injury and property damage could result.

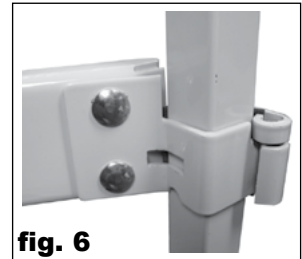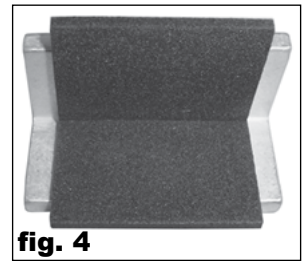

MAINTENANCE

⚠ WARNING: Inspect IV Pole Holder before each use and tighten as needed to ensure secure installation.

WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original purchaser only that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF’s customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for three (3) years. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com

Follow us on   

© 2011, GF Health Products, Inc. All Rights Reserved.
Graham-Field and Lumex are trademarks of
GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. is an ISO 13485 Certified Company.
Manufactured for GF Health Products, Inc.
Made in Taiwan