




# Decannulation Stopper

## Item: 3255 Series

### Directions for use:

- The Decannulation Stopper is for use only with a trachea tube. Proper fit is attained when the conical stopper goes partially into but not completely through the inner cannula.
- Do not use without the string as packaged. The string is attached for safety purposes.
- Do not attempt to change or modify the shape or size of the stopper.
- Use only cold sterilization methods.

 **CAUTION: Federal law restricts this device to sale by or on the order of a physician.**

Manufactured for: GF Health Products, Inc., One Graham-Field Way,  
Atlanta, Georgia 30340-3140 • [www.grahamfield.com](http://www.grahamfield.com) • Made in U.S.A.

3255-INS-LAB-RevB19