# Directions For Use

## Application

- Clean skin thoroughly prior to the application of electrodes. Electrodes do not stick well if any lotion, oil, makeup, dirt, etc. is left on the skin.
- Insert device's lead wire pins into the electrode wire connectors.
- Remove the electrodes from the protective liner and apply firmly to the skin.

## Removal

- Lift electrode edges and peel. Do not pull on the lead wires when removing electrodes from the skin.
- Place the electrodes on the liner and disconnect the lead wire by twisting and pulling at the electrode wire connectors.

## Care and Storage

- Between uses, store the electrodes in the bag provided in a cool, dry place.
- The life of the electrodes varies depending on skin conditions, storage temperature, amount of use, type of stimulation, and stimulation site. Electrode life may be extended by carefully following the directions above.

## Important

- Electrodes are intended for single patient use only.
- Do not apply to broken skin. Should a skin rash occur, discontinue use and contact your physician. Do not stimulate while driving a motorized vehicle or operating machinery. Do not exceed 0.1 watts/cm<sup>2</sup>.

## Contraindication

- Electrodes must not be used for stimulation on persons with cardiac pacemakers, implanted defibrillators, or other implanted metallic or electronic devices.

## Precautions

- Some people may experience skin irritation or hypersensitivity due to the electrical stimulation, the electrode materials, or the electrical conductive medium (gel).
- Electrodes should be kept out of the reach of children.
- The size, shape and type of electrodes may affect the safety and effectiveness of your electrical stimulation treatments. Using electrodes that are too small could result in discomfort or skin burns. Use the electrodes as prescribed by your physician.

## Adverse Reactions

- Skin irritation and burns beneath the electrodes have been reported with the use of electrodes applied to the skin.
- Headache and other painful sensations have been reported during or following the application of electrical stimulation to the head, face, and near the eyes.

## Warnings

- The long term effects of prolonged use of cutaneous electrodes for electrical stimulation are unknown.
- Electrodes should not be applied over the neck region. Severe spasm of the muscles may occur and the contractions may be strong enough to close the airway or cause difficulty breathing. Stimulation using electrodes placed over the neck could also have adverse effects on the heart rhythm or blood pressure.
- Electrodes should not be applied across the chest because the introduction of electrical current in the chest may cause rhythm disturbances to the heart.
- Electrodes should not be applied over, or in proximity to, cancerous lesions.
- Electrodes should not be shared with other persons. Each person should have their own set of electrodes; otherwise, undesirable skin reactions may occur.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.
- Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**grahamfield** SELF-ADHESIVE ELECTRODES

# T.E.N.S. & Muscle Stimulator Electrodes

- **PRE-WIRED**
- **REUSABLE**

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Manufactured for:  
**GF Health Products, Inc.**  
**Atlanta, GA 30340-3140**  
**Made in Taiwan**  
**www.grahamfield.com**

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