3-Wheel Cruiser

Crucero de 3 ruedas

Items 609201P, 609201M, 609201B

SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. Each individual should **ALWAYS** consult with their physician or other health care professional to determine proper adjustment and usage.
2. The brakes should be in the locked position when pausing for extended time period.
3. All wheels **MUST** be in contact with the floor at all times during use. This will ensure the rollator is properly balanced.
4. Maximum Weight Capacity 250 lb - **EVENLY DISTRIBUTED**. **ALWAYS** observe the user weight limitation on the hangtag of this product and on the chart referenced at left.
5. Ensure handgrips **DO NOT** twist on rollator handle - otherwise damage or injury may occur.
6. **ALWAYS** test to see that the rollator and attachments are properly and securely locked in the open position **BEFORE** using.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. **ALWAYS** use caution when using the rollator on wet and slippery surfaces. **DO NOT** use the rollator on stairs and steep inclines.
9. **DO NOT** hang anything from the frame of the rollator. Items should be placed in the basket or tote bag.
10. The rollator basket has a weight limitation of 11 lbs (5 kg). The tote bag has a weight limitation of 10 lbs (4.5 kg). Items placed in either the basket or the bag should not protrude from the basket or bag.
11. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use **EXTREME** caution when reaching for any object.
12. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
13. Inspect rollator weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
14. If the hand brakes fail to stop the Rollator, contact your GF Health Products, Inc. authorized distributor immediately for adjustment or replacement.

GENERAL WARNINGS

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- **DO NOT** use the rollator as a wheelchair or transport device. Rollators are not intended to be propelled while seated.
- If components are damaged or missing, contact your distributor immediately. **DO NOT** substitute parts.
- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all hand and height adjustments are secure, and that wheels and moving objects are in good working order before using this rollator.
- When using the rollator in a stationary position, the hand brakes **MUST** be locked.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.
- **DO NOT** attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change to the weight distribution of the rollator and may cause the rollator to tip, resulting in injury or damage.

 **WARNING:** Cancer and Reproductive Harm - www.p65warnings.ca.gov.

INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

ATTACHING BASKET, POUCH OR TOTE BAG

Basket:

Install the basket underneath the seat. On some rollator models, the basket is designed to fit underneath the seat or in the front by positioning the hooks on the front seat support frame.

Pouch and Tote Bag:

Hang the tote bag between the rollator frame tubes and secure with snap button flaps.

MAINTENANCE

- For dust and dirt, wipe product gently with damp, soft cloth.
- Inspect wheels periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Verify operation of the brakes and have them adjusted if necessary by your GF Health Products, Inc. authorized distributor.
- Inspect product weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.
- Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair or other debris that could interfere with free wheel operation.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

INSTRUCTIONS FOR USE

Please consult a healthcare professional for the proper adjustment.

OPENING THE ROLLATOR

1. Remove Rollator and other components from carton.
2. Spread handgrips apart to open frame completely.
3. Push down locking mechanism to secure the frame.

NOTE: Ensure locking mechanism is locked and secure before each use.

FOLDING THE ROLLATOR

1. Pull up on the locking mechanism knob towards the rear part of the frame to release the lock.
2. Push the two handgrips together.

USING THE HAND BRAKE

Brakes should be used in one of the following methods:

1. To Engage Brake: Push down on lower part of hand brake.
2. To Remain Stationary: Hold the hand brake up.
3. To Continue Mobility: Release the brake. To unlock, pull up on the top portion of the brake handle.

CAUTION: Always lock hand brakes when user is immobile.

ADJUSTING THE HEIGHT

1. Consult with healthcare professional for proper adjustment.
2. Install the adjustment knob into one of the adjustment holes by turning clockwise.
3. Repeat steps 1-2 for the other side.

Make sure that the height adjustment is at the same height for both handles to ensure safety and proper use.

NOTE: After adjusting the height, the brakes of the rollator must be checked and, if necessary, adjusted.

LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling **(678) 291-3207**, sending a fax request to **(770) 368-2386** or by e-mailing a request to **cs@grahamfield.com**. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: **1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges.** Other exclusions apply. Additional limitations relative to this warranty may be found on www.grahamfield.com Terms and Conditions page. State laws governing the warranty of this product may provide more expanded rights than those set forth in this warranty.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

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Features

- Improves mobility and allows for greater independence for the user on the go
- Robust & easy to fold design makes rollator easy to use, fold and store for convenient transport
- Ergonomic hand grips and hand brakes allow user to operate with ease
- Complete with pouch for storage of personal items
- 8" wheels are ideal for indoor and outdoor use
- Maximum Weight Capacity 250 lb - EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2019, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com