Bariatric Steel Crutches

Muletas de Acero Bariátricas

Items 3614A & 3615A

SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 600 lb. - **EVENLY DISTRIBUTED.**
3. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
4. If components are damaged or missing, contact your distributor immediately. **DO NOT** use substitute parts.
5. Before use, ensure that crutch tips are not ripped, worn or missing, are properly seated and straight on each crutch.
6. Always use caution when using this product on wet or slippery surfaces. **DO NOT** use crutches on stairs or steep inclines.
7. Inspect crutches weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
8. Before use, ensure that push buttons protrude fully through adjustment holes.
9. Proper height adjustments must be made to both the leg extension and the crutch handle. A dealer or therapist should aid in the proper placement for maximum balance and support.
10. Wing nuts **MUST** be tight at all times.

GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

MAINTENANCE

- Wipe crutches weekly with a damp cloth.
- Grip and underarm pad may be cleaned with mild soap or detergent. **DO NOT** use organic solvents, such as acetone, which may damage grip.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check hardware and positioning weekly for tightness.
- Replace crutch tips as needed.
- To guarantee the best service from your crutches, always specify Lumex replacement parts, available from your GF Health Products, Inc. distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ALUMINUM CRUTCHES SIZE

Selection Chart

Item Number	Size	User Height
3614A	Adult	5' 2" – 5' 10"
3615A	Tall	5' 10" – 6' 6"

ADJUSTING HANDLE POSITION

1. Remove the wing nut and bolt securing the handle to the crutch.
2. Align the handle with the desired set of adjustment holes.
3. Secure with wing nut and bolt.
4. Repeat for other crutch.

HEIGHT ADJUSTMENT

1. Use double push buttons in center tube to adjust height in 1" increments.

Caution: Be sure that push buttons are fully protruding through the same respective adjustment hole of each leg extension. This ensures that the adjustable legs are securely locked into position and an even height adjustment is obtained.

LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE


This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling **(678) 291-3207**, sending a fax request to **(770) 368-2386** or by e-mailing a request to **cs@grahamfield.com**. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: **1)** Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; **2)** Products not installed, used, or properly cleaned and maintained; and **3)** Labor or shipping charges. Other exclusions apply. Additional limitations relative to this warranty may be found on www.grahamfield.com Terms and Conditions page. State laws governing the warranty of this product may provide more expanded rights than those set forth in this warranty.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

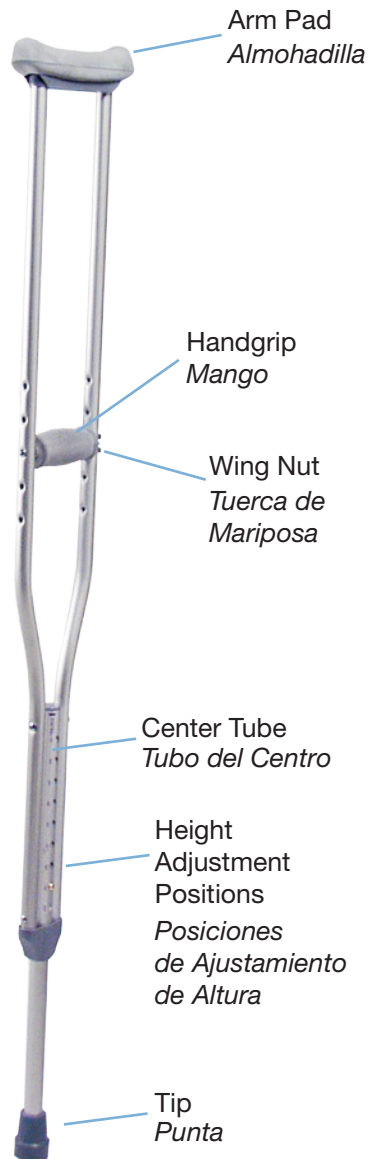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Features

- Heavy duty steel construction supports larger user
- Reinforced center tube gives a stronger support to secure and maintain proper balance and stability
- Easy height and handle adjustment lets you adjust to best fit your needs
- Padded underarm pads and grips provide you with a comfortable and durable fit
- Maximum Weight Capacity 600 lb - EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2017, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com