LUMEX[®]

by graham-field

UpRise Onyx Folding Walker

Andador Plegable UpRise Ónix

Item 700175C-2

graham
field

SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. Each individual should **ALWAYS** consult with their physician or other health care professional to determine proper adjustment and usage.
2. Before use, be sure that the walker is in its open and locked position.
3. All legs **MUST** be in contact with the floor at all times during use. This will ensure the walker is properly balanced.
4. This product has a maximum user weight limitation of 400 lbs - **EVENLY DISTRIBUTED. ALWAYS** observe the user weight limitation on the hangtag.
5. Ensure handgrips **DO NOT** twist on walker – otherwise damage or injury may occur.
6. **ALWAYS** test to see that the walker and attachments are properly and securely locked in the open position **BEFORE** using.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. **ALWAYS** use caution when using the walker on wet and slippery surfaces. **DO NOT** use the walker on stairs and steep inclines.
9. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use **EXTREME** caution when reaching for any object.
10. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
11. Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.

GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure, and that wheels (if applicable) and moving objects are in good working order before using this walker.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov

INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

STABILITY WARNINGS

Be careful when reaching for objects when using the walker. Reaching for objects can change the weight distribution of the walker and may cause the walker to tip, resulting in injury or damage.

MAINTENANCE

- For dust and dirt, wipe walker gently with damp, soft cloth.
- Inspect wheels (if applicable) periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the walker.
- Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ASSEMBLY OF THE WALKER

1. Remove closed walker from carton.
2. Install legs by depressing the height adjustment button and slide onto walker frame until the button engages in the desired adjustment hole. Adjust all leg attachments to the same height.

WARNING: Ensure all push buttons protrude fully from height -adjustment holes, all legs are adjusted to the same height, and wheeled leg attachments are positioned on the front and towards the outside of the walker.

3. To open walker, push handgrips and sides out away from each other until the release mechanisms lock into place.

CLOSING THE WALKER


1. Push downward on release button and hold.
2. Grab one handgrip and push inwards towards center of walker.
3. Repeat for other handgrip.

ADJUSTING THE HEIGHT

NOTE: Height adjustment should be made to desired height before operation. Consult with healthcare professional for proper adjustment.

1. To adjust the height of the leg attachment, depress the adjustment button and slide upwards or downwards to the desired height.
2. Adjust the height of the walker by aligning all four leg attachments to the same position height.

When walking, the user should grasp one handgrip in each hand, positioning body between the two rear walker legs, and walking at a comfortable pace. The rear walker legs should be in contact with the floor at all times.

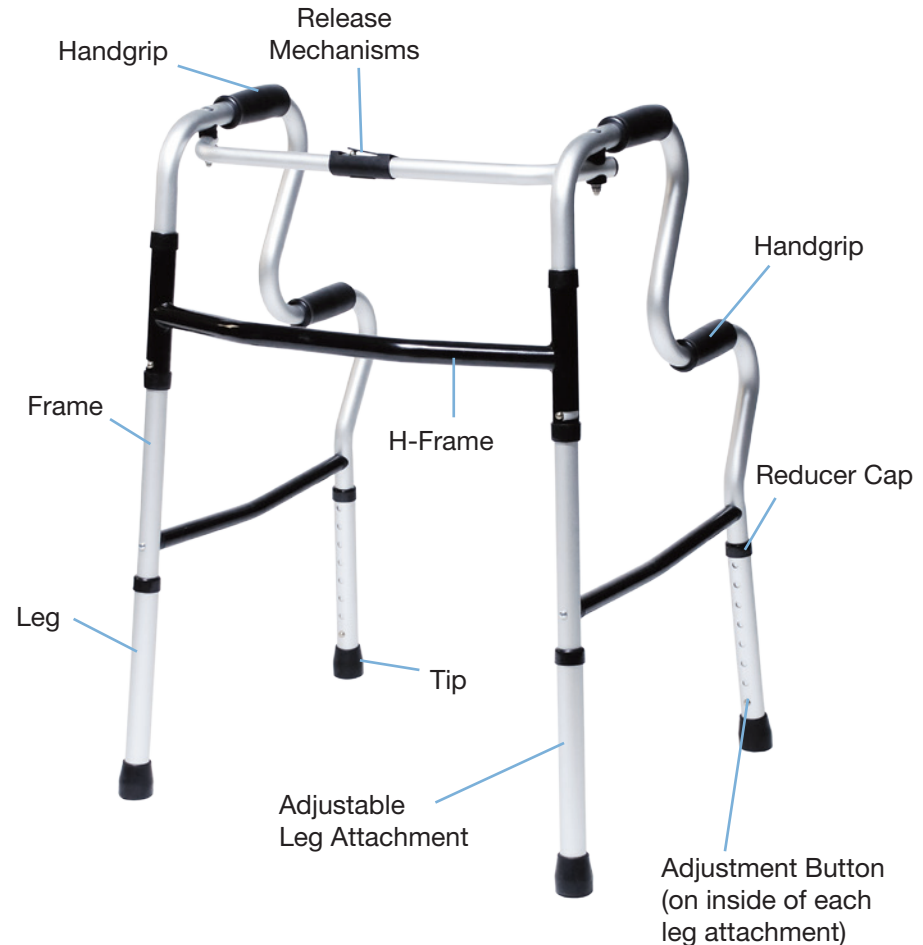LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling **(678) 291-3207**, sending a fax request to **(770) 368-2386** or by e-mailing a request to **cs@grahamfield.com**. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: **1)** Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; **2)** Products not installed, used, or properly cleaned and maintained; and **3)** Labor or shipping charges. Other exclusions apply. Additional limitations relative to this warranty may be found on www.grahamfield.com Terms and Conditions page. State laws governing the warranty of this product may provide more expanded rights than those set forth in this warranty.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

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Features

- Combination of folding walker and rising aid all in one
- Secondary handles provide stable assistance from a seated to a standing position
- Can also be used as a portable toilet safety frame
- 400 lb weight capacity - EVENLY DISTRIBUTED
- Adjustable height fits a broad range of user heights
- Attractive, upscale Onyx finish

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com

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- Si los mangos están sueltos, **NO LOS** use. Llame a su distribuidor autorizado de GF Health Products, Inc.
- Para el mejor servicio de su andador, siempre use partes de reemplazo de GF Health Products, Inc.