



Contents:

- 12 Bolts
- 12 Lock Nuts
- Lower Extension Leg Tubes
- Upper Extension Leg Tubes

Installation:

1. Remove casters from leg sockets of recliner by unscrewing nuts and bolts.
Note: Save them for use later when rebolting leg.
2. Insert the square end of the upper tube (A) into the lower tube (B) of extension leg until the first holes (C & CC) of each part line up. Place a screw and a bolt through hole (at C) and hand tighten.
Note: All bolts should face the same direction.
3. Push crimped end of assembled extension leg into open end of chair leg tapping gently with hammer or other blunt instrument if necessary to ensure that edges of chair leg and extension leg meet flush.
4. Fasten the extension leg onto the chair leg with the nut and bolt originally taken from the chair. Insert bolt through the bottom hole of the chair leg (D) and fasten with nut.
5. Place caster (E) into open end of the extension leg and secure in place with nut and bolt after checking that the bottom of the extension leg and the caster flange meet each other.
6. Tighten all nuts and bolts with two wrenches so that lock nuts are locked into position.
7. Repeat procedure with remaining legs.

