IMPORTANT SAFETY INFORMATION: PLEASE READ

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the commode. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the commode is not properly assembled and installed, personal injury and damage to the commode could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Gendron replacement parts. The use of non-Gendron replacement parts could cause personal injury and damage to the commode.**
- ⚠ **WARNING: This product has a maximum weight capacity of 750 lb (340.2 kg), EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Cancer and Reproductive Harm - www.p65warnings.ca.gov.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

523326 / 523330 Bariatric Commode

INTENDED USE

The Gendron 523326 / 523330 Bariatric Commode may be used as a bedside commode, raised toilet seat, and toilet safety frame. It is intended to provide assistance to those who have difficulty sitting down or standing up from the toilet or need additional support and stability during toileting.

SETUP (REFER TO PICTURE BELOW)

523326 / 523330 Bariatric Commode

1. Remove frame, pail, and cover from carton.
2. With frame in the upright position, place pail and cover in frame from the back.

ADJUSTING COMMODE HEIGHT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for the other three legs.

⚠ WARNING: Ensure all four commode legs are adjusted to the same height and all height adjustment buttons protrude fully through adjustment holes before use.

INSERT THE BACK SUPPORT

1. Remove back support from carton.
2. Insert back support into channel guides until back support locking buttons click into place in channel guide holes.

⚠ WARNING: Ensure that locking buttons protrude fully through holes before use.

USE COMMODE OVER TOILET

1. Set commode to desired height (described in previous steps).
2. Remove commode pail and lid.
3. Remove back support (described in previous steps).
4. Lift toilet bowl seat.
5. Place commode over toilet with opening positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat with disinfectant after each use.

USE AS BEDSIDE COMMODE

1. Set commode to desired height (described in previous steps).
2. Place commode pail on pail supports. Ensure that pail is properly positioned below commode opening.
3. When user is finished using commode, place commode pail lid over commode pail to contain odors.
4. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

MAINTENANCE

- ⚠ **WARNING: Inspect frequently to ensure that all attaching hardware is tight.**
- ⚠ **WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.**
- ▲ **NOTICE: Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.**

After each use: Dry with a soft cloth.

Cleaning: Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.

To guarantee the best service from your product, always specify Gendron replacement parts, available from your GF Health Products, Inc authorized distributor.

LIMITED WARRANTY

Scope of Warranty

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

Obtaining Warranty Service

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

Exclusions

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;

- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

Entire Warranty, Exclusive Remedy and Consequential Damages Disclaimer

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

Notes:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com / www.gendroninc.com

© 2022, GF Health Products, Inc. All Rights Reserved.
Gendron is a trademark of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485:2016 / MDSAP Certified Company.
Manufactured for GF Health Products, Inc.,
One Graham-Field Way, Atlanta GA 30340-3140
Made in U.S.A.

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© 2022, GF Health Products, Inc. Todos los derechos reservados.
GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. es una empresa certificada ISO 13485:2016 / MDSAP.
Fabricado para GF Health Products, Inc.
Fabricado en U.S.A.