

**Congratulations on your purchase of the Lumex Select Convertible Mattress. This guide covers the unpacking instructions. You will also find a User Manual inside the box with more details.**

1. Check for any obvious damage to the packing or its contents before opening. If damage is evident, DO NOT USE. Contact carrier / distributor for further instruction.
2. Remove mattress from the box.
3. Position the mattress near the bed base.
4. Use a pair of scissors and not a knife. A knife has a higher chance in slicing or damaging your mattress.
5. The mattress will start to unwrap. Be careful as sometimes it can start to expand rapidly.
6. Once unwrapped, the compressed mattress foam may take up to 48 hours to fully recover. Environmental factors such as cooler temperatures may cause expansion to take longer. Mattress will expand to + / -1" dimensions from the size mattress that you ordered.



+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)