LUMEX[®]

by graham-field

Shower Transport Chair

Silla de Transporte de Ducha

Items 7910A-1 & 7915A-1

graham
field

QTY. 1 EA

SAFETY GUIDELINES - PLEASE READ BEFORE USE

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- This product has a maximum weight limit of 250 lb
- EVENLY DISTRIBUTED.
- Users with limited physical capabilities should be supervised or assisted when using this product.
- Casters **MUST** be in contact with the floor at all times during use.
- Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.
- This product should not be used without proper instruction from a healthcare professional.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ATTACHING THE BACKREST (7910A-1 & 7915A-1)

1. Remove chair and back support from the box. Place chair on a flat surface.
2. Lock rear casters to prevent chair from moving during assembly.
3. Locate the channel guides (2 each) on the back side of the chair frame near the seat. Align the backrest support tubes with the channel guides.
4. Depress the push buttons on the back support tubing and slide the tubing through the channel guides until the push buttons protrude fully through the channel guide holes.

ATTACHING THE FOOTRESTS (ONLY FOR 7915A-1)

1. Locate the footrests (2 each).
2. Insert 1 footrest into the chair leg extension.
3. Ensure footplate is able to fold down to the inside.
4. Move footrest upward or downward to desired length, ensuring alignment holes match with holes in chair leg extension.
5. Once the holes are aligned, insert a screw into the hole, rotating the screw clockwise. Repeat this step for the second screw.
6. Use a screw driver to tighten the screws.
CAUTION: Do not over tighten.
7. Repeat these steps for the remaining footrest.

Back Support

Toilet Seat

Caster

Footrest Frame

LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling **(678) 291-3207**, sending a fax request to **(770) 368-2386** or by e-mailing a request to **cs@grahamfield.com**. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: **1)** Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; **2)** Products not installed, used, or properly cleaned and maintained; and **3)** Labor or shipping charges. Other exclusions apply. Additional limitations relative to this warranty may be found on www.grahamfield.com Terms and Conditions page. State laws governing the warranty of this product may provide more expanded rights than those set forth in this warranty.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

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7910A-1/7915A-1

- Designed for transport to and from the shower
- Anodized, non-corrosive aluminum frame construction will not rust
- 5" swivel casters with rear wheel locks facilitate maneuverability in tight spaces
- Rolls easily over most toilets
- Allows complete access to body for easy cleaning during bathing
- Maximum Weight Capacity 250 lb - EVENLY DISTRIBUTED

7915A-1 Only

- Powder coated steel front brace with footrests
- Foot plates fold up and out of the way for unobstructed entry and exit

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2005, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com