- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the Lumex Knee Walker. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the Knee Walker is not properly assembled, personal injury and damage to the Knee Walker could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Knee Walker.**
- ⚠ **WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Cancer and Reproductive Harm - [www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**
- ⚠ **WARNING: Consult your healthcare professional and GF distributor for assistance in developing and learning safe and effective techniques for performing your daily activities according to your individual physical abilities and needs, and to ensure the Knee Walker handle height and knee pad height are properly adjusted for your use.**
- ⚠ **WARNING: DO NOT lean or pull on the steering mechanism while using the Knee Walker; this could cause instability.**
- ⚠ **WARNING: DO NOT attempt to reach or retrieve objects while using the Knee Walker; this could cause instability.**
- ⚠ **WARNING: Do not use the Knee Walker on stairs or curbs. Personal injury and damage to the Knee Walker could result.**
- ⚠ **WARNING: DO NOT operate the Knee Walker on hilly or rough terrain, sand, wet or icy surfaces, or surfaces with impaired traction. Personal injury and damage to the Knee Walker could result.**
- ⚠ **WARNING: Before using the Knee Walker, you should be trained by a healthcare professional in its operation. When you begin using the Knee Walker, practice using it with an attendant, AT A SLOW PACE, until you are confident.**
- ⚠ **WARNING: Prolonged use of a knee walker can cause possible complications. Always consult your healthcare professional on the use of any mobility aid for extended periods of time.**
- ⚠ **WARNING: Knee walkers are often provided as rented medical equipment. If you are a provider or authorized distributor, you MUST perform routine maintenance and inspection to ensure the walker is ready to use for the next user. Failure to do so will void the warranty and could create a hazardous condition resulting in serious personal injury.**

**LX8000 Knee Walker**


## INTENDED USE

The Knee Walker is a mobility device intended for use by individuals who are unable to bear weight on the lower extremity of one leg. The device features a folding frame with height adjustable handle and knee pad to support the injured leg, while allowing the other leg to be used for propulsion.

**⚠ WARNING: DO NOT sit on the Knee Walker and self propel. DO NOT use the Knee Walker as a wheelchair or transport chair.**

## UNPACKING

1. Carefully remove all components, shown and listed below, from carton.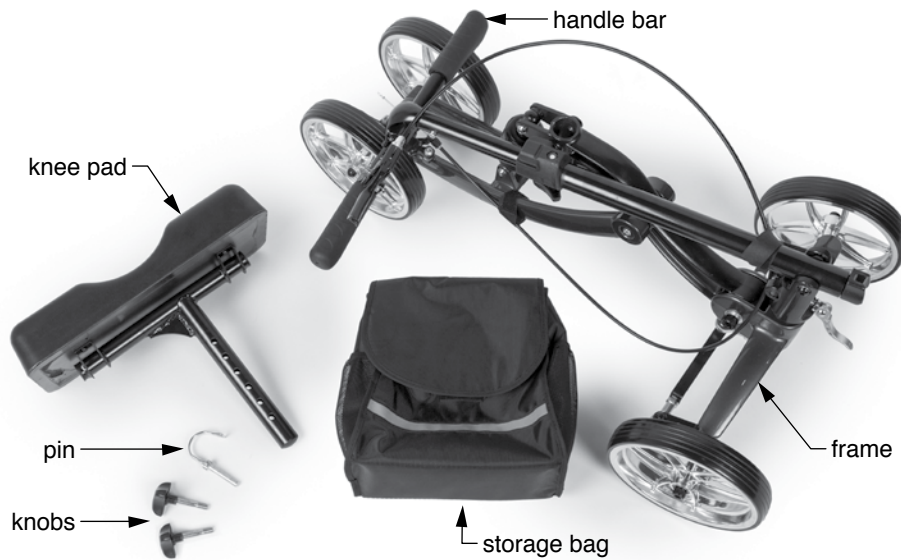

LX8000 Knee Walker components

Item	Quantity	Description
1	1	Handle Bar
2	1	Frame with Wheels
3	1	Knee Pad


Item	Quantity	Description
4	2	Knob
5	1	Pin
6	1	Storage Bag

## ASSEMBLY (REFER TO PICTURES)

### Secure handle bar

1. Rotate the handle bar up to vertical position as shown at right.
2. Rotate the handle bar lock clip up to lock it as shown at far right.
3. If the handle bar is wobbly and insecure in the base, open the lock clip and turn it clockwise to tighten it; if the handle bar is too tight, open the lock clip and turn it counterclockwise to loosen it.
4. Close the lock clip to secure handle bar.


**⚠ WARNING: Ensure the lock clip is locked and the handle bar is secure before proceeding.**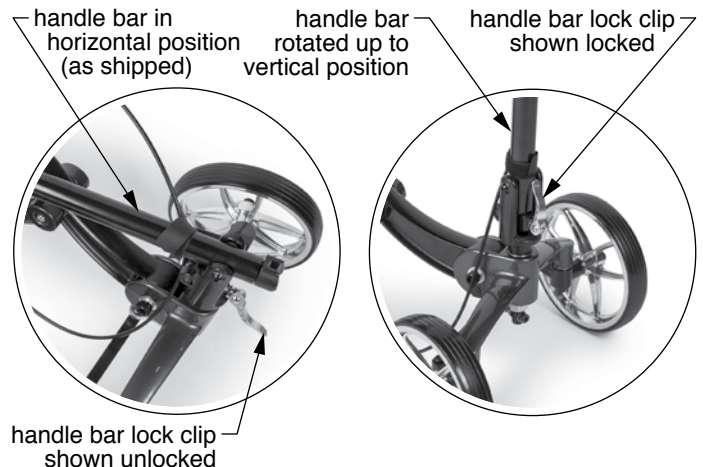


Secure handle bar

## Install knee pad

1. Insert the knee pad post into the frame receptacle; the knee pad post has a series of height adjustment holes, shown at right, that align with the frame receptacle holes, to provide three height positions. Choose the height adjustment holes that will provide the desired knee pad height before proceeding.
2. Ensure holes are aligned, and insert the knob shank completely through both the knee pad post holes and the frame receptacle holes.
3. Turn the knob clockwise until fully tightened.
4. Insert the locking pin completely through both the knee pad post holes and the frame receptacle holes until the pin's shoulder rests against the knee pad post.
5. Lock the pin in place with the wire lock by pivoting the wire lock around the back of the knee pad post and clipping it over the end of the pin, as shown at right.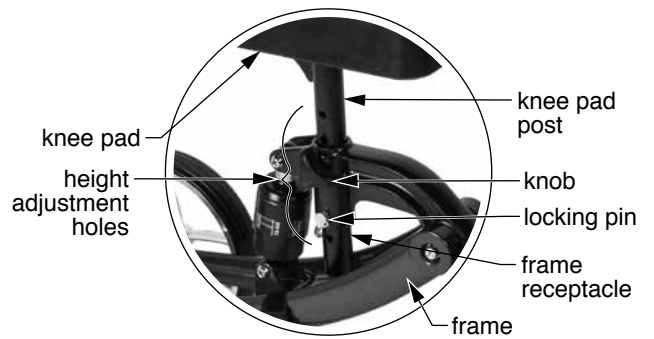

Install knee pad

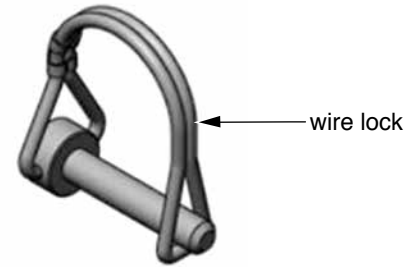
Locking pin

**⚠ WARNING:** Ensure both the entire knob shank and entire pin length completely penetrate the entire diameter of both frame receptacle and knee pad post, the locking pin is locked in place, and the knee pad is securely locked into the frame receptacle before proceeding.

## Adjust handle bar height

*Info: Consult your healthcare professional to determine proper handle bar height adjustment. We recommend adjusting handle bar height while standing, with the good leg straight and the injured leg resting in the center of the knee pad.*

1. Unscrew the two-piece knob to release the handle bar assembly from the steering column.
2. Place the handle bars at the desired height inside in the steering column, choosing the height adjustment hole that will provide the desired handle bar height before proceeding.
3. Ensure holes are aligned. Install the knob through the handle bar assembly and steering column holes into the back piece of the knob.
4. Turn knob clockwise until fully tightened.

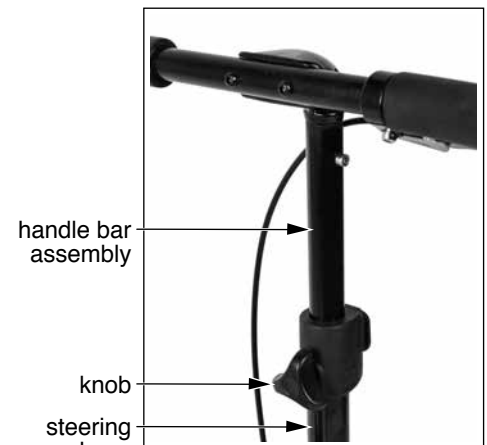Adjust handle bar height

## Install storage bag

1. Install storage bag in front of handle bars, as shown at right, with pouch facing forward and handle at top.
2. Secure storage bag securely with the three hook and loop straps around handle bar shaft and both handle bars as shown at right.

Install storage bag

## Secure brake cable

Secure brake cable to the steering column in front and the frame at back of knee walker with the two hook and loop brake straps as shown at right.

▲ **NOTICE:** Ensure brake cable is secured but not too tightly bound to operate, using the two hook and loop brake straps provided.

Secure brake cables

## OPERATION

⚠ **WARNING:** DO NOT use the Knee Walker if the brakes are not working properly.

⚠ **WARNING:** Always be aware of your surroundings and watch where you're going while using knee walker.


⚠ **WARNING:** Always lock the brakes before attempting to transfer to or from the knee walker.

⚠ **WARNING:** ONLY one person should operate the knee walker at a time.

## Using the rear-wheel brakes

**Temporarily apply brakes:** use your fingertips to squeeze brake lever to right handgrip. To release brakes: release fingers.

**Lock brakes:** pull the brake lever toward you and push the locking pin on top of the lever down — the pin will lock the wheels in place. To release brakes: squeeze the lever to disengage the pin and unlock the wheels.

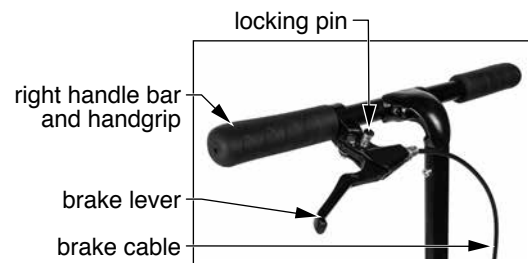Using the brake

## Operating the Knee Walker

⚠ **WARNING:** Before using the Knee Walker, ensure knee pad and handle bars are at a comfortable height for you, and adjust if not.

⚠ **WARNING:** Before using the Knee Walker, practice using it with an attendant until you are confident. Ensure you are able to operate the brakes before beginning.

1. Position the injured leg on the knee pad centered side to side with limb distributed evenly along length of knee pad. Ensure knee is resting entirely on knee pad.
2. Begin by using your propulsion leg for very small steps. As you gain confidence, increase your speed until it is safe and comfortable.

Operating the knee walker

## MAINTENANCE AND CLEANING

- To remove dust and dirt, wipe knee walker gently with a clean, soft, damp cloth.
- To clean, use a clean, soft cloth and a mild soap and water solution, rinse with clear water, then dry with a soft cloth.
- Inspect wheels periodically for wear or damage. Verify that wheels are free of hair or any other obstruction to free wheel operation. Inspect caster stems for assembly tightness.

▲ **NOTICE: DO NOT use solvents, abrasive cleaners, or scouring pads to clean this product.**

⚠ **WARNING: Ensure proper function of brakes before each use.**

⚠ **WARNING: If handgrips or brakes are loose, DO NOT USE KNEE WALKER. Contact your Graham-Field authorized distributor immediately for repair.**

⚠ **WARNING: Inspect the knee walker periodically for proper function and wear. Ensure locked handle height; check fit and tightness of all nuts, bolts and knobs. If any component is not in proper working order or appears worn, contact your Graham-Field authorized distributor immediately for repair and/or appropriate replacement parts.**

⚠ **WARNING: The use of non-Lumex replacement parts will void the warranty, and could create a hazardous condition resulting in serious personal injury.**

## SPECIFICATIONS, LX8000 KNEE WALKER

Dimensions	Handle Height	Minimum		36.7"
		Maximum		40.6"
	Knee Pad	Height	Minimum	17.5"
			Maximum	20.5"
		Length		14.2"
		Width		6.7"
	Unit overall	Length		40"
		Width		18"
Wheel Diameter		8"		
Maximum Weight Capacity		300 lb, EVENLY DISTRIBUTED		

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, the warranted components and time period are set forth below:

Frame:..... three years  
Brakes:..... six months  
Knee Pad:..... six months  
Wheels:..... six months

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

1.770.368.4700

Information contained herein is subject to change.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

© 2019, GF Health Products, Inc. All Rights Reserved.  
Lumex is a trademark of GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140

GF Health Products, Inc. is an ISO 13485 Certified Company.

Manufactured for GF Health Products, Inc.  
Made in China

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© 2019, GF Health Products, Inc. Todos los derechos reservados.
Lumex es una marca de GF Health Products, Inc.
GF Health Products, Inc. es una empresa certificada ISO 13485.
Fabricado para GF Health Products, Inc.