**SAVE THESE INSTRUCTIONS FOR FUTURE USE**

**CONTENTS**

INTENDED USE.....	1	OPERATION .....	5
SAFETY GUIDELINES—READ BEFORE USE .....	1	CLEANING AND MAINTENANCE .....	6
UNPACKING .....	2	SPECIFICATIONS.....	7
ASSEMBLY .....	3	LIMITED WARRANTY .....	8

**INTENDED USE**

The intended use of the RJ4700 / RJ4718 Set N'Go Four-Wheel Rollator is as a Rollator, a walking aid with wheels, manipulated by both arms.

**⚠ WARNING: This product has a maximum weight capacity of:**  
RJ4700: 300 lb (136 kg), **EVENLY DISTRIBUTED.**  
RJ4718: 350 lb (158 kg), **EVENLY DISTRIBUTED.**

**⚠ WARNING: DO NOT use this rollator as a wheelchair or transport device. Rollators ARE NOT intended to be propelled while seated.**

**SAFETY GUIDELINES—READ BEFORE USE**

Note the following special statements, used throughout this manual, and their significance:

**⚠ WARNING: Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.**

**⚠ CAUTION: Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury.**

**▲ NOTICE: Indicates a potential hazard or unsafe practice that, if not avoided, could result in product/property damage.**

***Info: Provides application recommendations or other useful information to ensure that you get the most from your product.***

**⚠ WARNING: Important! Read and understand these instructions before assembling or using the Set N'Go Rollator. Ensure that the Rollator is assembled as described in this document before use. If the Rollator is not properly assembled and adjusted, personal injury and/or damage to the Rollator could result.**

**⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.**

**⚠ WARNING: If components are damaged or missing, contact your Graham-Field<sup>®</sup> authorized distributor immediately. DO NOT use substitute parts.**


**⚠ WARNING: GF Health Products, Inc. (“GF”) assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

**⚠ WARNING: To ensure the Rollator is properly balanced, all wheels must be in contact with the floor at all times during use.**

**⚠ WARNING: Always use caution when using the Rollator on wet or slippery surfaces. DO NOT use the Rollator on stairs or steep inclines.**

**⚠ WARNING: DO NOT hang anything from the Rollator frame. Place items in the pouch.**

## UNPACKING

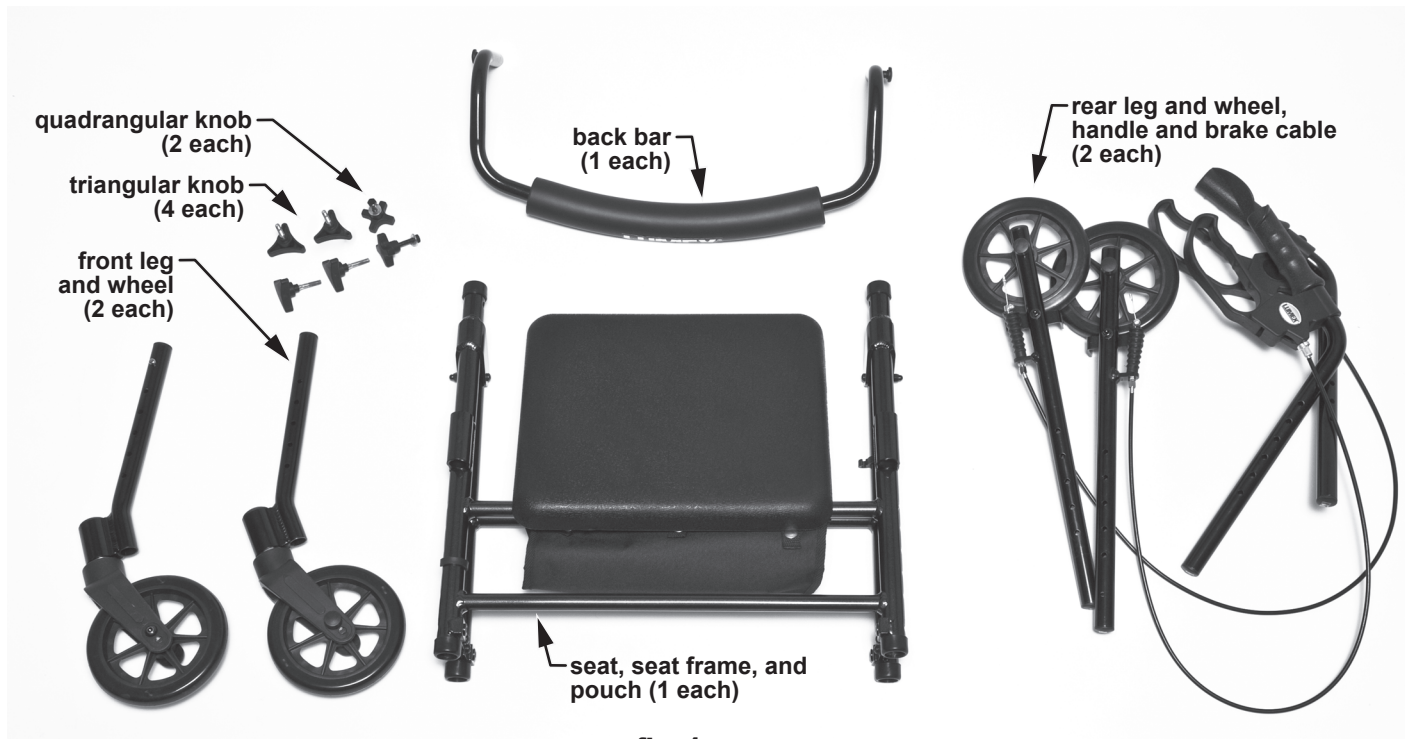**fig. 1**  
**RJ4700 / RJ4718 components**

Unless the Rollator is to be used immediately, retain box and packaging materials for storage until use is required.

1. Carefully remove all components from the carton.
2. Inspect all components (shown above in fig. 1).
3. If damage is evident to the contents, please notify the carrier and your Graham-Field authorized distributor.

## ASSEMBLY

Refer to pictures at right during assembly.

**Info:** As shown in fig. 2, each frame leg tube has a number corresponding to one of the four legs. Match each number at upper end of leg to each number on frame tube.

**Info:** Consult your healthcare professional to determine proper seat height adjustment.

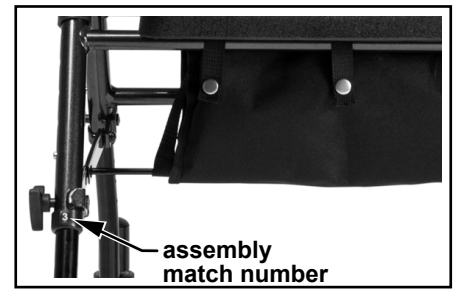
fig. 2

### Install the Legs

1. Lay the Rollator on its side to install the legs.
2. Install both front legs: Match leg number to frame tube number and insert leg into tube until locking button pops into place at appropriate seat height.
3. Install both rear legs: Match leg number to frame tube number and insert leg into tube until locking button pops into place at appropriate seat height.
4. Install triangular locking knobs to secure all legs: Insert knob into hole as shown in fig. 3 and turn clockwise until tight.

**⚠ WARNING:** Ensure all four legs are locked in place at the same height, and leg knobs are tight, before proceeding.

### Open the Rollator

1. Position Rollator upright on wheels.
2. Open Rollator frame; spread front and rear legs apart.
3. Push down on both side crossbars so that locking mechanisms lock legs open.
4. Fold seat down so that it rests on both seat frame tubes.

**⚠ WARNING:** Ensure both side crossbar locking mechanisms are locked in place before proceeding.

fig. 3

## Install the Handles

Install both handles in frame handle receptacles as shown in fig. 4.

## Adjust the Handle Height

**Info: Consult your healthcare professional to determine proper handle height adjustment.**

1. Install the height adjustment knob in the appropriate adjustment hole with knob facing outward; install nut on knob and tighten. Repeat for the other side. Ensure both handles are adjusted to the same height.

**Info: After adjusting handle height, the brakes must be checked and, if necessary, adjusted by a Graham-Field authorized distributor.**

2. Secure both brake cables onto frame with brake cable clips as shown in fig. 4.

**⚠ WARNING: Ensure both handles are adjusted to the same height, handle knobs are tight, and brake cables are secured in clips before proceeding.**

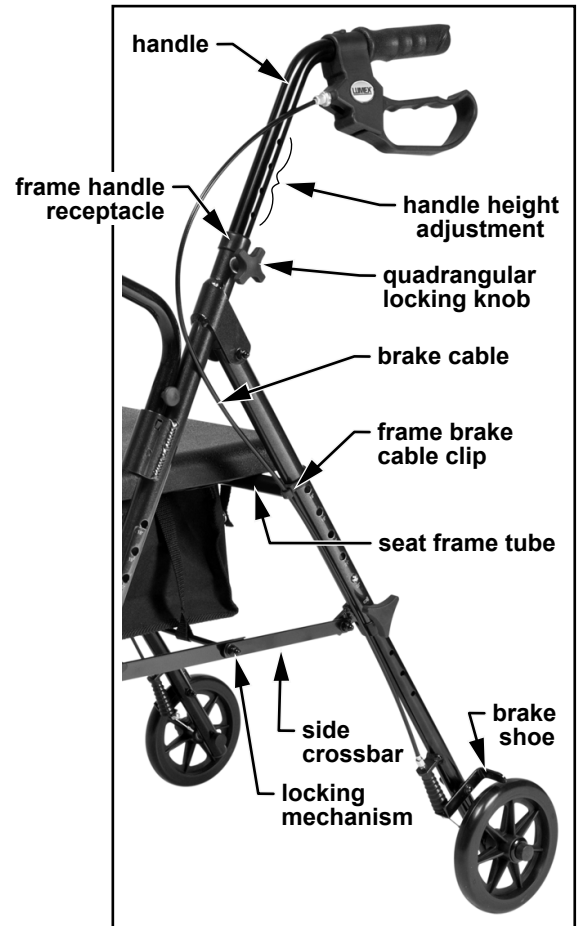
fig. 4

## Secure the Pouch

Fasten each snap on front and back of rollator to secure the pouch to the seat frame tubes as shown in fig. 5.

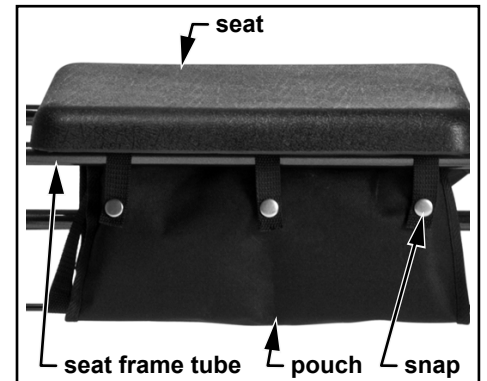
fig. 5

## Install the Back Bar

Install both ends of back bar into frame back bar receptacles as shown in fig. 6; push down until both locking buttons snap into place.

**⚠ WARNING: Ensure both ends of back bar are locked in place before proceeding.**

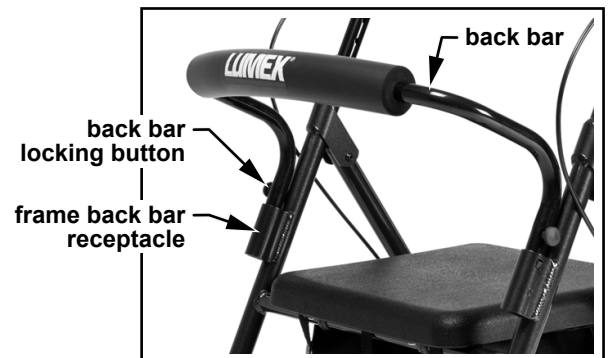
fig. 6

## OPERATION

- ⚠ **WARNING: BEFORE EACH USE**, ensure all locking buttons protrude fully from holes, all knobs are tight, the pouch is securely attached, and all attaching hardware is securely tightened.
- ⚠ **WARNING: Ensure that hand brakes stop Rollator and lock in place appropriately when operating or sitting on the Rollator.** If your hand brakes are not operating correctly, please see your Graham-Field authorized distributor immediately for service.
- ⚠ **WARNING: The Rollator pouch has a maximum weight capacity of 10 lb (4.5 kg).** Ensure items placed in the pouch do not protrude from the pouch.
- ⚠ **WARNING: Ensure back bar is always in place during use. DO NOT use the back bar to support user's entire body weight.**

### Walking With Your Rollator

Stand behind your Rollator with a hand placed securely on each handgrip. Move your Rollator slowly forward and select a pace that is comfortable for you. Always be sure the rear wheels are a few inches ahead of your body.

### Stopping With Your Rollator

The Rollator has loop locking hand brakes. To slow down, squeeze the hand brakes upward toward the handgrips. To hold the Rollator in a stationary position, push the hand brakes down, away from the handgrips, until the hand brakes lock into place; the rear wheels will be locked in place in parked position. To release, pull the hand brakes upward toward the handles until the rear wheels move freely.

### To Sit on Rollator Rest Seat

1. Push the hand brakes down, away from the handles, until the hand brakes lock into parked position.
  2. Test the wheels before sitting down to ensure the Rollator will not roll.
  3. Before sitting, touch the back of your legs to the edge of the seat to ensure proper positioning of the Rollator.
  4. Hold onto handles if necessary to help you lower your body onto the seat.
- ⚠ **WARNING: Ensure the brakes are in the locked position before using the rest seat.**
  - ⚠ **WARNING: Distribute your body weight evenly on the Rollator rest seat. DO NOT lean on one side of the Rollator.**
  - ⚠ **WARNING: Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place both feet securely on the floor. Use extreme caution when reaching for any object.**
  - ⚠ **WARNING: DO NOT use this rollator as a wheelchair or transport device. Rollators ARE NOT intended to be propelled while seated.**

**fig. 7**  
**RJ4700 Set N' Go Rollator**

## Folding the Rollator for Transport or Storage

1. Remove accessories, if any.
2. Lift the seat, move the pouch out of the way, and pull up on the folding pull strap attached to crossbar underneath seat.
3. Rollator is ready for transport and storage after folding. You may elect to remove handles before storage. The folding strap can be used to keep rollator frame folded for storage and/or transport.

## CLEANING AND MAINTENANCE

- To remove dust and dirt, wipe Rollator gently with a clean, soft, damp cloth.
- Inspect wheels periodically for wear or damage. Verify that wheels are free of hair or any other obstruction to free wheel operation. Inspect caster stems for assembly tightness. Use a clean, soft cloth and a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.

**▲ NOTICE: DO NOT use solvents, abrasive cleaners, or scouring pads to clean this product.**

**⚠ WARNING: Inspect the Rollator weekly for proper function and wear. Check fit and tightness of all nuts, bolts and knobs weekly. If any component is not in proper working order or appears worn, contact your Graham-Field authorized distributor immediately for repair and/or appropriate replacement parts.**

**⚠ WARNING: If handgrips or hand brakes are loose, DO NOT USE ROLLATOR. Contact your Graham-Field authorized distributor immediately for repair.**

**⚠ WARNING: The use of non-Lumex replacement parts will void the warranty, and could create a hazardous condition resulting in serious personal injury.**

## SPECIFICATIONS

		Model	4700 Set N'Go	4718 Set N'Go Wide
<b>Dimensions</b>	Width	Inside Handgrips	16.3" (414 mm)	19" (483 mm)
		Overall	24.5" (622 mm)	27" (686 mm)
	Depth	At Minimum Seat Height	25" (635 mm)	27.5" (699 mm)
		At Maximum Seat Height	29" (737 mm)	31.5" (800 mm)
	Height	At Minimum Handle Height	29" (737 mm)	32" (813 mm)
		At Maximum Handle Height	38" (965 mm)	38" (965 mm)
	Seat	Height	18" - 21.5" (457 mm - 546 mm)	19.5" - 23.5" (495 mm - 597 mm)
		Width	14" (356 mm)	15.75" (400 mm)
		Depth	12" (305 mm)	12.4" (315 mm)
	Casters	Diameter	6" (152 mm)	7.5" (191 mm)
<b>Weight</b>	Maximum Weight Capacity, EVENLY DISTRIBUTED		300 lb (136 kg)	350 lb (158 kg)
	Product Weight		14 lb (6.4 kg)	16.5 lb (7.5 kg)

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

Frame:.....tres (3) years

All other components:.....six (6) months

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

Customers located in the United States who wish to report a warranty issue, must contact GF directly by calling 1.770.368.4700 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Customers located outside the United States must contact the Distributor from whom they purchased the products. In both cases, further directions will be provided once the initial contact is made. This limited warranty shall only apply to defects that are reported within the applicable warranty period. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: [www.grahamfield.com](http://www.grahamfield.com).
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

**LUMEX**<sup>®</sup>  
by graham-field

Manufactured For:

GF Health Products, Inc.  
One Graham-Field Way, Atlanta, GA 30340  
Made in China

© 2011, GF Health Products, Inc. All Rights Reserved.  
Lumex is a registered trademark of GF Health Products, Inc.

LX\_RJ4700-INS-LAB-RevC23

+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

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© 2011, GF Health Products, Inc. Todos los derechos reservados.
Graham-Field y Lumex son marcas de GF Health Products, Inc.