**SAVE THESE INSTRUCTIONS FOR FUTURE USE**

**IMPORTANT SAFETY INFORMATION: PLEASE READ**

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the Bath Seat. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Bath Seat is not properly assembled and adjusted, personal injury and damage to the Bath Seat could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF Health Products, Inc. ("Graham-Field") distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Bath Seat.**
- ⚠ **WARNING: GF Health Products, Inc. ("GF") assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

**INTENDED USE**

The Bath Seat is intended for use by individuals who require seating support while bathing or showering. It is designed to be placed fully inside the bathtub or shower.

**Contraindications**

- ⚠ **WARNING: This product has a maximum weight capacity of 350 lb (159 kg), EVENLY DISTRIBUTED.**
- ⚠ **WARNING: DO NOT use this product as a transfer bench or climbing device.**
- ⚠ **WARNING: Use this product only with tub floors wider than 18 3/4".**

**ASSEMBLY**

Remove the Bath Seat from the box and place it on a flat, dry surface.

7921R / 7921KD-1

## Attaching the Legs

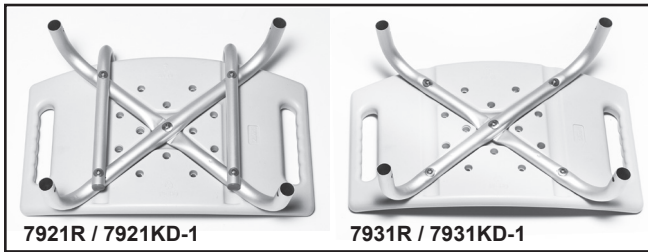
fig. 1

fig. 2

fig. 3

1. Locate the seat frame (see fig. 1); place it upside down on a flat, dry surface.
2. Locate the legs (see fig. 2).
3. Attach the legs to the seat frame (see fig. 3). Depress the locking button at the top of each leg and slide it into the seat frame until the button protrudes from the hole.
4. Repeat step 3 for each leg.

**⚠ WARNING: Ensure legs are securely installed and all hardware is tight before proceeding.**

## Attaching the Backrest: (Model 7921R and 7921KD Series Only)

fig. 4

fig. 5

1. Locate the backrest (see fig. 4).
2. Attach the backrest to the seat frame (see fig. 5). Depress the locking buttons and slide the backrest onto the seat frame until the buttons protrude from the holes.

**⚠ WARNING: Ensure backrest is securely installed and all hardware is tight before proceeding.**

## All Models

**⚠ WARNING: Ensure the Bath Seat is assembled as described. If the Bath Seat is not properly assembled and installed, personal injury and damage to the Bath Seat could result.**

## OPERATION

See assembled Bath Seat at right. Contact your GF distributor immediately if you see any issue with fit, form, function, or safety. For more information on GF Health Products, Inc. parts and services, please contact your GF distributor or [www.grahamfield.com](http://www.grahamfield.com).

### Adjusting Leg Height

Push in the adjustment button and move to the desired height position until it clicks into place. Ensure legs are locked at the same height and all leg adjustment buttons protrude fully through adjustment holes.

**⚠ WARNING: Ensure all Bath Seat components are secure before each use.**

### Center of Balance During Use

The point where the Bath Seat will tip forward, rearward or sideward is determined by its center of balance and stability, and the center of balance and stability is affected by the user's body position and weight distribution. If you fail to heed the following warnings, falling or tipping may occur and could cause severe personal injury.

**⚠ WARNING: To avoid falling or tipping:**

**Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping.**

**Do not put downward weight on one side or corner of the Bath Seat.**

**Do not put downward weight on the backrest.**

**Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach.**

**Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat.**

**Never lean backward to reach an object.**

## MAINTENANCE

**⚠ WARNING: Check the Bath Seat weekly for signs of damage or cracking. If damage or cracking of any component of the Bath Seat is found, DO NOT use the Bath Seat. Contact your Graham-Field distributor.**

Inspect the Bath Seat weekly and ensure all components are secure.

To clean the Bath Seat, wash it with soap and water or a mild detergent, then rinse. Ensure the Bath Seat is rinsed well before use.

**fig. 6**

# LIMITED WARRANTY

## SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the Original Purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the Original Purchaser, to the extent permitted. Original Purchaser is one who purchases this product new and unused from GF or a Certified GF Dealer/Distributor.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

Frame.....	three (3) years
Seat/back, extension legs, and arms.....	six (6) months
Normal wear components such as tips, wheels/casters, and handgrips.....	three (3) months

\*Labor is not included in the warranty.

†The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the Original Purchaser, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

## OBTAINING WARRANTY SERVICE

Contact the Dealer/Distributor from whom the product was purchased. If there is not a Dealer/Distributor, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). The Customer Service Representative will provide specific directions. Failure to abide by the specific directions will result in denial of the warranty claim.

## EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

## ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES AND COUNTRIES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE INSTANCES, GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

## NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: [www.grahamfield.com](http://www.grahamfield.com).
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

Manufactured For:  
GF Health Products, Inc.  
One Graham-Field Way, Atlanta, GA 30340  
Made in China

+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

© 2005, GF Health Products, Inc. All Rights Reserved.  
Lumex is a registered trademark of GF Health Products, Inc.

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© 2005, GF Health Products, Inc. Todos los derechos reservados.
Graham-Field y Lumex son marcas de GF Health Products, Inc.