

Contents:

- (1) Partially assembled linen cart
- (4) Four caster wheels (2 locking, 2 non-locking) in brown box
- (2) Two handles
- (1) Linen cover (when proper model number is ordered)
- (6) Six 1/2" stainless steel screws

Assembly Instructions:

STEP 1

Remove partially assembled linen cart from shipping container/box.

IF LINEN CART IS SHIPPED COMPLETELY UNASSEMBLED, PLEASE SET UP AS FOLLOWS:

STEP 1A

Included with the sub-assembly are the required heights/vertical pipes. Install pipes onto the frames, securing with the 1/2" screws. The shelves should be assembled in the same order they are banded together.

ONCE ASSEMBLED, PROCEED TO THE FOLLOWING STEPS.

STEP 2

Casters:

Insert the casters into the female fittings and secure with 1/2" stainless steel screws.

STEP 3

Cart Cover:

Drape the linen cover over the cart frame ensuring that the buttons and Velcro opening corresponds with the snaps on the frame. Secure. The Velcro enclosure should allow the user to open and close the cover easily.

STEP 4

Handles:

Install the handles into the 4-Way and Cross fittings and secure with the additional 1/2" screws.



IN ORDER FOR LINEN CART TO FUNCTION PROPERLY AND TO AVOID PERSONAL INJURY, ENSURE ALL FASTENERS/SCREWS ARE SECURED. IN THE EVENT THIS PRODUCT IS NOT USED FOR ITS INTENDED PURPOSE OR NOT ASSEMBLED PROPERLY, GF HEALTH PRODUCTS, INC. (GRAHAM-FIELD) DOES NOT ASSUME RESPONSIBILITY.