LUMEX[®]

by graham-field

Adjustable Offset Cane with Nitrile Grip

Baston Ajustable Offset con Manga Nitrilo

Items 5941A, 5942A, 5943A, 5944A

graham
field

QTY. 1 EA

SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 250 lb. - **EVENLY DISTRIBUTED**. The product is **NOT** designed to support the total weight of individuals.
3. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
4. If components are damaged or missing, contact your distributor immediately. **DO NOT** use substitute parts.
5. Before use, ensure tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
6. Always use caution when using this product on wet or slippery surfaces.
7. Inspect product weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
8. Single push button height adjustment **MUST** be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. **ALWAYS** test to see that cane handle is properly and securely locked in place **BEFORE** using.
9. Make sure locking collet is securely tightened at all times.

GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

MAINTENANCE

- Wipe product weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent. **DO NOT** use organic solvents, such as acetone, which may damage grip.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

INSTRUCTIONS FOR USE

Please Read all Instructions Before Use.

This cane is designed for users with recommended heights ranging from 4' 10" to 6' 4". Please consult a healthcare professional for the proper height adjustment and usage.

HEIGHT ADJUSTMENT

1. Loosen locking collet by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height..

NOTE: *An audible "click" will be heard when push button securely protrudes through the adjustment hole.*

3. Secure locking collet by turning clockwise until tight.

Item Number	Color
5941A	Bronze
5942A	Black
5943A	Silver
5944A	Combo Pack - 2 of each

LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only (“Customer”) for:

Frame: three (3) years

Tips, Wheels/Casters, and Handgrips: three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing cs@grahamfield.com. Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

For additional information related to warranty claims, please visit GF's terms and conditions located on www.grahamfield.com

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Features

- Soft, contoured foam handle provides a comfortable, secure handgrip
- Easy height adjustment from 31" – 39" in 1" increments with locking mechanism for added security
- Attractive, designer finish
- 250 lb maximum weight capacity - EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2011, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com