**SAVE THESE INSTRUCTIONS FOR FUTURE USE**

You will need: Screwdriver, drill, bit and mounting hardware as described in Installation type table below. Optional: Level, electronic stud locator.

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using grab bars. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If you are unable to perform this installation as described, please contact a licensed professional for installation. If the grab bar is not properly installed, personal injury and property damage could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.**
- ⚠ **WARNING: This product has a maximum weight capacity of 250 lb (113 kg), EVENLY DISTRIBUTED.**
- ⚠ **WARNING: GF Health Products, Inc. ("GF") assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

**INSTALLATION**

**Installation location**

- ⚠ **WARNING: Ensure that there is no electrical wiring or plumbing at the chosen installation location.**
  - ⚠ **WARNING: At least two screws in each flange MUST be mounted securely in a wood stud. If your installation site features standard 16"-centered wood studs, mount 12" grab bars vertically, and 24" grab bars vertically or at an angle, to ensure both ends are securely mounted in wood studs. Wood studs are located on 16" centers in most residential construction, but your installation may differ.**
1. Choose a mounting location most convenient for user. Place grab bar on wall, **aligned with stud locations**, and mark 3 mounting hole locations for each flange. **Note: If mounting grab bar horizontally or vertically, you may wish to use a level.**
  2. Determine which holes will be used for sheet metal screws and which for toggle bolts, if any.

**Installation type**

Installation type and wall surface	Drill bit needed	Mounting hardware required
Wood stud, tile wall	1/8" bit & 1/4" masonry bit	Sheet metal screw (provided)
Wood stud, untiled wall	1/8" bit	Sheet metal screw (provided)
Drywall (no wood stud), tiled or untiled wall	1/2" masonry bit	Toggle bolt: #10 or 3/16" flat head x at least 2 1/2" long


## Drill

1. Choose the appropriate mounting method from a, b, or c.
  - a. Sheet metal screw mount (into wood stud) on tile wall:**  
See figure 3. Drill 1/4" diameter hole *only* through tile with masonry bit, then drill 1/8" diameter approximately 1 3/4" deep through drywall and into wood stud.
  - b. Sheet metal screw mount (into wood stud) on untiled wall:** See figure 3. Drill 1/8" diameter approximately 1 3/4" deep through outer wall, drywall, and into wood stud.
  - c. Toggle bolt mount (no wood stud) on tile or untiled wall:**  
See figure 4. Drill 1/2" diameter through outer wall or tile and drywall with masonry bit.
2. After all holes are drilled, wipe wall with damp cloth to remove plaster powder.
3. Seal any unused holes in wall: Use finger to apply, press in, and smooth tub and tile caulking.

## Mount

1. To seal, use finger to apply tub and tile caulking around installation edge of flange and outer edges of bolt or screw heads before installation.
2. Attach grab bar and rubber gaskets to wall using appropriate mounting hardware as indicated in Figures 3 and 4. Install first one flange, then the other. Do not tighten completely until both flanges are installed. If installing toggle bolts, install them before sheet metal screws.
  - a. Toggle bolt mount:** Insert toggle bolt through grab bar flange and gasket, then install toggle onto bolt and install bolt as specified in toggle bolt instructions.
  - b. Sheet metal screw mount:** Insert supplied sheet metal screw through grab bar flange and gasket into wall.
3. Use screwdriver to tighten all screws securely but do not overtighten, as this may damage tile or wall.


**⚠ WARNING: Ensure that the grab bar is installed as described before use. If the grab bar is not properly installed, personal injury and property damage could result.**

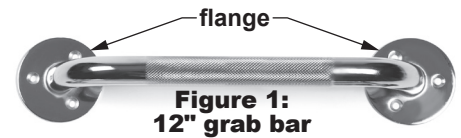
**Figure 1:**  
12" grab bar

**Figure 2:**  
flange (rubber gasket, mounted beneath flange, is not shown)

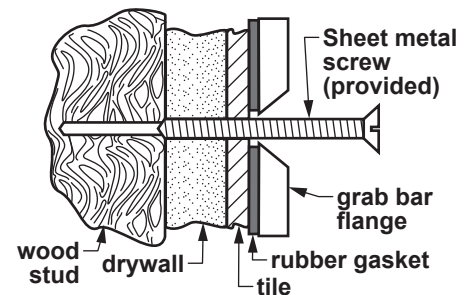**Figure 3:**  
sheet metal screw mount

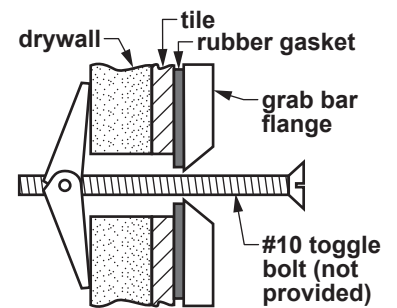**Figure 4:**  
toggle bolt mount

## OPERATION

**⚠ WARNING: Users who have limited physical capabilities must be assisted or supervised when using the grab bar.**

## MAINTENANCE

**⚠ WARNING: Inspect grab bar weekly and tighten as needed to ensure secure installation. DO NOT use grab bar if looseness is detected.**

To clean grab bar, wash it with soap and water or a mild detergent, then rinse it with water. Ensure that grab bar is rinsed well before use.

**▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the grab bar.**

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the Original Purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the Original Purchaser, to the extent permitted. Original Purchaser is one who purchases this product new and unused from GF or a Certified GF Dealer/Distributor.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

Grab Bar.....three (3) years

Hardware.....one (1) year

\*Labor is not included in the warranty.

†The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the Original Purchaser, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

Contact the Dealer/Distributor from whom the product was purchased. If there is not a Dealer/Distributor, you must contact GF directly by calling (770) 368-4700, sending a fax request to 770-368-4932 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). The Customer Service Representative will provide specific directions. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES AND COUNTRIES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE INSTANCES, GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document. For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: [www.grahamfield.com](http://www.grahamfield.com).
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

**LUMEX**<sup>®</sup>  
by graham-field

Manufactured For:

GF Health Products, Inc.  
One Graham-Field Way, Atlanta, GA 30340  
Made in China

© 2005, GF Health Products, Inc. All Rights Reserved.  
Lumex is a registered trademark of GF Health Products, Inc.

LX\_GF0500038-INS-LAB-RevE23

+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

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© 2005, GF Health Products, Inc. Todos los derechos reservados.
Graham-Field y Lumex son marcas de GF Health Products, Inc.