# LUMEX<sup>®</sup>

by graham-field

## **Walkabout Junior 4-Wheel Rollator**

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graham  
field

QTY. 1 EA

## SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. Intended usage: The rollator has been designed to assist people when walking.
2. Each individual should **ALWAYS** consult with their physician or other health care professional to determine proper adjustment and usage.
3. The brakes **MUST** be in the locked position before using the seat.
4. All wheels **MUST** be in contact with the floor at all times during use. This will ensure the rollator is properly balanced.
5. This product has a maximum user weight limitation of 300 lbs. **ALWAYS** observe the user weight limitation on the hangtag of this product and on the chart referenced at left.
6. Ensure handgrips **DO NOT** twist on rollator handle - otherwise damage or injury may occur.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. **ALWAYS** use caution when using the rollator on wet and slippery surfaces. **DO NOT** use the rollator on stairs, steep inclines or uneven terrain.
9. **DO NOT** hang anything from the frame of the rollator. Items should be placed in the pouch.
10. The rollator basket has a weight limitation of 11 lbs (5 kg). The tote bag has a weight limitation of 10 lbs (5 kg). Items placed in either the basket or the bag should not protrude from the basket or bag.
11. The backrest should **ALWAYS** be in place and is **NOT** designed to support the total body weight of the user.
12. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use **EXTREME** caution when reaching for any object.
13. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
14. Inspect rollator weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
15. If the hand brakes fail to stop the Rollator, contact your GF Health Products, Inc. authorized distributor immediately for adjustment or replacement.

## GENERAL WARNINGS

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- **DO NOT** use the rollator as a wheelchair or transport device. Rollators are not intended to be propelled while seated.
- If components are damaged or missing, contact your distributor immediately. **DO NOT** substitute parts.
- **DO NOT** use the rollator if there is any damage to the frame.
- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all hand and height adjustments are secure, and that wheels and moving objects are in good working order before using this rollator.
- When using the rollator in a stationary position, the hand brakes **MUST** be locked.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.
- **DO NOT** attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change to the weight distribution of the rollator and may cause the rollator to tip, resulting in injury or damage.

 **WARNING:** Cancer and Reproductive Harm-  
[www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).

## INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

## ATTACHING BASKET, POUCH OR TOTE BAG

### **Basket:**

Install the basket underneath the seat. On some rollator models, the basket is designed to fit underneath the seat or in the front by positioning the hooks on the front seat support frame.

### **Pouch and Tote Bag:**

Hang the tote bag between the rollator frame tubes and secure with snap button flaps.

## MAINTENANCE

- For dust and dirt, wipe product gently with damp, soft cloth.
- Inspect wheels periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Verify operation of the brakes and have them adjusted if necessary by your GF Health Products, Inc. authorized distributor.
- Inspect product weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.
- Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair or other debris that could interfere with free wheel operation.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## INSTRUCTIONS FOR USE

Please consult a healthcare professional for the proper adjustment.

### ASSEMBLING THE ROLLATOR

1. Remove Rollator and other components from carton.
2. Unfold frame; spread front and back legs apart.
3. Push down locking mechanism to secure legs in place.  
**NOTE:** Ensure the side crossbars (locking mechanism) lock down.
4. Insert handgrip handle bars into frame.  
**NOTE:** After installation, make sure brake cable is secured to frame with rollator clip.
5. Secure positioning of handle bars with adjustment knob.
6. Install backrest into frame and secure with push buttons.

### FOLDING THE ROLLATOR

1. If necessary, remove the accessories and lift the seat.
2. Hold one handle and pull up on the Folding Pull Strap that is attached to the crossbar (locking mechanism) or the pouch underneath the seat.

### USING THE HAND BRAKE

Brakes should be used in one of the following methods:

1. To Engage Brake: Push down on lower part of hand brake.
2. To Remain Stationary: Hold the hand brake up.
3. To Continue Mobility: Release the brake. To unlock, pull up on the top portion of the brake handle.

**CAUTION:** Always lock hand brakes when user is immobile.

### ADJUSTING THE HEIGHT

1. Remove the adjustment knob and bolt by turning counterclockwise.
2. Consult with healthcare professional for proper adjustment.
3. Install the adjustment knob into one of the adjustment holes by turning clockwise.
4. Repeat STEPS 1– 3 for the other side.

Make sure that the height adjustment is at the same height for both handles to ensure safety and proper use.

**NOTE:** After adjusting the height, the brakes of the rollator must be checked and, if necessary, adjusted.

## LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only ("Customer") for:

**Frame:** three (3) years

**Tips, Wheels/Casters, and Handgrips:** three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

*For additional information related to warranty claims, please visit GF's terms and conditions located on [www.grahamfield.com](http://www.grahamfield.com)*

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Rueda

## Features

- Improves mobility and allows for greater independence for the user on the go
- Smaller frame is ideal for children and petite users
- Lightweight design makes rollator easy to use, fold and store for convenient transport
- Padded seat and backrest provide additional comfort
- Ergonomic hand grips and hand brakes allow user to operate with ease
- Complete with pouch for storage of personal items
- 6" wheels allow for ease of movement
- 300 lb weight capacity - EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2009, GF Health Products, Inc.,  
One Graham-Field Way, Atlanta, Georgia 30340.  
Made in China. [www.grahamfield.com](http://www.grahamfield.com)