

- ⚠ **WARNING: Important! Read and understand these instructions before using the Grafco tourniquet. Keep these instructions for later use.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation or use of this product.**
1. When applying the tourniquet, place the opposite end from the label against the skin so the hook and loop fastener side is facing outward.
 2. Hold the tourniquet firmly with one hand and at the same time stretch the tourniquet tightly around the desired area.
 3. Attach the hook and loop fastener so the label is facing outward.

