LUMEX®

by graham-field

Silver Collection Steel Drop Arm Commode

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graham
field

QTY. 1 EA

SAFETY GUIDELINES - PLEASE READ BEFORE USE

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Users with limited physical capabilities should be supervised or assisted when using the commode.
- This product has a maximum weight capacity of 300lbs - **EVENLY DISTRIBUTED**.
- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.
- Leg extensions with rubber tips **MUST** be in contact with the floor at all times.
- Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.
- Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.
- Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them **IMMEDIATELY** if any of these conditions exist.
- The commode seat **MUST** be installed before sitting on the commode.
- **DO NOT** place weight on front of arm; doing so could cause the arm to drop unexpectedly. Place weight vertically on armrest **ONLY** (see figure 1).
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

SETUP

1. Remove frame, seat, lid, pail, cover, and splash shield from carton.
2. Attach seat and then lid to rear support of frame. Place splash shield or pail and cover in frame.

NOTE: When installing the seat onto the frame, make sure that the seat is evenly spaced between the side frames to ensure proper stability.

NOTE: Push down firmly until the seat snaps into place.

NOTE: DO NOT use excessive force when installing the seat and lid.

ADJUSTING LEG HEIGHT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.

NOTE: An audible “click” will be heard.

2. Repeat for other 3 legs.

CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

INSERTING BACK BAR

1. Remove back bar from carton.
2. Insert back bar into channel guides on back of commode until back bar push buttons protrude fully through channel guide holes (you will hear an audible click).

USING COMMUNE OVER TOILET

1. Set commode to desired height (described in previous steps).
2. Remove commode pail and lid.
3. Position splash shield in place onto pail supports.
4. Remove back support (described in previous steps).
5. Lift toilet bowl seat.
6. Place commode over toilet with splash shield positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat and splash guard with disinfectant after each use.

USING AS BEDSIDE COMMODE

1. Set commode to desired height (described in previous steps).
2. Lift commode seat and lid and place Commode pail onto pail supports. Ensure that pail is properly positioned below commode opening.
3. Place commode seat down. Commode is now ready for use.
4. When user is finished using commode, place commode pail lid over commode pail to contain odors.
5. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

ARM ADJUSTMENT

- **Drop Armrest:** to drop the armrest, push the release lever inwards to release the armrest. This releases the armrest allowing for the transfer on and off the commode.
- **Raise Armrest:** to raise the armrest, lift the armrest to the upright position, ensuring that the push button protrudes fully from locking hole.

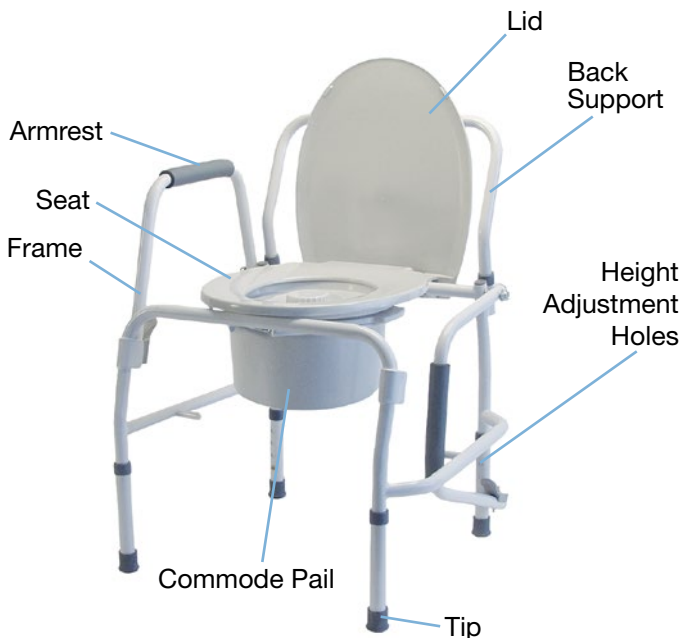
Figure 1

LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only ("Customer") for:

Frame: five (5) years

Tips, Wheels/Casters, and Handgrips: three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing cs@grahamfield.com. Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

For additional information related to warranty claims, please visit GF's terms and conditions located on www.grahamfield.com

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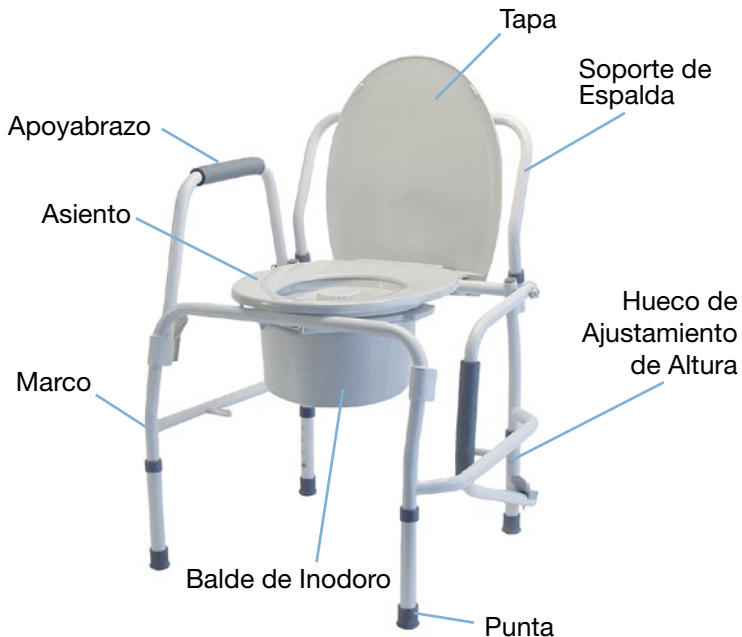
Figura 1

Features

- Arms drop independently below seat level for lateral transfers
- Easy-to-release drop arm mechanism may be operated by patients lacking manual dexterity
- Arms automatically lock in raised position
- Removable back bar facilitates use as a bedside commode, raised toilet seat or toilet safety frame
- Welded steel construction increases strength and stability
- Cushioned armrests enhance user comfort
- Easy to attach plastic seat [6433A, 6433A-2]
- Comes complete with commode pail, cover and splash shield
- 300 lb Weight Capacity EVENLY DISTRIBUTED
- Designed to meet the requirements of HCPCS code: E0165

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2005, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com