# LUMEX<sup>®</sup>

by graham-field

## Silver Collection Low Profile Quad Cane

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graham  
field

QTY. 1 EA

## SAFETY GUIDELINES - PLEASE READ BEFORE USE

1. This product should not be used without proper instruction from a healthcare professional.
2. This product has a maximum user weight limitation of 300 lb. - **EVENLY DISTRIBUTED**. The product is **NOT** designed to support the total weight of individuals.
3. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
4. If components are damaged or missing, contact your distributor immediately. **DO NOT** use substitute parts.
5. Before use, ensure tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
6. Always use caution when using this product on wet or slippery surfaces.
7. Inspect product weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
8. Single push button height adjustment **MUST** be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. **ALWAYS** test to see that cane handle is properly and securely locked in place **BEFORE** using.
9. Make sure locking collet is securely tightened at all times.
10. When using a Quad cane, the cane should be placed on a level walking surface with all four legs in contact with the surface. Use of cane in a fashion other than this can lead to eventual cane damage and/or personal injury.

## GENERAL WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** substitute parts.

## GENERAL WARNING (CONT.)

- Before use, ensure that all assembled parts are secure and all hardware is tightened.
- Care should be taken to ensure that all height adjustments are secure.
- **ALWAYS** observe the user weight limitation on the hangtag of this product.

 **WARNING:** Cancer and Reproductive Harm-  
[www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).

## INSTALLATION WARNINGS

- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Inspect product weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
- **ALWAYS** test to see that the product is properly secured in open position **BEFORE** use and ensure all hardware is securely tightened.

## MAINTENANCE

- Wipe product weekly with a damp cloth.
- Grips may be cleaned with mild soap or detergent. **DO NOT** use organic solvents, such as acetone, which may damage grip.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check locking collet weekly for tightness.
- Replace cane tip as needed.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## INSTRUCTIONS FOR USE

*Please Read all Instructions Before Use.*

This cane is designed for users with recommended heights ranging from 4' 10" to 6' 4". Please consult a healthcare professional for the proper height adjustment and usage.

## HEIGHT ADJUSTMENT

1. Remove locking knob by twisting counter-clockwise.
2. Adjust cane height: Depress push button and slide lower section upward or downward to desired height.

**NOTE:** *An audible "click" will be heard when push button securely protrudes through the adjustment hole.*

3. Secure locking knob by reinserting and turning clockwise til tight.

## ADJUSTING FOR RIGHT OR LEFT HANDED USE

1. Loosen locking knob by turning screw knob counter-clockwise.
2. Press the push button and turn cane handle 180° until push button locks in adjustment hole.
3. Secure locking knob by turning screw knob clockwise until tight. DO NOT over tighten.

Item Number	Large Base (12" x 8")
6140A	Silver
6140BK	Black
6140BZ	Bronze
Item Number	Small Base (8" x 6")
6141A	Silver
6141BK	Black
6141BZ	Bronze

## LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only (“Customer”) for:

**Frame:** three (3) years

**Tips, Wheels/Casters, and Handgrips:** three (3) months

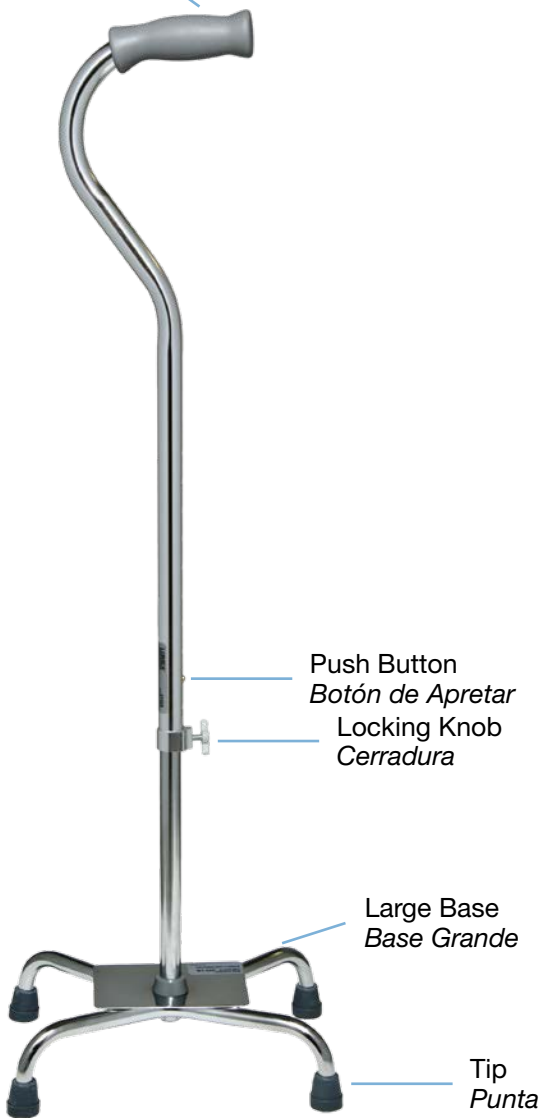
The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

*For additional information related to warranty claims, please visit GF's terms and conditions located on [www.grahamfield.com](http://www.grahamfield.com)*

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## Features

- Contoured handle provides a comfortable, secure handgrip
- Easy height adjustment from 29 1/4 - 38 1/4" in 1" increments with locking mechanism for added security
- Maximum weight capacity 300 lb - EVENLY DISTRIBUTED
- Designed for both left and right-handed use
- Meets the requirements for HCPCS E0105
- Lifetime Limited Warranty
- Latex free

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2012, GF Health Products, Inc.,  
One Graham-Field Way, Atlanta, Georgia 30340.  
Made in China. [www.grahamfield.com](http://www.grahamfield.com)