Transfer Bench

de Banco de Transferir

Items 7927KD-1 & 7927KD-2

SAFETY GUIDELINES - PLEASE READ BEFORE USE

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Users with limited physical capabilities should be supervised or assisted when using the transfer bench.
- Item has a maximum weight capacity of 350 lb. **EVENLY DISTRIBUTED.**
- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.
- Leg extensions with rubber tips **MUST** be in contact with the floor at all times.
- Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.
- Make certain **ALL** screws and nuts are tight at all times.
- Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them **IMMEDIATELY** if any of these conditions exist.
- The transfer bench seat **MUST** be installed before sitting on the transfer bench.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately.

 **WARNING:** Cancer and Reproductive Harm-
www.p65warnings.ca.gov.

MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

ATTACHING THE LEG EXTENSIONS (IF NOT ATTACHED ALREADY)

1. Place frame on its side on a level surface.
2. Attach leg extensions with suction feet to the frame on the same side as where the support arm will be. Attach leg extensions with flange tips to frame on opposite side of frame.
3. The transfer bench legs with suction feet should be the two legs that are to be placed inside the tub.

NOTE: Ensure seat support tubes buttons protrude fully through channel guide holes.

ATTACHING THE BACKREST

1. Determine the side on which the backrest should be mounted by knowing which direction the user will face when sitting on the bench in the tub.
2. Position the bench upright on a level surface.
3. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
4. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

NOTE: Ensure seat support tubes buttons protrude fully through channel guide holes.

CAUTION: Backrest is not designed to support the full weight of the user.

CAUTION: Ensure backrest is facing down correctly before inserting to avoid improper installation.

ATTACHING THE SUPPORT ARM

1. To attach the support arm, position the seat arm against the leg frame and slide the armrest tubing into the tube openings until it clicks into place. Make sure that the push buttons fully protrude through the holes in the bracket.

NOTE: An audible “click” will be heard.

2. To remove the support arm, depress the push buttons and pull up.

NOTE: The end with the Support Arm and Suction Feet is the side that should be inside the tub.

ADJUSTING LEG HEIGHT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for other 3 legs.

CAUTION: Ensure that all four legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

USING YOUR TRANSFER BENCH

NOTE: Be sure to move slowly throughout use to prevent injuries.

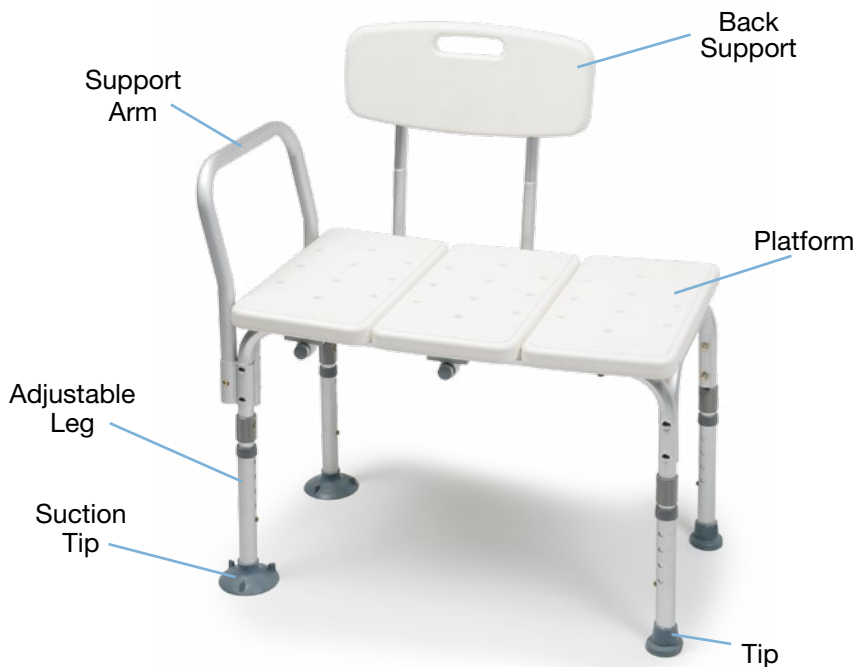
NOTE: Children or those who need assistance should NOT be left unattended during use of this product.

From a seated position (such as a wheelchair):

1. Place your wheelchair next to the bench, facing the front of the bathtub.
2. Engage wheelchair locks.
3. Transfer your torso to the transfer bench first, and then lift your legs into the tub. Move slowly to prevent injuries.
4. After bathing, lift your legs out of the tub first, then your torso.

From a standing position:

1. Sit on the side of the transfer bench that is outside of the tub.
2. Extend your arms to grab the support arm of the bench and slide your body over onto the bench. Lift each leg over the tub slowly.
3. Position yourself to a comfortable position in center of bench.
4. After bathing, lift your legs out of the tub first, then your torso slowly.

LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only ("Customer") for:

Frame: five (5) years

Tips, Wheels/Casters, and Handgrips: three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing cs@grahamfield.com. Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

For additional information related to warranty claims, please visit GF's terms and conditions located on www.grahamfield.com

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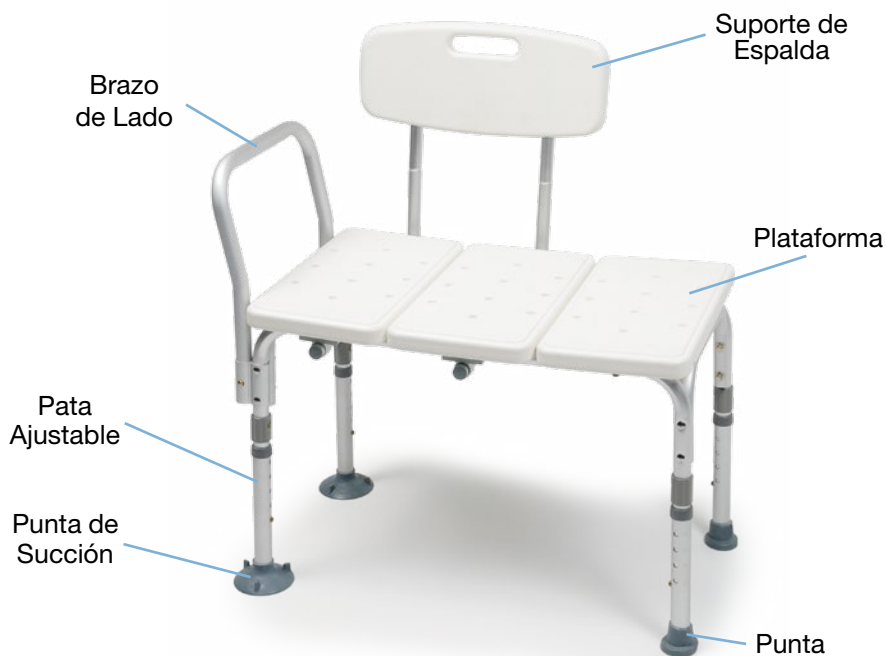
Features

- Provides assistance to those who have difficulty stepping over the bathtub walls
- Anodized aluminum frame is lightweight, durable, and rust resistant
- Drainage holes in seat minimizes water build up
- Reversible backrest to accommodate any bathroom
- Support Arm aids in lateral transfer
- Maximum weight capacity 350lbs EVENLY DISTRIBUTED
- Designed to meet the requirements of HCPCS code: E0247

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.,
One Graham-Field Way, Atlanta, Georgia 30340.
Made in China. www.grahamfield.com