# LUMEX<sup>®</sup>

by graham-field

## Drop Arm Versamode™

Versamode™ Braço de Caída

Item 6810A

graham  
field

QTY. 1 EA

## SAFETY GUIDELINES - PLEASE READ BEFORE USE

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Users with limited physical capabilities should be supervised or assisted when using the commode.
- This product has a maximum weight capacity of 300lbs - **EVENLY DISTRIBUTED**.
- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.
- Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.
- The commode seat **MUST** be installed before sitting on the commode.
- **DO NOT** place weight on front of arm; doing so could cause the arm to drop unexpectedly. Place weight vertically on armrest **ONLY** (see figure 1).
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately.

 **WARNING:** Cancer and Reproductive Harm-  
[www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).

## MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## ASSEMBLY

1. Carefully remove components from box.
2. Install back and arm assembly on base and seat frame assembly by inserting back tubes into base frame tubes. If necessary loosen, but do not remove, knobs.
3. Tighten knobs at rear seat until secure — do not overtighten.
4. Install footrests on base frame footrest brackets. Lock footrests into place.
5. Slide the commode pail in under the seat from the back

**NOTE:** Before use, ensure all components are securely installed and VersaMode is assembled as described above.

## DROP ARMREST

The VersaMode armrests drop to facilitate transfer. To drop the armrest, pull the release lever outward until the armrest releases, lift the armrest up out of the socket, then swing it downward.

## LOCK REAR CASTERS

The rear casters lock into place to prevent movement. To lock, step on the rear pedal until it latches. To unlock, step on the rear pedal until it unlatches.

## SWING AWAY / REMOVE FOOTRESTS

The VersaMode footrests swing away or detach to facilitate transfer. To swing the footrest away, depress the footrest release lever and swing it out of the way. To remove the footrest, swing it away, then lift it up off the brackets. To lock the footrest in place, swing it in toward the center of the VersaMode until it locks into place.

## LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only ("Customer") for:

**Frame:** five (5) years

**Tips, Wheels/Casters, and Handgrips:** three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF'S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

*For additional information related to warranty claims, please visit GF's terms and conditions located on [www.grahamfield.com](http://www.grahamfield.com)*

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## Features

- Arms drop independently below seat level for lateral transfers
- Cushioned armrests enhance user comfort
- Comes complete with commode pail, cover and splash shield
- 300 lb Weight Capacity **EVENLY DISTRIBUTED**
- Designed to meet the requirements of HCPCS code: E0165

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2005, GF Health Products, Inc.,  
One Graham-Field Way, Atlanta, Georgia 30340.  
Made in China. [www.grahamfield.com](http://www.grahamfield.com)